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62 Instructions of use: Handle the condom carefully, including removal from the package so as to avoid damage to the condom by fingernails, jewellery etc. The condom should be placed on the erect penis before any contact occurs between the penis and the partner's body to limit the prevention of sexually transmitted infections and pregnancy. Hold the reservoir between thumb and forefinger and remove any air remaining in the tip of the condom by squeezing it between thumb and forefinger. Unroll the condom over the penis. Stop and check if you feel the condom slipping or tightening excessively until the penis has been fully covered. If the condom is not fully on the penis before you begin to breathe, you may have to choose another condom size. Withdraw the penis soon after ejaculation, while holding the condom firmly in place at the base of the penis. If an additional lubricant is desired, use the correct type of lubricant which is recommended for use with condoms and avoid the use of oil-based lubricants, such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., as these are deleterious to the integrity of the condom. Consult a doctor or pharmacist about the compatibility of topical medicines that might come in contact with the condom. Seek medical assistance as soon as possible, at least within 72 h should a condom tear or burst during use. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package. Dispose the condom in a waste bin. Do not flush down the toilet. The condom is for single use only and if reused there might be an increased risk of failure or infection. The condom is made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if a user is allergic to latex. No contraceptive can guarantee 100% protection.

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R&S consumer goods GmbH
AMOR Gummiwaren GmbH | August-