3_{IN}1

STANDARD FIT

ANTI-SNURK SET

PRO

HANDLEIDING

NL

NAPSON

Anti-Snurk Pro Set

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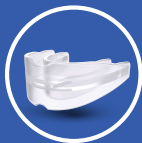
NAPSON

Anti-Snoring Pro Set

Congratulations on your purchase of the Anti-Snoring Brace and Anti-Snoring Nasal Spreaders!

This set of products is designed to help you experience a peaceful, snore-free night. The anti-snoring brace works by pushing the lower jaw forward a few millimetres, while the nasal spreaders and magnetic nose clip improve airflow through the nose. Together, they provide a complete solution for reducing snoring and improving your breathing while sleeping.

This manual contains instructions on proper use, maintenance and other important information for safe and effective use of all products in this set.

Safety Instructions

WARNING: This product is suitable for adult use only and should not be used by children or people with severe breathing problems, such as sleep apnoea. Consult your doctor before using the product if you have any doubts about your health condition.

Do not use: if you have jaw problems, loose teeth or other serious dental problems unless you have consulted a dentist.

Keep the product clean: Clean it thoroughly after each use to prevent bacterial growth.

Storage: Always store the brace in the antibacterial storage box provided to keep it clean and hygienic.

Using Instructions

Preparing the Anti-Snoring Brace

1 - Place spatula: Insert the spatula into the opening of the snoring brace

2 - Water preparation: Add boiling water to a bowl and prepare a bowl of cold water.

3 - Heating: Let the hot water cool for 30 seconds and then dip the brace into the water for 30 seconds to make the material flexible

4 - Place: Remove the brace from the water, let it drain for 15 seconds and bring the brace to the mouth with the spatula

5 - Shaping: Align the bracket straight with the teeth. Shape with fingers, cheeks, lips and tongue. Suck air from the mouthpiece and let sit for 1 minute.

Caution: do NOT bite!

6 - Cooling: Place the brace in cold water for 2 minutes, remove the spatula and allow the brace to set.

7 - Check: Check the fit and repeat if necessary.

Use:

1. Place the brace in your mouth before going to sleep.
2. The lower jaw is pushed slightly forward, which helps open the airways.
3. This reduces the risk of snoring by increasing the tension in the soft tissue at the back of the throat and increasing the space for airflow.

Using Instructions

Use of Nasal Spreaders (Soft and Hard)

Placement of the nasal spreaders:

1. Clean the nasal spreaders with water and mild soap before each use.
2. Choose the right size and type of nasal spreader (soft or hard) that feels most comfortable for you.
3. Carefully insert the nasal spreader into both nostrils. Make sure the nasal spreader sits firmly but does not feel uncomfortable.

Function of nasal spreaders

The nasal spreaders help open the nasal cavities, improving airflow and reducing snoring. They are effective for people who snore due to a blocked nose or narrow nasal cavities.

The soft nasal spreaders offer a more comfortable fit, while the hard nasal spreaders provide firmer support.

Use of the Magnetic Nose Clip

Placement of the magnetic nose clip:

1. Clean the nose clip before each use with water and mild soap.
2. Place the nose clip on the outside of the nostrils. The magnets help keep the nostrils open and improve airflow.

Function of the magnetic nose clip

The magnetic nose clip improves airflow through the nose, reducing the risk of snoring. The magnetic force provides a gentle but effective opening of the nostrils.

Stop with snoring

Anti-Snoring Brace: Dental designed and clinically proven effective in relieving snoring, clenching and teeth grinding. The ergonomic design adapts to the shape of your teeth and allows you to sleep comfortably without interruptions caused by snoring.

Nasal Spreaders and Magnetic Nose Clip: These devices improve airflow through the nose and reduce nasal snoring. Suitable for people who suffer from nasal congestion or narrow airways while sleeping.

Maintenance and Cleaning

Daily cleaning: After each use, clean the products with lukewarm water and mild soap. Rinse thoroughly and allow them to dry before placing them in the antibacterial storage box provided.

Periodic maintenance: Check the products regularly for wear or damage. Replace them if necessary to ensure optimal performance.

Technical specifications

Anti-Snoring Brace: Made of FDA-approved medical material, BPA-free, odourless and latex-free. Advanced micro-adjustability allows users to adjust the brace to fit their mouth for ideal fit and effectiveness.

Nasal Spreaders (Soft and Hard): Made of medical silicone material that is soft to the skin and effectively opens the nostrils for better airflow.

Magnetic Nose Clip: Made of medical silicone material with integrated magnets for gentle yet effective opening of nasal cavities.

Troubleshooting

The brace or nasal spreaders do not fit properly: Repeat the process of soaking the brace in hot and cold water for a better fit, or choose a different size nasal spreader if it feels uncomfortable.

Continues to snore: Check that the products are placed correctly and make sure they are properly fitted to your body. Consult a doctor if the problem persists.

Warranty & Customer service

These products come with a 12-month warranty. Please contact our customer service for replacements or problems with the product.

Contact information:

Email contact@napson.nl for further support.

Legal Information

CE marking: These products comply with European health and safety standards.

Recycling instructions: Do not dispose of these products with normal household waste. Follow local recycling guidelines for medical devices.

By following this guide, you can get the most out of your anti-snoring brace and anti-snoring devices and enjoy a peaceful, snore-free night.