dutchblue.com

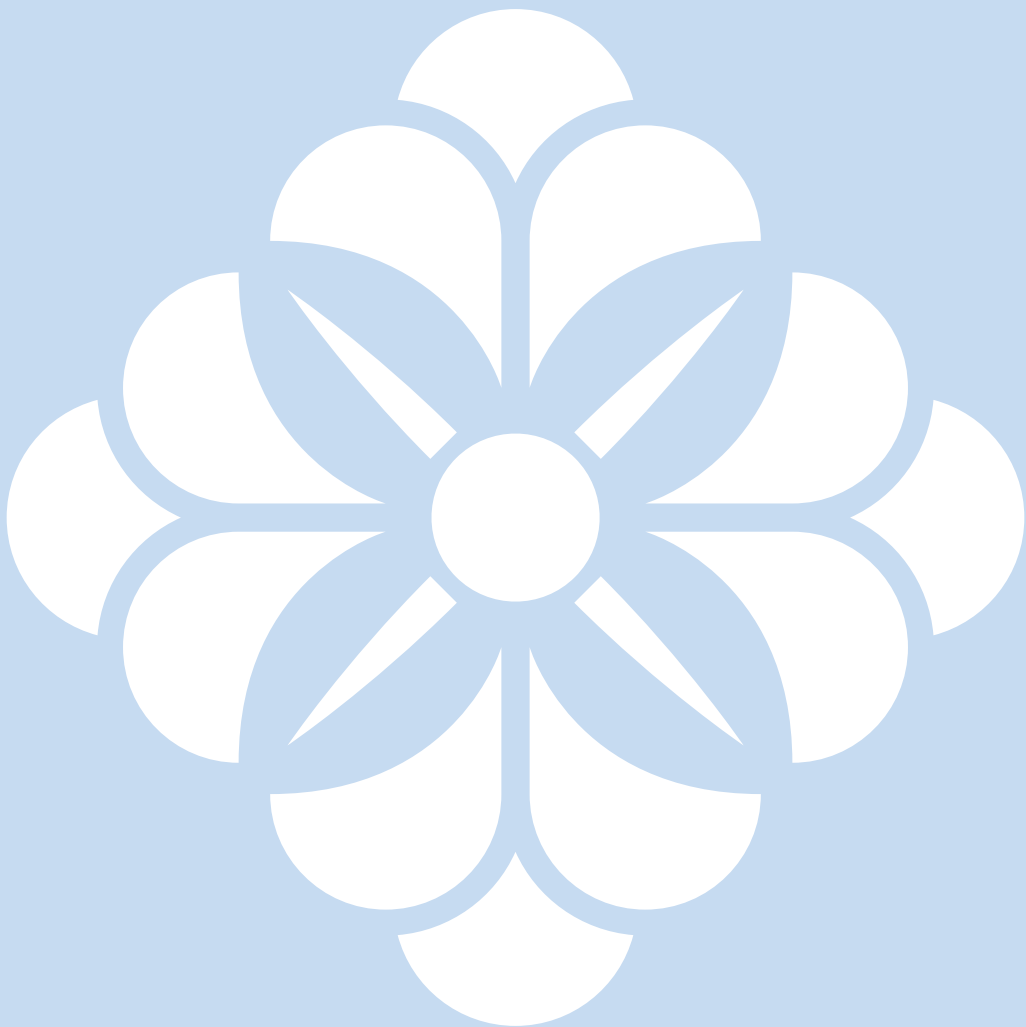
HIPPIES

User Manual | Hip protection pants
Gebrauchsanweisung | Hüftschutzhose
Mode d'emploi | Culottes
Gebruikershandleiding | Heupbeschermbroeken
Manual de uso | Pantalónes de protección de cadera
Istruzioni per uso | Pantaloncini per la protezione dell'anca
Instrukcja użytkowania | Spodenki chroniące biodra
Manual do usuário | Cuecas protetoras de anca
Manual de utilizare | Pantaloni de protecție a șoldului
Εγχειρίδιο χρήστη | Παντελόνι προστασίας ισχίου
Használati utasítás | Csípővédő nadrág
Priručnik za korisnike | Hlače za zaštitu kukova
Ръководство за употреба | Панталон с протектор за бедра

REF 4000 series

HIPPIES

English	User Manual	4
Deutsch	Gebrauchsanweisung	6
Français	Mode d'emploi	8
Nederlands	Gebruikershandleiding	10
Español	Manual de uso	12
Italiano	Istruzioni per uso	14
Polski	Instrukcja użytkowania	16
Português	Manual de Utilização	18
Română	Manual de utilizare	20
Ελληνικά	Εγχειρίδιο χρήστη	22
Magyar	Használati utasítás	24
Hrvatski	Priručnik za korisnike	26
Български	Ръководство за употреба	28

This manual contains information for the products:
REF 4000, REF 4001, REF 4002, REF 4003, REF 4004, REF 4005
REF 4010-S, REF 4011-M, REF 4012-L, REF 4013-XL, REF 4014-XXL
REF 4020-S, REF 4021-M, REF 4022-L, REF 4023-XXL, REF 4025-XXL
and REF 4090

ENGLISH

General information

HIPPIES are supplied as a set of 4 hip protection pants in one size with 2 protectors or as a set of 3 hip protection pants with 2 sewn-in protectors each. The hip protector pants also come as a set of 4 without protectors. The HIPPIES hip protection pants with protectors can be worn comfortably during the day and at night, and can be easily combined with incontinence products.

Composition

Pants
95% cotton 5% elastane

Protectors
Polyurethane

4 Hip protection pants with 2 protectors

Size	REF
S	4000
M	4001
L	4002
XL	4003
XXL	4004
XXXL	4005

4 Hip protection pants without protectors

Size	REF
S	4010
M	4011
L	4012
XL	4013
XXL	4014
XXXL	4015

3 Hip protection pants with 2 sewn-in protectors each

Size	REF
S	4020
M	4021
L	4022
XL	4023
XXL	4024
XXXL	4025

2 protectors

Size	REF
1 set	4090

ATTENTION:

Always check the protectors for damage etc. HIPPIES cannot prevent all hip injuries but contribute significantly to reducing the risks. Avoid contact with sharp objects.

ENGLISH

Instructions for cleaning

Hip protector pants

- The hip protector pants are machine washable at 60°C. They can be dried at up to 80°C.

Hip protector pants with sewn-in protectors

- The hip protector pants with sewn-in protectors can be machine washed as a set at 60°C.
- After washing it takes about 36 hours until the protectors are completely dry.

Protectors

- Removable protectors can also be cleaned with a soft cloth and mild detergent or clinical cleaning agents.
- Please wash manually at up to 60°C. Do not wash in the washing machine
- Never use disinfection agents, phenol or solvents.
- Do not use bleaching agents as they can damage the HIPPIES over time.

Instructions for use

Inserting the protector in the pants:

- Both sides of the hip protector pants have been fitted with inside pockets with flaps.
- Before putting on the hip protector pants insert both protectors into the pockets, with the narrow part pointing downwards and the flat side facing inwards.
- The pockets are closed with a flap to ensure that the protectors remain securely in place.
- Once both protectors have been inserted into the pants, the hip protector pants with protectors can be worn in the customary fashion.

Hip protection pants with sewn-in protectors:

- The hip protector pants with sewn-in protectors can be worn immediately in the customary fashion.

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makes care affordable

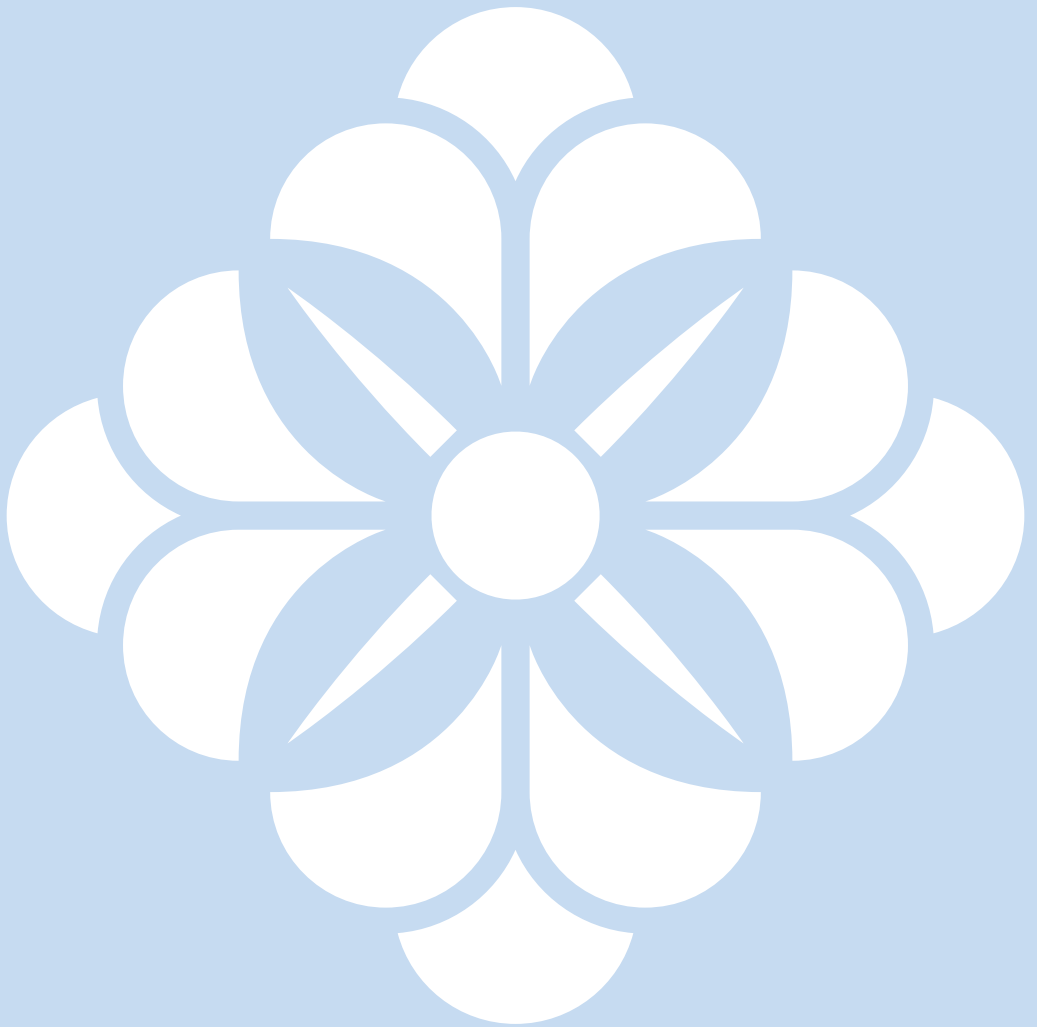
About Dutchblue:

We were born with a dream: We want to make care affordable for those who need it. Dutchblue strives to improve the daily work of as much people in care as possible. This will be achieved by offering a carefully selected, high quality, product range at very low prices.

How can dutchblue be so advantageous?

- We have low prices because we produce in large numbers
- We have a small product range that is carefully selected. This makes us fast and flexible, hence cheaper.
- We deliver directly to our customers. Therefore there is no one in between that will raise the price.
- We don't work with expensive external employees. This lowers the operatin expenses and thereby the prices.

dutchblue.com

Bulgaria

Parizhka Komuna 26
9000 Varna

Deutschland

Subbelrather Str. 15a
50823 Köln

Österreich

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France

57 rue d'Amsterdam
75008 Paris

België / Belgique

Square de Meeûs 37
1000 Brussels

Nederland

Faradayweg 20
2627 AX Delft

- EN** Thank you for choosing our product.
DE Vielen Dank, dass Sie sich für unser Produkt entschieden haben.
FR Merci d'avoir choisi notre produit.
NL Dank u voor het kiezen van ons product.
BG Благодарим ви, че избрахте нашия продукт.