koalaa

Siliconen Borstkolf | Silicon Breast Pump |
Silicon Milchpumpe

NL Silicone borstkolf

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EN Silicone Breast Pump

1. Make sure that the Koalaa® Silicone Breast Pump is thoroughly cleaned and sterilized before use.
 2. Place the lid of silicone breast pump over your nipple: Ensure your nipple is in the neck of the pump and comfortable.
 3. **Apply suction:** You may need to adjust a couple of times for efficacy. Once the breast pump is secure and comfortable, you can squeeze the base of the pump (not interfering with the top/suction) to create some movement.
 4. Gently massage the top of your breast to encourage milk flow.
- Tip:** Placing a warm flannel on your breast will help milk flow also.
5. Let the breast pump do its thing! Once milk begins to flow, simply leave the pump to pump your milk unaided.
 6. **Empty the breast pump:** Once the pump is full, empty into the milk storage bottle / bag of your choice and reposition. Or choose to place the lid on top.
 7. **Clean after each use:** This product can be sterilized with any steam sterilizing system or in boiling water. See our Koalaa® double bottle warmer which includes sterilization.
 8. Place the lid after every use back on top of the breast pump.

*Please always consult with a medical professional regarding dietary requirements if you or your child is ill or on medication.

Thank you for purchasing our product!

We strive to offer better products, customer-friendly services, better prices and the best service for our customers. We have a wide range of products, suitable for everyone's choice. We therefore invite you to take a look at our website www.koalaa-shop.com

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