

1. Introduction

We are pleased that you have purchased a quality product from Homecare Innovation. This user manual contains a detailed product description and essential instructions for the correct use of the product. Therefore, please read this manual carefully before using the product. Please pay attention to all precautions, especially the safety instructions, and always comply with them in order to achieve a safe and long-lasting use of the product.

Intended use of the product

The product is a medical device used by elderly people or people with mobility impairments who need equipment to overcome obstacles, such as steps, with or without the assistance of a caregiver. The ramps are designed to compensate for level differences down to 4 mm and up to 60 mm. With the additional option of modular construction. The ramps can be used indoors as well as outdoors. No other use than that described in the purpose statement is permitted.

Indications

Ramps are indicated for people with mobility limitations who due to problems with walking, standing or sitting are forced to use mobility devices such as wheelchairs, scooters, rollators etc. Ramps are indicated for elimination of architectural obstacles by smooth connection of the level differences that a wheelchair user has to overcome and thus they allow the wheelchair user to reach places normally inaccessible to him/her. The ramp is intended for use independently or with the help of an attendant. Using the ramp for purposes other than those indicated above, is prohibited.

Contraindications

Contraindications cannot be tied to a specific clinical picture. The choice of a suitable ramp type and length must always be made with the help of a specialist doctor or physiotherapist. Observe the safety information in chapter 2.

If you have any questions, please contact your dealer or Homecare Innovation.

Homecare Innovation Steenbakkerij 16 2913 LJ Nieuwerkerk aan den IJssel Nederland www.homecareinnovation.nl	Phone: 010-2420913 Email: info@homecareinnovation.nl
--	---

2. Safety requirements

- Only use the product in perfect condition.
- If any defects or faults are found in the product, the dealer should be contacted immediately.
- Pay attention to the information on the product label.
- Use the product only for the stated purpose.
- Make sure that this user manual is read by all persons using the ramp. Damage caused by not following the user manual will not be compensated.
- Use the ramp only for its intended purpose.
- Only use ramps that are in good technical condition.
- The ramp may only be used on a stable surface.
- The product must never be subjected to loads exceeding a total of 350 kg.

3. Description

Product description

Homecare Innovation rubber ramps are made of TPE and contain no toxic substances (PVC free). The product has a grooved surface which makes it slip-resistant. The ramps can be used both indoors and outdoors.

Parts supplied


- ✓ This instruction manual
- ✓ 1 or more rubberramps

First control

Check above that nothing is missing.
If anything is missing, contact the retailer of the product immediately.

Mounting and assembly instructions

The rubber ramp mounting kit consists of two mounting brackets that are installed in the doorstep. It is then possible to mount rubber ramps without gluing the ramp to the substrate.

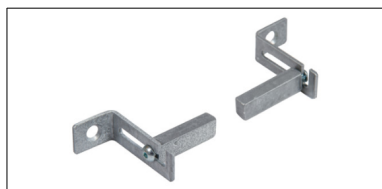

Mounting kit



Mounting bracket mounted into the doorstep




Double adhesive tape




Degreaser for removal

Cutting of rubber ramps

The rubber ramps are easily adjusted in width with a sharp hobby knife or saw blade. For higher ramps, it is necessary to cut several times in the same groove. After each cut, break the ramp backwards to facilitate the next cut.

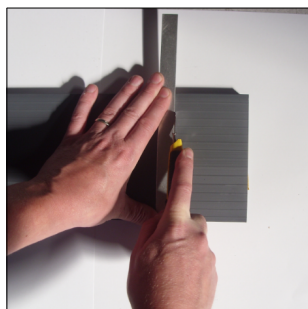

Low ramp cutting

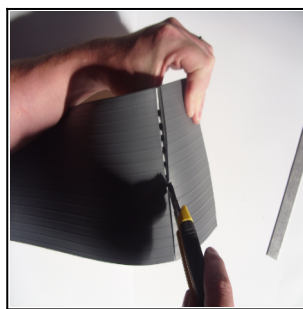


High ramp cutting




Ramp ready for use

Possibility to combine the ramps with modular construction

Measurements on the outside, inside and possibly the height of a rubber ramp. Choose suitable ramps. Can also be built on top of each other.

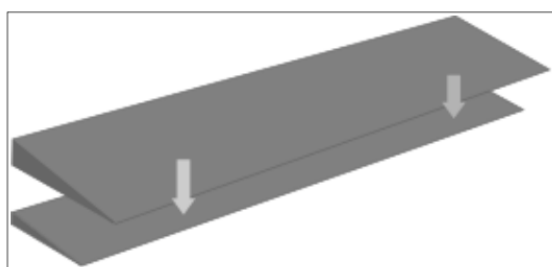

Combined ramps for desired height

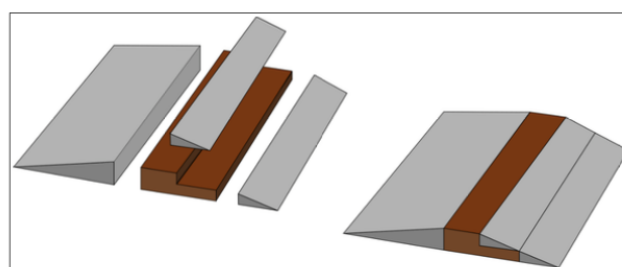

Adaptation of ramps to doorsteps of different heights

Accessories

The following accessories are available for the rubber ramps:


- Mounting set for rubber ramp zinc plated, 28-60 mm height
- Jigsaw blade for rubber ramp
- Double adhesive tape, width 20 mm, length 50 meter
- Double adhesive tape, width 35 mm, length 50 meter
- Degreaser for removal of glue 1/2 L

Transport recommendations

The ramps are removable if not glued and can therefore be easily moved to a new location if necessary.

4. Technical Data:

Height	Depth	Width	Weight
4 mm	40 mm	900 mm	130 g
4 mm	40 mm	1000 mm	150 g
6 mm	60 mm	900 mm	310 g
6 mm	60 mm	1000 mm	350 g
8 mm	80 mm	900 mm	490 g
8 mm	80 mm	1000 mm	540 g
10 mm	100 mm	900 mm	770 g
10 mm	100 mm	1000 mm	860 g
12 mm	120 mm	900 mm	1070 g
12 mm	120 mm	1000 mm	1190 g
16 mm	150 mm	900 mm	1230 g
16 mm	150 mm	1000 mm	1370 g
20 mm	150 mm	900 mm	1850 g
20 mm	150 mm	1000 mm	2050 g
24 mm	150 mm	900 mm	1950 g
24 mm	150 mm	1000 mm	2170 g
28 mm	200 mm	900 mm	2180 g
32 mm	200 mm	900 mm	2960 g
36 mm	200 mm	900 mm	3990 g
40 mm	250 mm	900 mm	3550 g
44 mm	250 mm	900 mm	4250 g
48 mm	250 mm	900 mm	5190 g
52 mm	300 mm	900 mm	5800 g
56 mm	300 mm	900 mm	6500 g
60 mm	300 mm	900 mm	7700 g

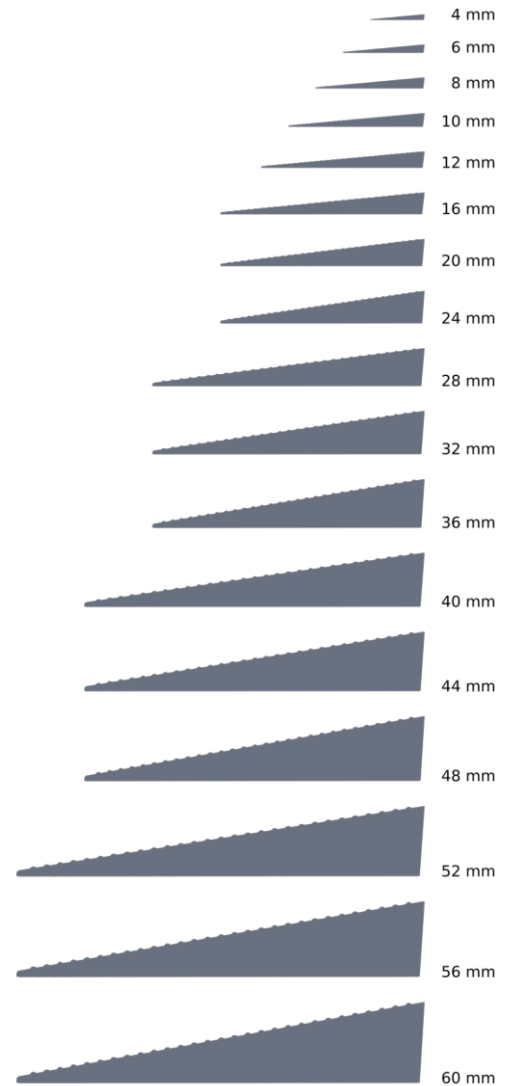

Rubber ramps are also available in length 700 mm in sizes 10-48 mm (all models with self-adhesive tape).

NOTE! Dimensions and weight of rubber ramps may vary.

Product label

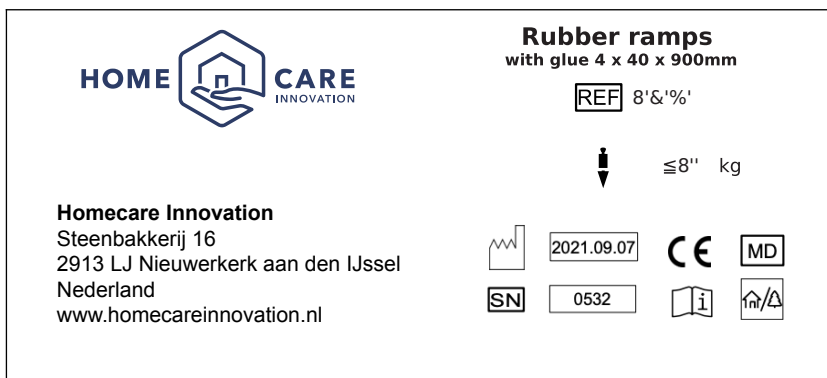


Image of the product label

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

5. Care & Maintenance

The rubber ramps are maintenance-free and can be easily cleaned with ordinary detergents.
Machine washable according to The Danish Serum Institute guidelines for MRSA in home care and in nursing homes.

- ✓ Cleaning and disinfection of the product

Recommended service life: Cleaning and disinfection of the product

Storage

We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Most common problems and solutions

Symptoms	possible cause	what to do?
The rubber ramp is unstable or does not stay in the desired position	The surface is not level	Use the mounting kit to attach the ramp to the doorstep or seek alternative attachment methods*

On delicate surfaces such as carpets and wooden floors, it is not recommended to attach rubber ramps with adhesive tape.

7. Disposal of the product

The device cannot be deposited with normal garbage. It must be delivered to the public station for re-use.

8. Guarantee terms

Homecare Innovation offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CEmarking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Homecare Innovation directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.