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Talking calendar clock for visually impaired and blind people

User Manual - English

10 Inch | Model: Geasy 10 Inch

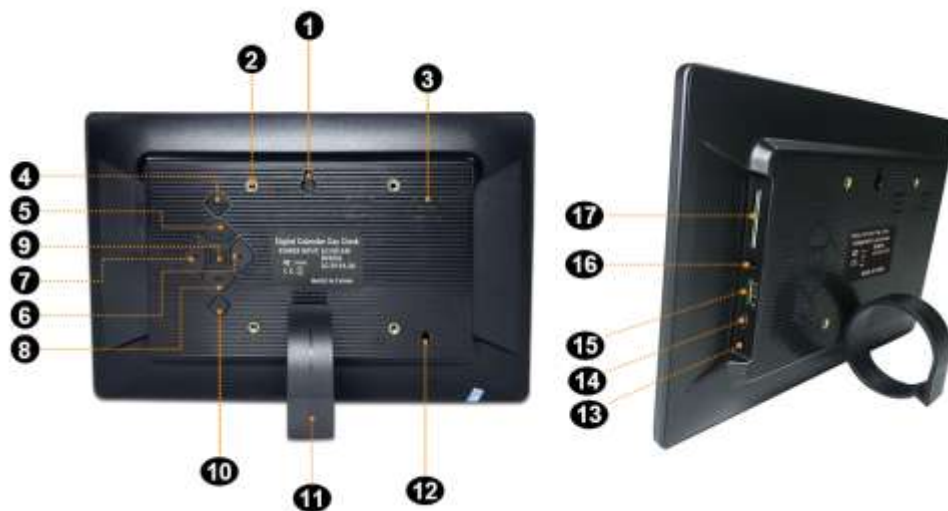
This manual belongs to the model shown above. Button layout and connections may differ by size. Always use the manual for the correct size.

Product overview

This calendar clock is designed for people with visual impairment or blindness. It displays time, day, date, month and part of the day in a large high-contrast format and can also speak time information in supported languages. **The display language and menu are available in several languages. The speech function works in Dutch, English, German and French.**

Buttons and connections

- 1. Hanging hole
- 2. VESA mount point
- 3. Speaker
- 4. Power button
- 5. Up button
- 6. Left button
- 7. Right button
- 8. Down button
- 9. OK/Enter button
- 10. Menu
- 11. Stand
- 12. Security lock
- 13. DC input
- 14. Micro-USB port
- 15. USB port
- 16. Headphone jack
- 17. SD card slot

Getting started

1. Connect the supplied 5V/2A power adapter to the DC input. The clock starts automatically.
2. Open the settings menu with Menu/Setup on the clock or remote control.
3. Set the language, time and date first.

Placement

The clock can be placed upright using the supplied stand. Place the clock on a flat, dry and stable surface. The clock can also be wall-mounted using a suitable VESA bracket or the hanging hole. After placement, check that the clock is secure and cannot fall.

Changing the display

Press and hold OK/Enter for 3-5 seconds to switch to the next clock display. Repeat until the desired display is shown. Press OK/Enter briefly to confirm a selection.

Setting the language

The clock starts in English by default. In the menu, go to Language. First press OK/Enter to select this function. The selected language line will show yellow text on a red background. You can then use left/right to choose the desired language. Confirm with OK/Enter.

Menu options

- Language: choose the desired display language.
- Set Time: adjust hours and minutes.
- Set Date: adjust day, month and year.
- Time Format: choose 12-hour or 24-hour display.
- Date Format: choose the day, month and year order.
- Brightness: choose automatic or level 1-5.
- Volume: adjust from 0 to 30.
- Font Colour: choose white, yellow or blue.

Alarms and reminders

You can set reminder alarms for appointments and daily routines. Open the alarm menu with OK/Enter, add an alarm, set time, date, repeat and label, and turn the alarm on or off. Up to 25 alarms can be set.

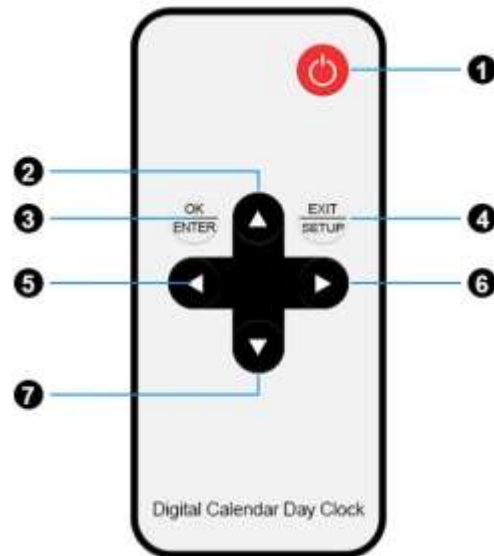
Speech function

On the main screen, press OK/Enter once to hear the time, weekday and part of the day. The preset alarms are spoken when they go off. This speech function works in Dutch, English, German and French. In other display languages the clock remains fully usable, but this specific speech function is not supported.

The automatic hourly announcement can be enabled or disabled in the settings menu. A time window can also be set, for example 08:00-18:00.

Remote control

The remote control has the same basic functions: power, OK/Enter, Setup/Exit and navigation up, down, left and right. Hold OK/Enter for 3-5 seconds to change the clock display.

Safe use and maintenance

Technical information

- Model/size: 10 inch.
- Power supply: 5V/2A power adapter.
- Use: indoor use only.
- Ports and connections: see the Buttons and connections overview in this manual.

Manufacturer and contact

- Geasy B.V.
- Oudenoord 115
- 3513EM Utrecht
- The Netherlands
- E-mail: info@geasy.nl

Safety instructions

- Use only the supplied adapter or a suitable 5V/2A adapter.
- Do not use the clock in damp rooms or near water.
- Do not pull the cable. Place the cable so that nobody can trip over it.
- Do not use the product if the cable, adapter or housing is damaged.
- Do not open, modify or repair the product yourself.

Cleaning and maintenance

- Unplug the clock before cleaning it.
- Clean the clock with a dry, soft cloth.
- Do not use water, solvents, abrasives or aggressive cleaning agents.

Disposal

- Do not dispose of this product with household waste.
- Take the product to a suitable collection point for electrical and electronic equipment in accordance with local regulations.
- Empty batteries must be disposed of separately in accordance with local rules.

Remote control and battery

- If the remote control uses a replaceable battery, always keep batteries out of reach of children.
- Insert the battery according to the indicated plus and minus polarity.
- Remove empty batteries from the remote control and dispose of them separately.

Warranty and customer service

- For questions, defects or missing parts, please contact Geasy at info@geasy.nl.

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