

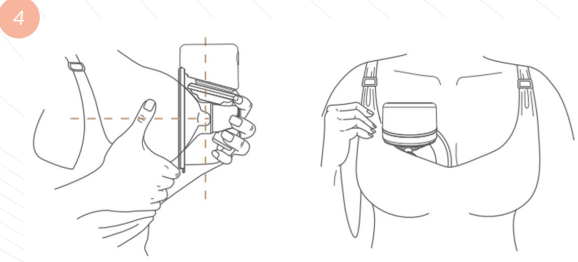
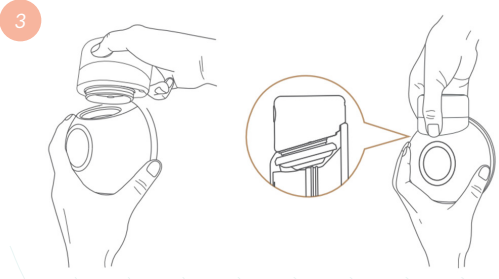
Sobia Breast Pump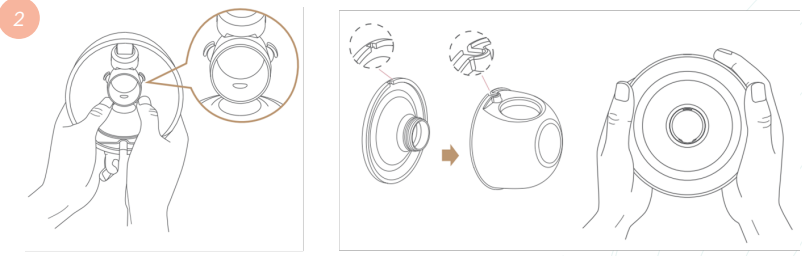

Instructions Manual

Please Read This Manual Before Use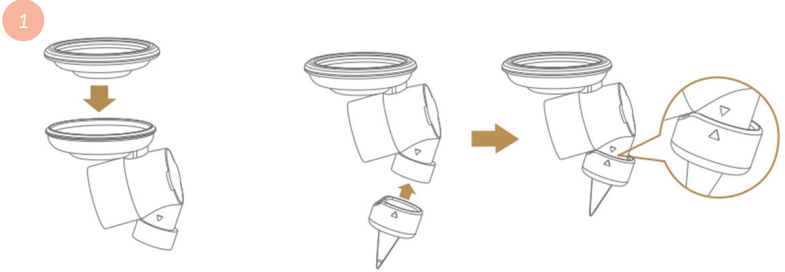

I.	Intended use	7
II.	WARNING AND SAFETY!	8
1.	Warnings	8
2.	Cautions	9
III.	Parts	10
IV.	Getting Started	11
1.	Cleaning before first use	11
2.	Tubing parts	12
V.	Using the breast pump	13
1.	Assembling the breast pump	13
2.	Flange size guide	15
3.	Motor unit part description	16
4.	Start pumping	17
5.	After use	18
6.	Charging	18
VI.	Storing breast milk	20
1.	Freezing	20
2.	Thawing	20
VII.	Ordering accessories	21
VIII.	Troubleshooting	21
IX.	Supplemental information	22
X.	Storage conditions	23
XI.	Technical information	23
1.	Specifications	23
2.	Different modes and suction powers	24

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Thank you for choosing Sobia's breast pump. Breastfeeding is one of nature's most beautiful gifts a mother can give to her child. Unfortunately, there are some situations and reasons why breastfeeding is not always possible. Sobia wants to make sure every mother can participate and provide the best equipment during this beautiful journey. The World Health Organisation and Healthcare professionals recommend breast milk as the primary source of nutrition during the first two years. After a period of 6 months, you can combine breastfeeding with solid food. Breast milk is by nature adapted to your baby's needs and helps to protect against infections and allergies. If you choose to breastfeed longer than anticipated, you can express and store your milk so that your baby can still enjoy the benefits, even if you cannot be there to provide it yourself. Our breast pumps are compact, quiet, discrete, and easy to take anywhere with you, allowing to express milk at your own convenience and to maintain your milk supply. The breast pumps are well engineered to foresee continuous voltage and strong suction power. This means the suction power will not weaken and still perform the same if the battery is on 100% or 5%. It takes a learning curve to find your comfortable level to breast pump.

Sobia is a brand created for babies, by mothers for mothers.

Breast pumps of Sobia are well known for the 4 modes:

- Massage
- Expression
- Lactation
- Auto

The massage modus is a rapid micro vibration to help stimulate milk production, relieve blocked ducts, swelling and lumps. By imitating the suction of a baby, the massage modes help to trigger the let-down reflex.

Once let-downs begin you can switch to Expression or Lactation mode, which are using both different frequencies and have their own characteristics. Thanks to the 12 different suction levels, you can choose what level you feel comfortable with during your journey. There is also an auto mode which will switch from Massage mode to Expression mode automatically after 5 times.

I. Intended use

The electric breast pumps of Sobia are intended to express and collect milk from the breast of a lactating woman during the journey of her baby. The device is intended for one single user only.

II. WARNING AND SAFETY!

Before using the breast pump read this manual carefully.

Warnings identify all instructions that are important for safety. Failure to observe these instructions can lead to personal injury or damage to the product.

1. Warnings

Never use breast pumps during pregnancy as pumping can induce labour.

a. Avoid choking, strangulation and injury:

- ▣ This breast pump is not intended for use by persons (including children) with reduced physical, sensory, or mental capabilities, or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised by or have received proper instruction concerning use of the appliance by a person responsible for their safety.
- ▣ Do not allow children or pets to play with the motor, adapter, packaging materials or accessories.

b. Prevent transmitting infectious diseases:

- ▣ If you are infected with Hepatitis B, Hepatitis C or HIV, expressing breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk. Always consult your healthcare professional when in doubt.

c. Avoid fire, electric shocks, or serious burns:

- ▣ Before each use inspect the breast pump for signs of damage. If the breast pump is damaged, does not work properly, it has been dropped or submerged in water do not use it.
- ▣ Never operate electrical devices if the cord or plug is damaged.
- ▣ Never use electrical products that have been exposed to water or other liquids:
 - Do not use while bathing or showering.
 - Never place or drop into water or other liquids.
 - Do not run water over the motor unit.
 - If an electrical product has been exposed to water or other liquids, do not touch, and turn off immediately.

d. Avoid contamination and to ensure hygiene:

- ▣ For hygienic reasons, the breast pump is only intended for repeated use by a single user.
- ▣ Clean, disinfect and sterilise all non-electric parts before first use. Clean and wash directly after each session.
- ▣ Using antibacterial or abrasive cleaning agents may cause damage on the breast pump.
- ▣ Wash your hands thoroughly with soap and water before you touch breast pump parts and breasts to prevent contamination.
- ▣ Do not use the device while operating a moving vehicle.

- Cleaning and maintenance must not be performed by children without supervision.

e. Avoid breast and nipple problems including pain:

- Do not use the breast pump if the silicone flange appears to be damaged or broken.
- Only use accessories and parts recommended by Sobia.
- Do not modify the breast pumps, if you do so the warranty becomes invalid.
- Never use the breast pump while you are sleepy or drowsy to avoid lack of attention during the session.
- It's advised to switch off the breast pump before removing the pump from your breast in order to release the vacuum.
- If the vacuum is uncomfortable or causes pain, switch off and remove the breast pump from your breast.
- Do not continue pumping for more than five minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.
- If the process becomes very uncomfortable or painful, stop using the pump and consult your healthcare professional.

f. Prevent damage and malfunction:

- Portable radio frequency (RF) such as mobile phones and other communication equipment, should not be used closer than 30cm to any electronic part of the breast pump. This could create interference with the breast pump and negatively impact the performance of the breast pump.

2. Cautions

a. Prevent damage to and malfunctioning of the breast pump:

- Prevent the motor unit from encountering water or any other liquid fluids.
- Never put the motor unit in any liquid fluids or dishwasher, as this causes permanent damage to these parts.
- Keep the silicone parts away from heated surfaces to avoid overheating and deformation.
- Do not charge and use the breast pump at the same time.
- Do not use near flammable materials.

b. Avoid health risks and reduce the risk of injury:

- As this is a single only product, the use by more than one person may present health risks.
- This product cannot be serviced or repaired. Do not repair yourself. Do not modify the device.
- Never use a damaged device.
- Replace defective or worn parts.
- Use the device only for its intended purpose as described in this manual.
- Do not use the device while sleeping or overly drowsy.
- Do not charge and breast pump at the same time.
- If tubing becomes mouldy, replace tubing as soon as possible.

- ▣ Clean, sanitise and sterilise all non-electrical parts that come into contact with your breast milk prior to first use.
- ▣ While some discomfort may be felt when first using a breast pump, it should not cause pain.
- ▣ If you are unsure about correct breast shield sizing, please see a lactation consultant/ breastfeeding specialist who can help you get a proper fit.

III. PARTS

- 1 Big size Flange 24mm & 27mm
- 2 2x Diaphragm
- 3 1x Linker
- 4 4x Valves
- 5 1x Milk Collector
- 6 1x Motor
- 7 USB Cable
- 8 2x Rubber band
- 9 4x Smaller flanges: 17mm/19mm/21mm

IV. Getting Started

1. Cleaning before first use

WARNING!

Do not wash the breast pump with antibacterial or abrasive cleaning agent as this may cause damage. Don't forget to sterilise before first use.

CAUTION!

- ▣ Never put the motor unit in water or in a dishwasher as this causes permanent damage.
- ▣ After each session, immediately wash all parts that came into contact with breast milk. This will prevent growth of bacteria and help remove breast milk residue.
- ▣ Wash before each session your hands thoroughly with soap and water.
- ▣ Only use drinking-quality tap water or bottled water for cleaning.
- ▣ Do not place pump parts in the sink to rinse or wash. Use a clean wash basin used only for infant feeding items.
- ▣ Do not use a dish towel to rub or pat items dry instead leave all parts to air- dry on a clean tea towel or drying rack.

Step 1: Disassembling

Parts that came in contact with milk should be cleaned manually or in the dishwasher after each use. All parts apart from the electric units (motor) can be washed or sterilised.

NOTE!

Don't forget to sterilise before first use.

When using the dishwasher please bear mind that the silicon parts can discolour after a time.

Step 2: Cleaning

a. Wash by hand

- i. Rinse the parts that come into direct contact with breast milk (except for the motor) with cool water.
- ii. Clean these parts with plenty of warm, soapy water.
- iii. Rinse the parts with cool water.
- iv. Leave all parts to air- dry on a clean tea towel or drying rack.

b. Wash in dishwasher

- i. Place the disassembled parts (except for the motor) on the top of the dishwasher
- ii. Put dishwashing liquid or tablet in the machine and run a standard program.
- iii. Leave all parts to air- dry on a clean tea towel or drying rack.

Step 3: Disinfecting

CAUTION!

When sterilising in boiling water, make sure the silicon parts do not touch the side of the pan. This can cause irreversible product deformation or damage that Sobia cannot be held liable for.

- i. Fill a household pot with enough water to cover all parts and place all parts in the pot. Let the water boil for max 5 minutes, water should not reach temperatures higher than 100°C.
- ii. Gently remove the parts from the water. Place the parts on a clean surface/tea towel or drying rack to air-dry.

2. Tubing parts

CAUTION!

If the tubing becomes mouldy, stop using immediately and replace tubing as soon as possible.

NOTE!

Do not use the breast pumps if the tubing becomes wet/mouldy, doing so will damage the motor.

Tubing only conveys an air flow from the motor (it does not come in contact with breast milk), normally cleaning of the tubing should not be necessary. Make sure to inspect the tubing to see if you can find condensation in the air tube. If so, wash with water only and dry immediately or replace it. If you see breast milk inside the tubing do not attempt to wash or sanitise it, replace with new tubing.

a. How to wash tubing

- i. Separate the tubing from the device.
- ii. Rinse the tubing by pouring cool water into the tubing end until it flows out of the other tubing end.
- iii. Wash the tubing in warm, soapy water.
- iv. Rinse the tubing with clean water.

b. How to dry tubing

- i. Shake out any water droplets inside the tubing.
- ii. Hang the tubing to air dry.
- iii. Make sure the tubing is completely dry before using it again.

V. Using the breast pump

Make sure to only use clean and dry parts for assembly..

1. Assembling the breast pump



WARNING!

When sterilised in hot water the breast pump parts can be very hot. To prevent burns, only start assembling the breast pump when disinfected parts have completely cooled down and air dried


Insert the silicone diaphragm into the connector and press it in with your hands to ensure that the edges are fastened tightly, securing a perfect seal.


Insert the silicone diaphragm into the connector and press it in with your hands to ensure that the edges are fastened tightly, securing a perfect seal.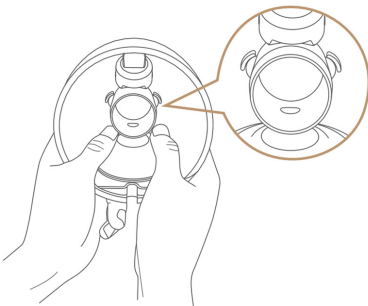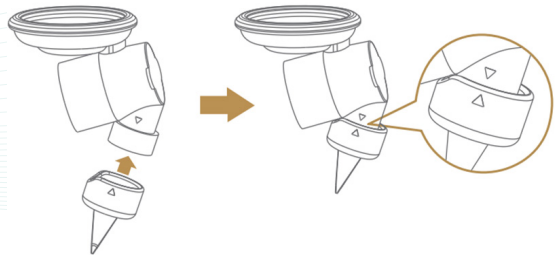


Place the assembled connector into the collection cup, with the "ears" of the connector aligned with the internal slot of the collection cup and press in the connector disc with both hands to fasten the edges firmly to ensure complete sealing.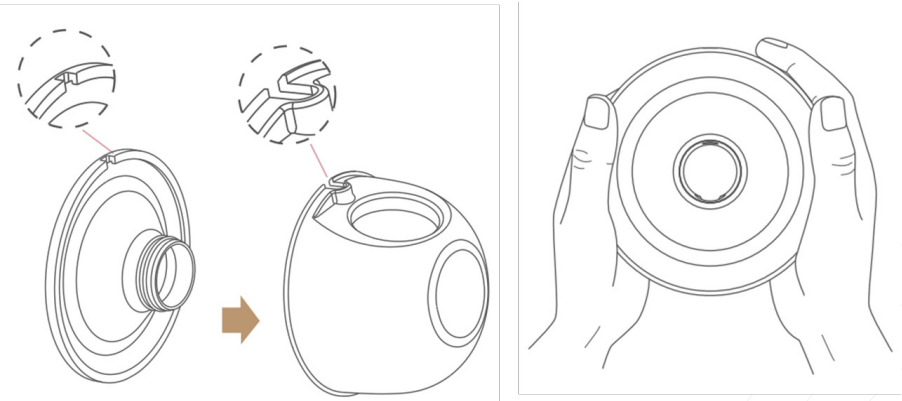

Install the silicone breast shield into the collection cup. Align the groove on the silicone shield with the groove on the collection cup. Press until everything falls into each other, and no air can enter the cup anymore.

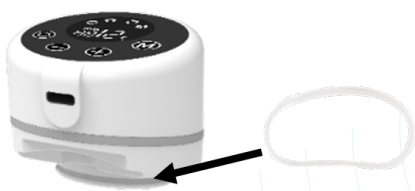
Just under the motor unit you can find a rubber band. This is crucial to make sure the motor is vacuum when put onto the milk collector. Make sure that there is no damage to the rubber band.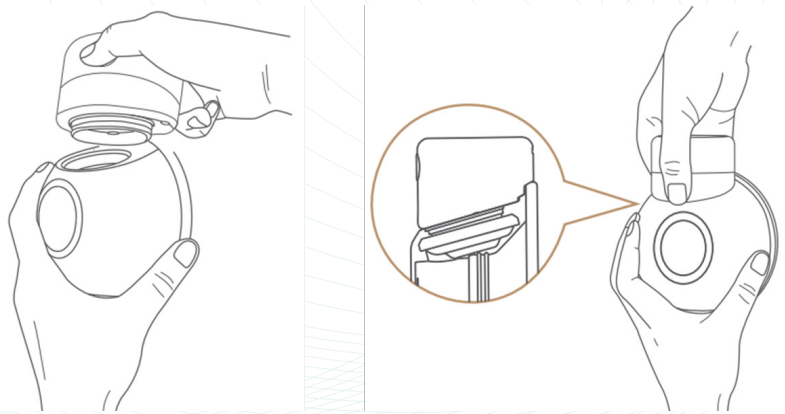

Insert the pump's motor onto the top of the collection cup and ensure that it fits perfectly.


2. Flange size guide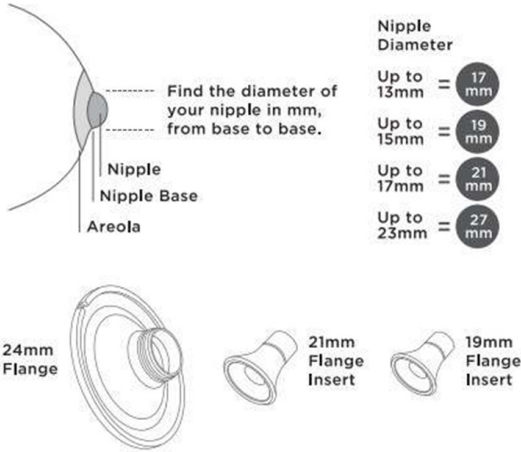

Measure the base of your nipple, where the nipple meets the areola. Do not measure your areola, just the width of your nipple. Add 4mm to your measurement. These extra millimetres allow for some space between your nipple and the flange tunnel, as well as for a bit of nipple expansion while pumping.


NOTE!

You can use coconut oil while pumping to soothe and help repair sore and cracked nipples. Simply apply a small amount of coconut oil directly to the nipple.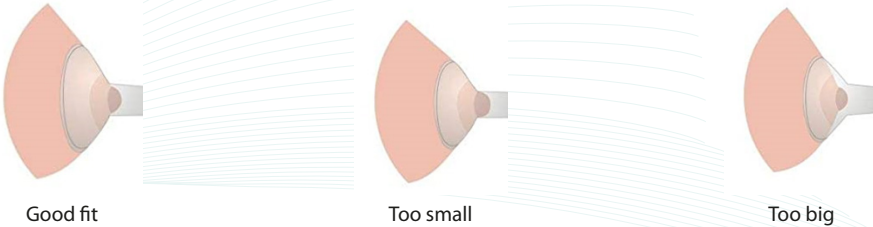


If the flange should be adapted, please do so following this picture:

Choose a smaller flange size and make sure it's good connected into the 24mm flange.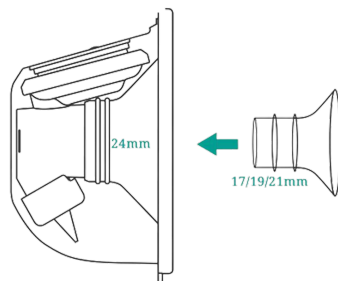

3. Motor unit part description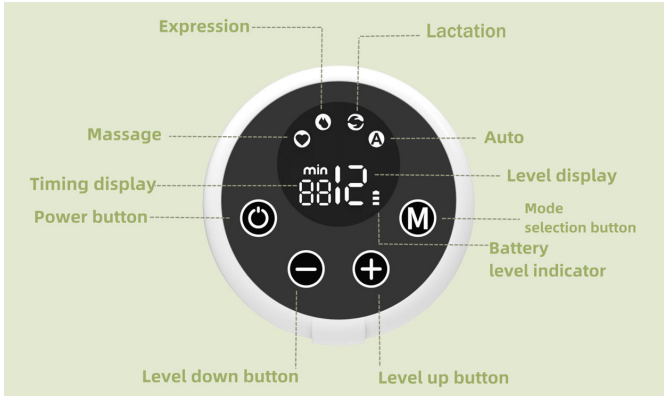

NOTE!

- ▣ Long press the power button (1-2 seconds) to start the breast pump, and the timing display will turn on.
- ▣ The machine has a memory function. When you turn it on again, it will automatically enter the mode you used during your last pump session.

CAUTION!

Do not use the breast pump while charging. Charge the motor first. When the battery is fully charged, unplug, and start using it.


This breast pump offers 4 modes starting with message mode when using the first time. Press "M" once to change modes.

1. Press it again for Expression.
2. Press it again for Lactation.
3. Press it again for Auto mode.
4. Press it again to restart the cycle from Message mode again.

Message, Expression and Milk flow modes come with 12 levels of adjustments, which you can increase "+" and decrease "-" depending on your preference.

In Auto mode, the pump will first initiate 5 times Message mode, then switch to 10 times Expression mode. Auto modes come with 6-12 levels of adjustments, which you can increase "+" and decrease "-" depending on your preference.

4. Start pumping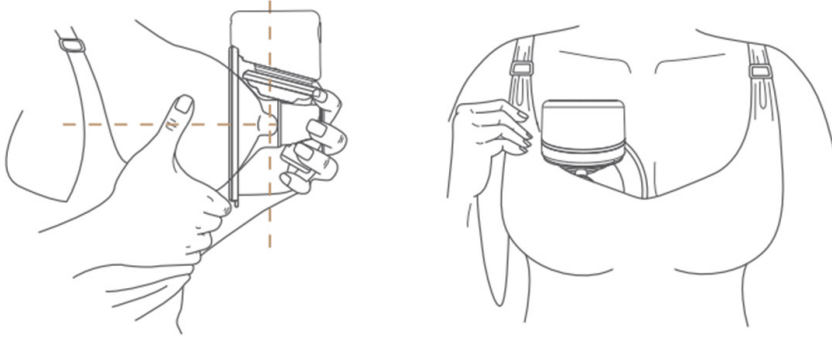

Press the silicone shield firmly against the breast, leave no gaps and make sure your nipples are centred. Keep it upright. Be careful not to distort the position of the product and the chest, otherwise the breast pump will not have suction. Long press the power button (1-2 seconds) to start the breast pump and the timing display will turn on.


CAUTION!

- ▣ Do not try to express with vacuum that is too high and uncomfortable. The pain, along with potential breast and nipple trauma, may decrease milk output.
- ▣ In case you express only minimal or no milk at all, contact your healthcare professional or breastfeeding specialist.

GOOD TO KNOW!

- ▣ Maximum comfort vacuum (which is different for every mother) is the highest vacuum setting where pumping still feels comfortable.
- ▣ Stimulation should be at a comfortable vacuum level, striving for a maximum level is not necessary.
- ▣ Reassess your maximum comfort vacuum throughout your pumping experience. It can change throughout time and each stage of lactation.

5. After use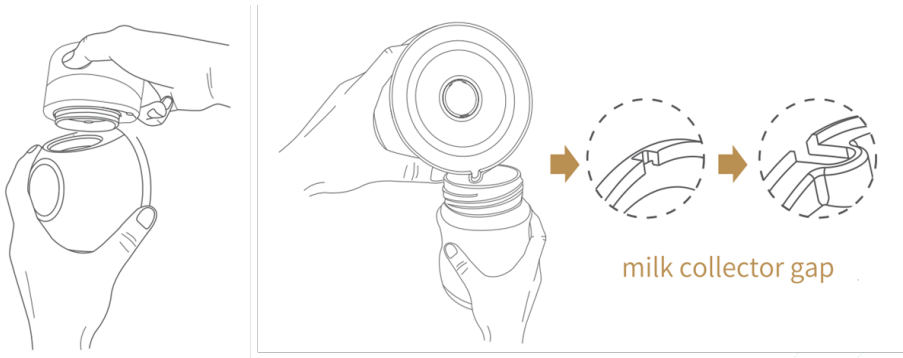


Once you have finished pumping, long press the power button (1-2 seconds) to turn off the breast pump, then carefully remove the breast pump from your breast.

NOTE!

The machine has a memory function. When you turn it on again, it will automatically enter the last mode you used during your last pump session.

After you finish pumping, please remove the breast pump from the breast. Keep the breast pump upright and take the motor of the top. Hold the milk collector to pour the breast milk into a bottle or milk bag. Make sure the milk collector gap is aligned with the opening of the bottle to prevent breast milk to be spilled.

6. Charging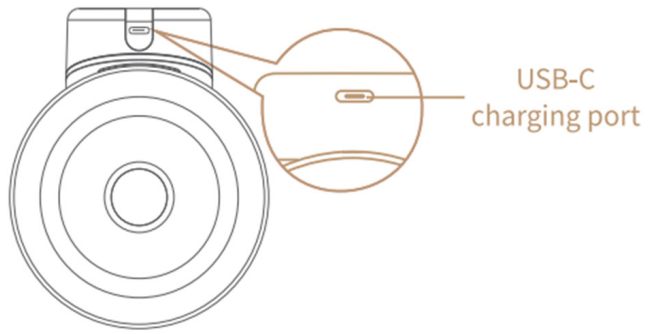


The USB power adapter is a common standard accessory in the market. This product is not provided and can be purchased separately if necessary. Power adapter specification: 5V=1A

NOTE!

- ▣ The device does not work when charging.
- ▣ This product has memory function, when you turn it on again, the machine will automatically enter to the working mode and suction level you used last time.

- ▣ The motor goes down after working for 20 minutes
- ▣ The continuous operation time vary depending on the battery or working conditions.
- ▣ The charging time may vary according to the remaining battery power, ambient temperature, or charging conditions.

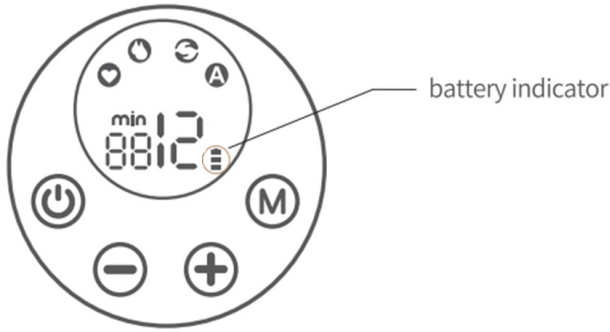Connect the USB-C cable to charge the motor.

WARNING!

- ▣ Never wash or sterilize the pump motor and USB cable. These components do not come into contact with your breast milk. You can wipe those parts clean with a soft cloth.
- ▣ Be gentle when cleaning the valve and the silicone diaphragm. The suction may drop if the valve or silicone diaphragm is torn or loose.

NOTE!

This product has a built-in lithium battery. When the battery indicator flashes 3 times, we recommend that you use a certified 5V⁼⁼1A adapter to charge the pump motor as soon as possible.

VI. Storing breast milk

Below you find guidelines for storing breast milk:

Room temperature 16 - 25°C	Refrigerator 4°C or colder	Freezer -18°C or colder	Thawed breast milk
Up to 4 hours.	Up to 3 days.	Up to 6-12 months.	Room temperature: Up to 2 hours. Refrigerator: Up to 24 hours Do not refreeze!

These guidelines for storage and thawing of breast milk are recommendations. Contact your lactation consultant or breastfeeding specialist for further information.

WARNING!

- ❑ Never refreeze thawed breast milk.
- ❑ Never add fresh breast milk to frozen breast milk.

1. Freezing

Repeated freeze/thaw cycles destroy the structure of milk. Therefore, never refreeze breast milk.

Freeze expressed breast milk in plastic bottles or milk storage bags. Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.

NOTE!

Always label the bottles or milk bags with the date of expression.

2. Thawing

Do not thaw or warm breast milk in a microwave or a pan with boiling water. This prevents burns and helps preserve important components of the breast milk. To preserve these breast milk components, thaw the milk overnight in the refrigerator. Alternatively, hold the bottle or bag under warm water (max.37°C).

VII. Ordering accessories

To buy accessories or spare parts, please contact your manufacturer.

VIII. Troubleshooting

Problem	Solution
<i>Experience pain when using breast pump.</i>	<ul style="list-style-type: none">• Choose a suction level which makes you feel comfortable.• Check if the silicone flange is undamaged.• Ensure that your nipple is placed in the centre of the cushion.• If the pain persists, stop using it.
<i>No suction / The suction is too low.</i>	<ul style="list-style-type: none">• If possible, try a higher suction level.• Check if the silicone flange is assembled correctly.• Check if the white valve is undamaged and assembled correctly.• Check if the other parts of breast pump are intact and assembled correctly.• Make sure that the pump is positioned correctly on the breast to allow a seal between breast pump and breast.
<i>Suction is too strong.</i>	<ul style="list-style-type: none">• If possible, try a lower suction level. When you express for the first time, start with the default suction level and increase/decrease the level when needed. During different expression sessions, different levels may feel most comfortable for you.• Check if the silicone flange is undamaged.
<i>I express little/no milk.</i>	<ul style="list-style-type: none">• If you are using the Sobia breast pump for the first time, you may need to acquire some practice before you are able to express any milk.
<i>Condensation appears in tubing</i>	<ul style="list-style-type: none">• Rinse out the tubing. Shake out water droplets and hang tubing to air-dry.
<i>Milk gets into tubing</i>	<ul style="list-style-type: none">• Turn off the pump immediately. Remove and clean the tubing.• Shake out water droplets and hang to air-dry.

Difficulty to start/stop the motor	<ul style="list-style-type: none"> To touchscreen is sensitive and you need to keep your finger for at least 2 seconds on the start-stop touch button.
Experience pain in nipples during pumping	<ul style="list-style-type: none"> You can use coconut oil while pumping to soothe and help repair sore, cracked nipples. Simply apply a small amount of coconut oil directly to the nipple.
The motor is not pumping vacuum	<ul style="list-style-type: none"> Make sure that under the motor unit the rubber band is good in place and not damaged.

IX. Supplemental information

Below are some common breastfeeding related conditions. If you experience any of these symptoms, contact your healthcare professional or breastfeeding specialist.

Breastfeeding related conditions	Description
<i>Pain sensation</i>	Pain felt on the breast or nipple because the suction is above your comfortable level.
<i>Sore nipples</i>	Persistent pain in the nipples at the beginning, throughout or between the different pumping sessions.
<i>Engorgement</i>	Swelling of the breast. The breast may feel hard, lumpy and tender. It may include erythema (redness) of the breast area and fever. Usually occurs during the first days of lactation.
<i>Bruise, thrombus</i>	A reddish-purple discoloration. When a bruise fades, it may become green and brown.
<i>Nipple trauma (injured tissue on nipple)</i>	<ul style="list-style-type: none"> Cracked nipples or fissures. Able to peel off skin on nipples. This normally occurs in combination with cracked nipples and/or blisters. Nipple tear Blister Cracked or torn nipples can lead to bleeding of the affected area.
<i>Clogged mammary ducts</i>	A red, tender lump on the breast. May include erythema (redness) of the breast area. Can lead to mastitis (breast inflammation) and fever if left untreated.

X. Storage conditions

Keep the breast pump out of direct sunlight as long exposure of sunlight may cause discoloration to the breast pump. Store the breast pump and accessories in a clean and dry place.

XI. Technical information

1. Specifications

Model	M6
Silicone Flange Size	27mm or 24mm Optional
Bottle Capacity	about 180ml
Battery Capacity	900mAh the battery has constant
Voltage Function.	A reddish-purple discoloration. When a bruise fades, it may become green and brown.
Net Weight	220g
Gross Weight	400g
Size	140mm X 115mm X 70mm
Vacuum	max 280-300mmHg
Mode	Massage/ Expression/Auto/lactation
Levels	12 levels adjustment
Power Input	DC5V1A
Power	4W
Charging Port	USB Type-C
Charging Time	about 2.5 hours
Usage Time	about 100 minutes
Shut Off	20 minutes auto shut off
Wat hours per battery	3.33Wh

Service life	500 hours
lithium content	about 16.6g
Lithium battery voltage	3.7V
Number of batteries	1 pcs
Sound	about 50 dB
Certificates	CE, FCC, UKCA, ROHS

2. Different modes and suction powers

Massage Mode				
Levels	Vacuum (MmHg)	Inhalation Time (Ms)	Released Time (Ms)	Sucking Per Minute
1	60	280	280	107
2	74	320	320	94
3	88	350	350	86
4	102	380	380	79
5	116	400	400	75
6	130	420	420	71
7	144	440	440	68
8	150	460	460	65
9	164	480	480	63
10	178	500	500	60
11	192	520	520	58
12	210	540	540	56

Note: One suction and one release for a cycle

Expression Mode

<i>Levels</i>	<i>Vacuum (MmHg)</i>	<i>Inhalation Time (Ms)</i>	<i>Released Time (Ms)</i>	<i>Sucking Per Minute</i>
1	120	600	600	50
2	135	660	600	48
3	150	720	600	45
4	165	780	700	41
5	180	840	700	39
6	195	930	700	37
7	210	990	900	32
8	225	1060	900	31
9	240	1120	900	30
10	255	1200	1000	27
11	270	1280	1000	26
12	285	1320	1000	26

Note: One suction and one release for a cycle

Lactation Mode

<i>Levels</i>	<i>Vacuum (MmHg)</i>	<i>Inhalation Time (Ms)</i>	<i>Released Time (Ms)</i>	<i>Sucking Per Minute</i>
1	175	1050	800	32
2	185	1110	800	31
3	195	1180	800	30
4	205	1240	800	29
5	215	1300	800	29
6	225	1350	800	28
7	235	1380	800	28
8	245	1450	800	27
9	255	1550	800	26
10	265	1700	800	24
11	275	1900	800	22
12	285	2000	800	21

Note: One suction and one release for a cycle

Auto Mode

<i>Levels</i>	<i>Vacuum (MmHg)</i>	<i>Inhalation Time (Ms)</i>	<i>Released Time (Ms)</i>
6	5 times massage mode + 10 times expression mode cycle		
7			
8			
9			
10			
11			
12			

Note: One suction and one release for a cycle

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