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CE 0344

- **Ovulation test with a high sensitivity.**
- **For self testing at home.**

Instructions for use Sensitest Ovulation test dipstick (LH)

Carefully read the instructions for use before you start testing. The Sensitest ovulation test is a simple way of ascertaining a woman's 2 to 3 fertile days in the month. Some 24-48 hours before ovulation commences, the quantity of Luteinizing Hormone (LH) increases in the urine. Immediately before ovulation, the measurable quantity of this hormone in the urine rises sharply. This LH peak lasts for 1 or 2 days only. The Sensitest ovulation test will indicate 2 clear lines (image 3). After the LH content in the urine is at its highest, ovulation takes place within 24 to 48 hours. You have the best chance to become pregnant during this period, following sexual intercourse. To find this LH peak, you will have to test for several consecutive days. You will first have to determine your cycle length. Take an average of the past cycles. The best day for starting to test can be read from the table (image 4). You count from the first day of your menstruation which is day 1. Continue testing until you can read the LH peak from the ovulation test. Ovulation then occurs within 24 to 48 hours. After the LH peak you can stop using the ovulation test.

Timing The test can be carried out at any time of day. Preferably **not** with morning urine. However, it is important that testing takes place at about the same time on all consecutive days.

What you need before you start testing

1. A sealed foil package with a Sensitest ovulation test dipstick. Do not use an ovulation test from an opened or punctured pack.
2. A clean, dry receptacle or plastic cup.
3. Urine from the woman whose ovulation must be confirmed.

The test in 7 steps

1. Take a closed pack and check that it indicates "Ovulation test". Do not use a test that has passed its shelf life.
2. Collect a little urine in a clean cup.
3. Open the pack with the test with dry, clean hands.
4. Hold the strip at the end where the red letters LH appear on the stick and dip the other end into the receptacle with urine for about 10 seconds.
5. This part may not be dipped beyond the line near "MAX" (see image).
6. Place the test on a smooth, dry base.
7. Wait 5 - 10 minutes before reading the result. It is important that the background is no longer discoloured. However, do not wait more than 30 minutes before reading the result.

Test result

- The result is **negative** if a single line appears (image 1). This is the control line (C) to see whether you have carried out the test correctly. The test line (T) is not visible or appears as a light pink line (image 1 and 2). The body has made insufficient LH for an LH peak to be reached. Both test results can be interpreted as negative.
- The result is **positive** if two lines appear (image 3). You can clearly see a control line (C) and a test line (T). Both lines are of the same colour or the test line (T) is darker than the control line (C). The level of LH in your body is at its highest at this point.

Result no lines

- If **no lines** are visible, the test has probably been carried out incorrectly. The dipstick has possibly absorbed too little urine. Hold the tip of the dipstick in the urine again for a few seconds.

If the result is negative

If the ovulation test cannot show an LH peak after repeatedly testing on consecutive days, it is possible that you have carried the test out on the wrong days. Calculate the average length of your cycle again. On your next cycle begin testing two days earlier than the table (image 4) indicates. If you obtain negative results after repeated attempts, we advise you to contact your family doctor or gynaecologist. Consult a doctor if irregular or unusually long cycles are experienced.

If the result is positive

If the test indicates that the LH level in your urine is at its highest, ovulation will take place within 24 to 48 hours. You have the best chance of becoming pregnant during this period, following sexual intercourse.

Restriction 1: Reliability of the ovulation test

With a 99% reliability, the Sensitest ovulation test dipstick is just as reliable as a comparable test by your family doctor or gynaecologist. However, your personal medical situation may influence the result of this test. Women with polycystic ovary syndrome may obtain an incorrect positive result. In case of doubt, consult your family doctor or gynaecologist.

Restriction 2: Do not use alcohol

Alcohol may influence the test result. Do not use ovulation tests after taking alcohol.

Restriction 3: Do not use as a contraception method

Do not use the ovulation test as a contraception method. Since sperm can remain active for a number of days, the chance of becoming pregnant remains.

Restriction 4: Applying the test

This test may be used only with collected urine. If you require an ovulation test to hold in the urine stream, use the Sensitest ovulation test midstream.

Restriction 5: Do not drink too much

Drinking large amounts of water or other liquids like coffee or tea could dilute your urine and produce light or unclear test results. You are advised to refrain from drinking large quantities of liquids during your testing period. Preferably stop drinking 2 hours before testing.


Restriction 6: Ovulation test and the pregnancy hormone (hCG)

The ovulation test gives a positive test result on the pregnancy hormone hCG. If you have a positive test result on your ovulation test during 4 or 5 consecutive days with a dark purple T-line (like image 3), you could consider doing a pregnancy test first. Up to 9 weeks after a miscarriage you could expect positive test results on an ovulation test as well.

Restriction 7: Sensitivity

Each type or brand ovulation test has its own sensitivity. Do not use different types or brands of ovulation tests simultaneously, because test results may vary.

- **Image 1 and 2: No LH peak**

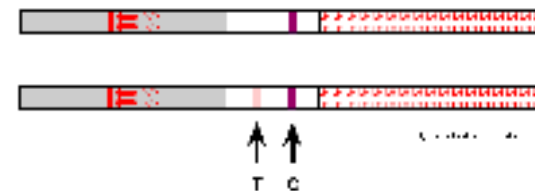- **Image 3: LH peak**

	For in vitro diagnostic use. For external use only, do not swallow. Keep out of reach of children.
	Read instructions for use before testing.
	For non recurring use.
	Keep between 4 and 30 degrees Celsius in a sealed pouch. Protect from heat. Do not freeze.
	Protect from humidity.
	Protect from direct sunlight.
	Final use-by date.
	Manufactured by: Sensitest Distributieweg 54 2645 EJ Delfgauw The Netherlands Tel +31(0)153642121 - www.sensitest.nl

- **Image 4**

Length of your Cycle	Start testing on day
22	5
23	6
24	7
25	8
26	9
27	10
28	11
29	12
30	13
31	14
32	15
33	16
34	17
35	18
36	19
37	20
38	21
39	22