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Manual MultiMotion M5

Version 1.2 July 2018

BEFORE USING THIS WHEELCHAIR, YOU AS WELL AS YOUR COMPANION SHOULD CAREFULLY READ THIS MANUAL.

General introduction

It is important that you carefully read this manual before using the wheelchair. The manual contains important information about the safe use and maintenance of your wheelchair.

The safety instructions in this manual are general instructions that must be seen as global guidelines. You may develop your own ways for common actions. However, we would like to advise you to call on an expert for assistance in developing safe and effective techniques related to your daily activities and your physical capabilities.

Your new wheelchair needs frequent and schematic maintenance. You can do a lot of this maintenance yourself. However, we would advise you to have your wheelchair checked by an expert once a year.

SAFETY PRECAUTIONS

MultiMotion is not responsible for injuries to the user or damage to property if the recommendations and warnings have not been followed when using the wheelchair. The wheelchair is, if the handling rules as described in this manual are followed, an extremely safe and stable product. However, it is possible that, if the user advice is not correctly observed, dangerous situations may arise.

General Safety Instructions

Protect your purchase by having your wheelchair checked regularly. If a part of your wheelchair does not function properly, it is possible that a dangerous situation arises. Periodic inspection, correct adjustment of your wheelchair and timely replacement of broken and worn parts will result in years of trouble-free use of your wheelchair. A qualified MultiMotion dealer, who will only use MultiMotion replacement parts, will ensure that the long service life is guaranteed through good and preventive maintenance. Look for all dealers at www.multi-motion.nl/dealer-pagina

General warning

You must keep your wheelchair in a good condition to ensure safety in use and handling.

Warnings for safe use:

- Do not use your wheelchair on streets or motorways, only on the sidewalks.
- The transport of your wheelchair is in all vehicles, including vehicles that have been specially adapted and / or developed for the transport of your wheelchair, at your own risk. We accept no liability for this. If you still want to transport your wheelchair, we advise you to contact the carrier, your supplier and / or a recognized manufacturer of taxi fixation systems to choose an appropriate system for your wheelchair and situation;
- Your wheelchair can not be used as a seat in a motor vehicle. Always use the permitted vehicle seat and use the required motor vehicle restrictions when you are in a motor vehicle. Transporting your wheelchair in a motor vehicle is at your own risk;
- Do not attempt to climb slopes without installed anti-tip wheels, and never try to get on or off sidewalks without assistance;
- Do NOT lean over the back of the wheelchair. This can result in the wheelchair tilting;

Our advice

Sit in the wheelchair

Before you sit in the wheelchair, it must be put on the brake on both sides. The brakes are against the rear wheels. Fold up the foot plates or swing them out. You stand against the wheelchair with your knees. Look for support with the armrests and sit as far back as possible. Place your feet on the footrests.

Riding with the wheelchair

A wheelchair with large rear wheels can be moved by the user himself by pulling the hoops forward. To steer you pull one hoop forward more than the other. Bear in mind that all pavements run down to the road and that a wheelchair that is not on the brake can drive.

Sidewalks

Driving up a sidewalk is easiest when the person pushing the wheelchair is driving up to the curb, then putting the foot on the wheelchair's head, which causes the front wheels to rise and then the front wheels will be put on the sidewalk and continues to move the rear wheels on the pavement. Driving down a pavement is best done by turning the wheelchair and going backwards on the sidewalk. Always take a curb at right angles.

Get up from the wheelchair

Place the wheelchair on the brake on both sides. Fold up the foot plates or swing them out. User puts the feet on the ground and presses up on both armrests.

Folding the wheelchair

If there is a cushion in it, take it out first. Pull the center of the mattress up with both hands until it stops. Do not forget to fold up the footplates first or remove the footrests completely.

Unfolding the wheelchair

Grasp the tubes on both sides of the seat and push them apart until it stops.

Flipping of the back support

The backrest of this wheelchair can be folded backwards by pressing the 2 unlocking clips on the back of the backrest and gently folding the backrest backwards. This way the wheelchair is folded extra compactly.

The recoil of the back support

Carefully pull the backrest up until it is locked by the locking pins. A clear click indicates that the back support is locked.

Removing the wheels

This wheelchair is equipped with so-called Quick-release wheels. This means that the wheels can easily be removed by means of a push button in the axle of the wheel. Press the button into the shaft and carefully remove the wheel. Hold the wheelchair so that it does not fall over.

Replacing the wheels

When you put the wheels back, it is important that you keep the button on the axles pressed when the wheel is pushed over the axle. Check that the wheel is properly secured, if this is the case it can never come loose unless the button on the wheel axle is pressed again.

Take the wheelchair in the car

The wheelchair can be transported in most cars. By removing the footrests, the wheelchair becomes smaller.

Use of the brakes

This wheelchair is equipped with brake levers on the push handles. With this the wheelchair can also be put on the parking position, this is done by squeezing the brakes and pushing the clip on the start of the brake lever.

Cleaning the wheelchair

It is best to clean the wheelchair with a light household soapy water.

User weight

The maximum user weight is 130 kg.

Service and Warranty

MultiMotion is not responsible for injuries to the user or damage to property if the recommendations and warnings have not been followed when using the wheelchair. The wheelchair is, if the handling rules as described in this manual are followed, an extremely safe and stable product. However, it is possible that, if the user advice is not correctly observed, dangerous situations may arise.

This wheelchair has a warranty period of 24 months. For warranty and service please contact the authorized dealer where you purchased your product. If you still have (technical) problems with the product after this period of 24 months, you can also contact the authorized dealer for a possible solution.

If it does not happen that you are not helped satisfactorily in terms of warranty and service, please send an email to MultiMotion Customer Service department at the address below: info@multi-motion.nl

Distributor:

MultiMotion
Weezenhof 61-58
6536 AM Nijmegen
Nederland
Tel. 024-6794583

Manufacturer:

Zhenjiang Max Medical Appliance Co., Ltd
NO703, Gaochuang yuan, Dingmao jing 12 road
Zhenjiang, Jiangsu China

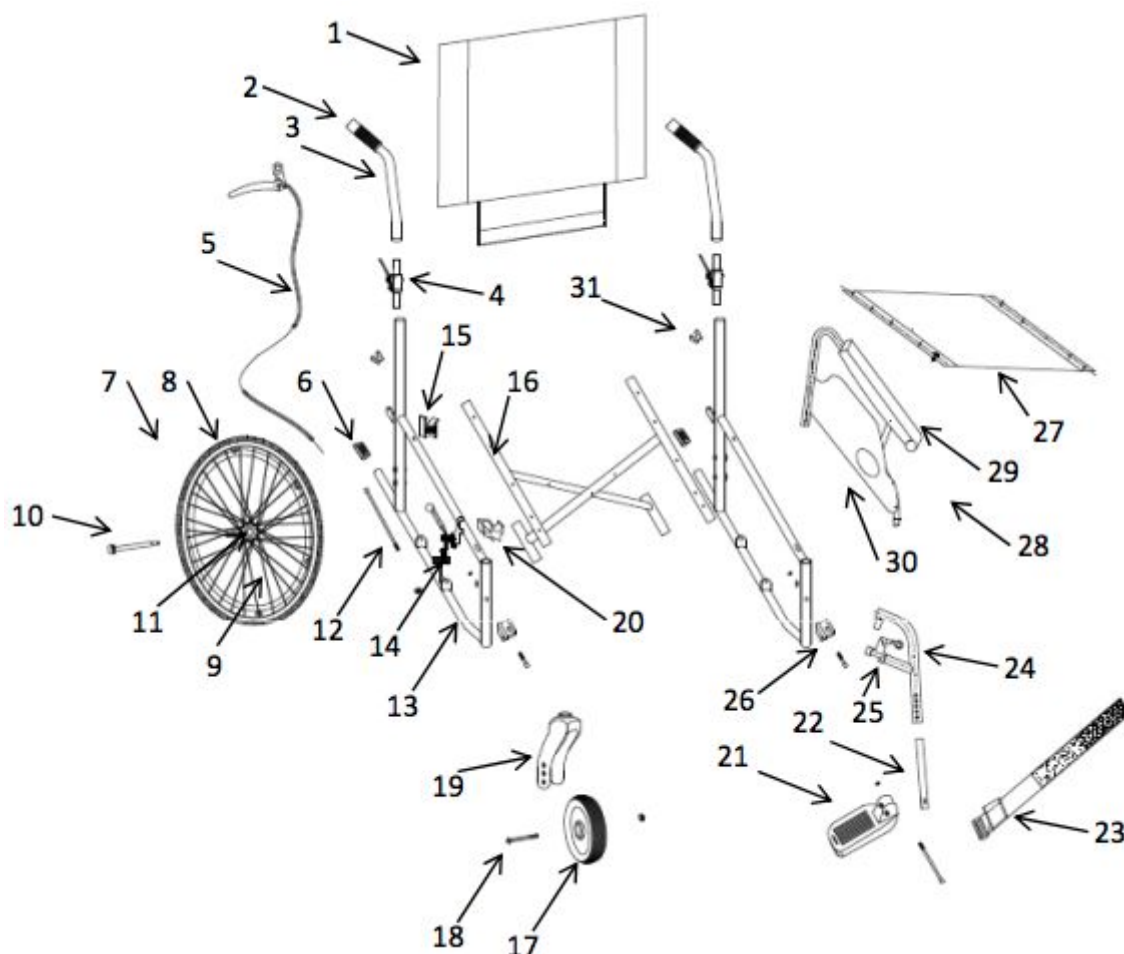
EU representative:

Lotus NL B.V.
Koningin Julianaplein 10
2595AA The Hague
Netherlands

MultiMotion has a “Declaration of Conformity”

Parts list

- | | |
|--------------------------------|--------------------------------------|
| 1. Back Upholstery (Nylon) | 19. Front wheel fork (Plastic) |
| 2. Handgrip (Plastic) | 20. Front seat holder (Plastic) |
| 3. Tube handgrip (Aluminium) | 21. Footrest plate (Plastic) |
| 4. Fold mechanism (Aluminium) | 22. Footrest tube (Steel) |
| 5. Brake cable (Plastic) | 23. Heel strap (Nylon) |
| 6. Tipping Sleeve (Plastic) | 24. Footrest frame (Aluminium) |
| 7. Rear wheel complete | 25. Footrest Release Latch (Plastic) |
| 8. Rear wheel Tire (PU) | 26. Footrest suspension (Plastic) |
| 9. Rear wheel Spoke (Steel) | 27. Seat upholstery (Nylon) |
| 10. Rear wheel Bolt (Steel) | 28. Armrest complete |
| 11. Rear wheel Bearing (Steel) | 29. Armrest (PU) |
| 12. Brake cable (PVC) | 30. Armrest Guard (Plastic) |
| 13. Frame (Aluminium) | 31. Armrest fixation (Plastic) |
| 14. Brake (Aluminium) | |
| 15. Rear seat holder(Plastic) | |
| 16. Cross frame (Aluminium) | |
| 17. Front wheel tire (PU) | |
| 18. Front wheel bolt (Steel) | |

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