dutchblue.com

LIFTIES- /RECLINIES CHAIRIES

- User Manual** | Lift and recliner chair
Gebrauchsanweisung | elektr. Aufsteh- und Liegesessel
Mode d'emploi | Fauteuil électrique inclinable
Gebruikershandleiding | Elektrische opstaan en lig fauteuil
Instrucciones de uso | Silla eléctrica elevable y reclinable
Istruzioni per uso | Poltrona elettrica elevabile e reclinabile alzapersona
Instrukcja użytkowania | Rozkładany fotel relaksacyjny do leżenia
Manual de utilização | Cadeira elevável e reclinável
Manual de utilizare | Scaun ascensor și reclinabil
Εγχειρίδιο χρήστη | Πολυθρόνα ανύψωσης με κλίση
Használati Utasítás | Emelő- és fekvőszék
Priručnik za korisnike | Sjedežnica i naslonjača
Ръководство за употреба | Електрически стол за повдигане

REF 5026-B, 5027-R, 5028-Y

LIFTIES- / RECLINIES CHAIRIES

English	User Manual	4
Deutsch	Gebrauchsanweisung	14
Français	Mode d'emploi	24
Nederlands	Gebruikershandleiding	34
Español	Instrucciones de uso	44
Italiano	Istruzioni per uso	54
Polski	Instrukcja użytkowania	64
Português	Manual de Utilização	74
Română	Manual de utilizare	84
Ελληνικά	Εγχειρίδιο χρήστη	94
Magyar	Használati utasítás	104
Hrvatski	Priručnik za korisnike	114
Български	Ръководство за употреба	124

ENGLISH

Before using the electrical lift and recliner chair, read this manual and save for future reference.

INTRODUCTION

This is an electrical lift and recliner chair designed to help people on and off the chair. Please read the operation manual thoroughly for the correct methods of usage, especially the notes on Safety.

This manual should be kept available for future reference after you have read it through.

CAUTION!

- ⚠ **Do not** attempt to adjust or operate the electrical lift and recliner chair without carefully reading and understanding all instructions in this manual.
- ⚠ Check ALL parts for shipping damages before use. **Do not use** in case of any shipping damages and please contact your healthcare provider for further instructions.
- ⚠ If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.
- ⚠ The user should **always** consult their physician or therapist to determine proper adjustment and usage.

SAFETY

Warnings!

- ⚠ The chair must be assembled correctly according to the Assembly instruction enclosed.
- ⚠ The manufacturers of the chair, their agents and distributors are not liable for any damages caused by improper assembly or usage of the chair.
- ⚠ The chair must be placed on a flat surface.
- ⚠ **Do not** use the chair in humid or dusty environment, as it may cause electrical shock or damage to the product. The chair must be in a dry room.
- ⚠ **Do not** use the chair in room temperature higher than 40°C.
- ⚠ **Do not** expose the chair to heaters, stoves or direct sunlight.
- ⚠ **Do not** use the chair outdoors. Indoor use only. Avoid exposure to rain, snow, ice, salt, or water. Maintain and store in a clean and dry environment.
- ⚠ When the chair is not in use, disconnect the power cord from the main supply to avoid overheating.
- ⚠ **Do not** rest or place any heavy object on the armrests, footrest or the back of the chair.
Pinch/crush points - prevent personal injury and equipment damage. **Do not** place appendages or objects in path of the moving parts.
- ⚠ **Do not** jump or play on the chair.
- ⚠ **Do not** sit or stand on the footrest.

ENGLISH

- △ **Do not** sleep on the chair during operation.
- △ Children must be supervised by an adult when using the lift chair, This chair is **not** intended for children.
- △ Keep pets away when the lift chair is in motion.
- △ Keep all parts of your body away from the lift and scissor mechanism.
- △ Check that the voltage corresponds to the specifications indicated for the chair.
- △ Electrical connection must be made in accordance with the local requirements.
- △ **Do not** disconnect the plug from the electrical outlet roughly.
- △ **Do not** connect or disconnect the plug from the electrical outlet with wet hands. It may result in electrical shock or machine damage.
- △ When disconnecting the plug, pull it out by holding the plug and not by the cord.
- △ **Do not** place the AC/DC adaptor cord under the chair as it may damage the cord and result in fire or electrical shock.
- △ **Do not** let the AC/DC cord hang down from the chair.
- △ **Do not** use the chair if the AC/DC cord or plug is damaged, or if the electrical outlet is excessively loose. Contact a professional for inspection.
- △ Prevent personal injury and equipment damage. **Do not** connect an extension cord to the AC/DC converter or the battery charge.
- △ If necessary, the power cable may be replaced with an identical one, which must only be replaced by a qualified technician.
- △ **Do not** touch the surface of the plug immediately after use, as it may become very hot.
- △ **Do not** operate the remote control with wet hands, sharp or heavy objects.
- △ **Do not** drop the remote control.
- △ **Do not** step or place heavy objects on the remote control wire.
- △ **Do not** put your fingers and hands in between the seat and the footrest, when the chair is reclining or when the chair is returning back from the reclined position to sitting position as it may cause injuries to the fingers or hands.
- △ **Do not** modify, add, remove, or disable any feature, part, or functions of the lift chair. Unauthorized modifications may result in personal injury and/or damage to the lift chair.
- △ This chair must be used in accordance with the instruction provided.
- △ This chair is **not** for professional use.
- △ The manufacturers of this chair, their agents and distributors are not liable for any damages to person(s) or property(-ies) resulting from improper use and misuse of the chair.

ENGLISH

 **Caution:** Should you use the chair in a cold room, do not increase the room temperature abruptly. It is recommended that you increase the temperature gradually to a normal level. In case the chair is stored in a cold place and should be brought into a warmer place, it is advisable to use it one hour later. This is necessary as its performance may not be at its optimum level as it may have dew on the mechanical parts. When a cold machine is suddenly used in a warm room, this may cause damage to the machine.

When the chair should not be used

In the following events, please switch the power OFF immediately and disconnect the AC/DC power cord from the electrical outlet.

-  If the chair is accidentally spilt with water. This may result in electrical shock.
-  If the upholstery is torn and the inside mechanism is exposed.
-  If you notice any abnormality in the main functioning or other abnormal condition during operation.
-  If there is a power failure, damage to the product may be caused when power is restored unexpectedly.
-  If there are severe lightnings.

Note: Check for tears to the upholstery cloth regularly.

SPECIFICATION

 **Warning!** Exceeding the weight capacity may result in body injury and/or damage to the lift chair.

 **Warning!** Prevent the risk of electrical shock, fire, falls and/or being pinched. Follow all instructions provided. Not following the instructions may result in personal injury and/or damage to the lift chair.

Weight The weight of the chair is 40 kg.
 User weight capacity is 135 kg.

Power AC 100-240
 DC 29 V ~ 50 Hz 70W

Working temperature -10 °C to 40 °C /50 °F to 104 °F/

ENGLISH

ASSEMBLY

The chair is packed in semi knock-down form consisting of the back and the seat. Assemble the chair according to the instructions given in the Assembly instruction section.

Assembly Instruction:


Note: Do not assemble the chair if any of the parts of the chair are damaged. Check all parts carefully before assembling the chair. In case of any issues, return to the vendor.

Step 1:

The chair is packed into two packages (chair base and chair back). Please, take them and the accessories bag out of the packages.

Step 2:

Put the armrests at the sides of the seat. Make sure the metal bracket slips into the metal bracket of the seat well.

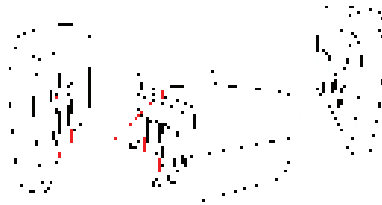Step 3:

Put the chair back on top of the chair base.

Make sure the metal bracket slips into the metal bracket of the chair base.

Press the metal brackets until you hear the “click” sound of the metal bracket installed properly.


ENGLISH

Step 4:

Find the motor wire under the chair base.

Connect the adaptor to the motor wiring and then connect the plug wiring to the adaptor.


Take the hand control out from the side pocket of the seat.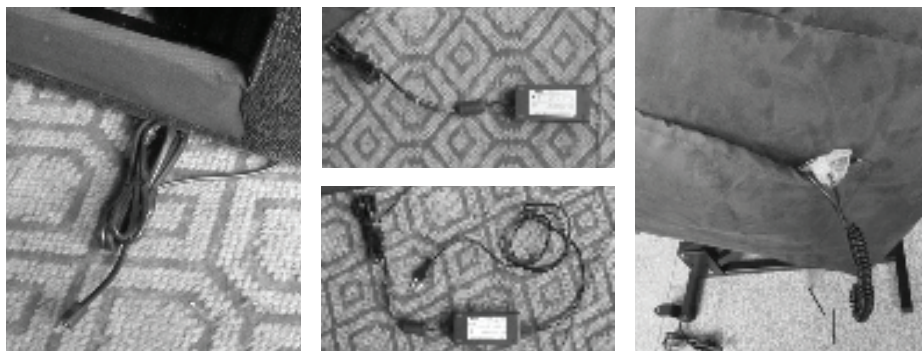

Once the chair is assembled according to the Assembly instructions, place the chair on a flat surface in a dry room. The chair is now ready for use.

- ⚠ **Note:** Make sure all the wires are properly connected, if not the chair will not function properly.
- ⚠ If the external transformer or hand control is damaged, immediately place it in a plastic bag and call your authorized dealer for instructions.
- ⚠ Electromagnetic and Radio Frequency Interference (EMI/RFI).
According to the EMC standards, this lift chair is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. This product has passed immunity testing.

- (1) Connector for the motor and wiring.
- (2) Connector for wiring and adaptor.
- (3) Connector for adaptor wiring and plug wiring.
- (4) Connector for the motor and the hand control.

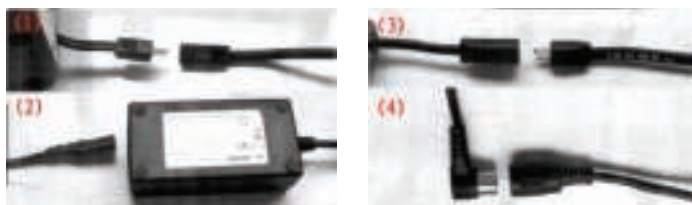ENGLISH

PLACING OF THE CHAIR

- Allow sufficient space in front and behind the chair for the reclining movement.
- Make sure there is an allowance of more than 55 cm behind the chair and more than 31 cm in front of the chair.
- Place the chair near a properly wired standard wall outlet.
- Place the chair on a flat surface/floor in a dry room.
- Adjust the leg levelers to keep the chair stable.
- Turn the leg levelers clockwise to raise the chair and counter-clockwise to lower the chair.
- Lay the low voltage connection cable, located at the back of the chair, where it will not be pinched between the wooden frame and the lifting mechanism. If the low voltage connection cable is not already connected to the external transformer, connect it.
- Ensure the external transformer, power cord, and the low voltage connection cable are not damaged or wet.

Caution:

- Make sure there is no obstruction behind or in front of the chair for a safe recline and lift during operation.
- Plug the electrical chord directly into the outlet. DON'T use an extension cord!
- No body parts or objects should be placed between the chair and the footrest.

Warning!

- Before attaching the back of the lift chair, place the hand control in center of the cushion. Improper hand control cable routing can lead to equipment damage & malfunction.

ENGLISH

USAGE

RECLINE

- Press the Recline button and the chair will recline.
- Once you have reached the desired position release the button and the chair will stop reclining.

RECLINE TO SIT

- Press the Sit button to return the chair from reclined position to sitting position.

Important: Do not put your hands in between the seat and the footrest or in between the metals parts when the chair is reclining or when the chair is returning back from reclined position as it may cause injuries to the fingers or the hands.

LIFT

- Press the Sit button to lift the chair from sitting position. Once you have reached the desired lifted position release the button and the chair will stop lifting.

LIFT TO SIT

- Press the Recline button to return the chair from lifted position to sitting position.

TO SIT AND STAND

- Release and hold the up/down switch in the “↑” position to raise the lift chair to standing position. Back into the lift chair and sit in the center of the seat. Use the armrests of the lift chair for support if necessary.
- Release and hold the up/down switch in the “↓” position until comfortable.
- To return to standing position, push and hold the up/down switch in the “↑” position. The lift chair will begin to rise. Release the switch when the lift chair reaches a height where you can stand up comfortably.

TO SIT AND RECLINE

- Release and hold the up/down switch in the “↓” position to reclining position until comfortable.
- Release and hold the up/down switch in the “↑” position to return the lift chair to the sitting position.
- To return to standing position, push and hold the up/down switch in the “↑” position. The lift chair will begin to rise. Release the switch when the lift chair reaches a height where you can stand up comfortably.

THERMAL SHUTOFF

- The lift chair is equipped with an internal thermal shut off switch located inside the external transformer to prevent the motor control box from overheating. If the motor stops during the lifting movement, allow the lift chair to remain in this stationary position for 10 minutes. This will allow the motor control box to cool down and reset itself. After cooling down, the normal use of the lift chair can be resumed.

ENGLISH

BATTERY BACKUP

- During a power failure, the lift chair's battery backup system will automatically switch on. This system is designed to take you from reclined to sitting position only. The battery backup system is powered by two 9V alkaline batteries (not included).
- Always make sure the lift chair's battery backup system is equipped with two charged 9V alkaline batteries, since the system does not recharge itself. Batteries are assumed charged when 9V alkaline batteries are replaced every time the battery backup system is activated during a power failure, or once a year if the battery backup system has not been activated.
- If you unplug the lift chair for an extended period of time (more than one hour), remove the batteries. The lift chair will draw power from the batteries even when it is not in use.

Caution:

- When sitting down in the chair, never "drop" into it if it is partially raised.
- The external transformer is packaged independently from the lift chair. Please DON'T discard any packaging before locating the external transformer.
- Make sure the external transformer is plugged into a properly wired outlet and check the circuit breaker box.
- Make sure the connection cable from the chair is properly plugged into the external transformer.
- Make sure the hand control and motor are plugged in properly.
- Check if the motor is plugged into the low voltage connection cable that goes into the external transformer.
- Check the up/down switch of the hand control to make sure there is a smooth movement from the "↓" and "↑" position. If the switch sticks in either position, the hand control may need to be replaced.
- If the lift chair rocks from corner to corner after you position it, the floor may be uneven or the carpet may be affecting the chair position. Adjust the leg levelers at the area where the chair is rocking.

ENGLISH

PRODUCT CARE AND STORAGE

How to store

- Wipe the dust and dirt off. Keep the chair clean and away from places with high humidity.
- If the chair is not in use for a long period of time, cover it with a piece of clean cloth to protect it from dust. Store in a dry room.
- Do not store the chair under direct sunlight or in places with high temperatures. This may result in the fading of the fabric color.

Self-maintenance

- Before cleaning, make sure the main switch is turned off and the AC plug is removed from the electrical outlet.

Main Body

- Synthetic and very mild soap can be used to clean the plastic, pipe and upholstery of the chair. Carefully wipe away all traces from the soap with dry cloth.
- To clean a dry spot, dip a piece of soft cloth into some warm soap water, squeeze the cloth dry, and apply it to the spot.
- Do not use cleaning agents like benzene or thinner, or any other detergents which may result in fading of the fabric color and scratching of the upholstery.
- Do not remove the upholstery for washing. Ironing of upholstery is strictly prohibited.

⚠ Fabric should not be in direct contact with any type of heating appliances. Personal injury or fire damage may occur.

Remote Controller Unit

- Use only dry cloth for cleaning of the unit, as usage of wet cloth may result in machine damage.
- Periodically check the hand control and all the power chords for visible damages.
- Keep the hand control away from all heated surfaces.
- If the hand control gets wet, unplug the transformer from the wall outlet and dry thoroughly before use. Ensure the hand control is out of the way before sitting in the chair.

⚠ Caution:

- If the circuit (controller) doesn't work well, please contact the supplier.
- DON'T repair it by yourself.

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- △ If the chair is accidentally spilt with water. This may result in electrical shock.
- △ If the upholstery is torn and the inside mechanism is exposed.
- △ If you notice any abnormality in the main functioning or other abnormal condition during operation.
- △ If there is a power failure, damage to the product may be caused when power is restored unexpectedly.
- △ If there are severe lightnings.

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Main Body

- Synthetic and very mild soap can be used to clean the plastic, pipe and upholstery of the chair. Carefully wipe away all traces from the soap with dry cloth.
 - To clean a dry spot, dip a piece of soft cloth into some warm soap water, squeeze the cloth dry, and apply it to the spot.
 - Do not use cleaning agents like benzene or thinner, or any other detergents which may result in fading of the fabric color and scratching of the upholstery.
 - Do not remove the upholstery for washing. Ironing of upholstery is strictly prohibited.
- ⚠ Fabric should not be in direct contact with any type of heating appliances. Personal injury or fire damage may occur.

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dutchblue makes care affordable

About Dutchblue:

We were born with a dream: We want to make care affordable for those who need it. Dutchblue strives to improve the daily work of as much people in care as possible. This will be achieved by offering a carefully selected, high quality, product range at very low prices.

How can dutchblue be so advantageous?

- We have low prices because we produce in large numbers
- We have a small product range that is carefully selected. This makes us fast and flexible, hence cheaper.
- We deliver directly to our customers. Therefore there is no one in between that will raise the price.
- We don't work with expensive external employees. This lowers the operatin expenses and thereby the prices.

dutchblue.com

Bulgaria

Parizhka Komuna 26
9000 Varna

Italia

Via Nizza 262
10126 Torino

Deutschland

Subbelrather Str. 15a
50823 Köln

Polska

ul. Puławska 12, lok. 3,
02-566 Warszawa

Österreich

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España

Calle Sierpes, 16
46940 Manises

France

57 rue d'Amsterdam
75008 Paris

België / Belgique

Square de Meeûs 37
1000 Brussels

Nederland

Faradayweg 20
2627 AX Delft

Schweiz / Suisse

Scheitlin-Papier AG
Industriestrasse 20
9300 Wittenbach