Optimera®

Mesh Vernevelaar / Mesh Nebulizer / Mesh Vernebler

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Introduction.....	22
1. Information about the product	23
1.1 Description about the product and accessories....	23
1.2 Product components	24
2. Operation of the device.....	24
3. First time use	25
4. Cleaning and disinfecting.....	30
5. Technical specifications.....	34
6. Cleaning and disinfecting.....	34
7. Warranty	37
8. Safety Information & Warnings	38

Introduction

Thank you for purchasing the Optimera® Mesh Nebulizer. Please read the user manual carefully before use to ensure safe operation of the device. Keep the manual well after reading it for future reference.

This nebulizer is intended for the treatment of the upper and lower respiratory tracts. It is designed for the treatment of respiratory diseases and the prevention of respiratory infections. It is widely used in the treatment of conditions such as colds, bronchitis, asthma, and pneumonia. It can also be used for the treatment of occupational diseases, caring for the throats of performers, and healing ENT (ear, nose, and throat) conditions such as chronic allergic rhinitis, acute laryngitis, acute pharyngitis, acute tonsillitis, and more.

Nebulizing and inhaling medication prescribed or recommended by a doctor can prevent respiratory diseases, treat the side effects of conditions, and accelerate healing. For more information about possible applications, consult a doctor. Medications should be taken as prescribed by the doctor. Inhalation should be done in a calm and quiet environment. Breathe in slowly and deeply to ensure the medication reaches the small bronchi deep in the lungs. Then, exhale normally.

1. Information about the product

The Optimera® Mesh Nebulizer is designed for the treatment of respiratory diseases and the prevention of respiratory infections. The medication liquid is pressed through the micro-mesh by rapid vibrations in the device, forming numerous micro-atomized particles. The mist can then be inhaled using the face mask or mouthpiece.

1.1 Description about the product and accessories

The nebulizer consists of the device and a medication container with a mesh atomizer, which facilitates the nebulization process. Accessories include aerosol masks, one for adults and one for children. There is also a mouth piece.

1. Device

2. Medicine container

3. Mouth piece

4. Child mask

5. Adult mask

1.2 Product components

The device complies with the requirements of IEC 60601-1-11 and can be used in a home environment. Children should be supervised by an adult. The device should not be used with anesthesia breathing systems and respiratory systems. Pentamidine is not applicable to the system.

2. Operation of the device

The device is powered by a specific circuit frequency that causes the piezoelectric ceramic to vibrate in harmony, making the metal mesh plate vibrate rapidly. The liquid medication is quickly pushed through the micro holes of the metal mesh plate, producing numerous micro-atomized particles.

These particles are inhaled for treatment using masks or mouthpieces and delivered to the patient's respiratory system. To prevent cross-contamination, we recommend that the nebulizer be used by only one person.

3. First time use

The nebulizer and accessories must be cleaned and disinfected before you use the device for the first time. For cleaning information, see "Cleaning and Disinfection." If you need to take multiple medications during your therapy, it is important to rinse the medication container after each use.

1. Remove the device from the packaging
2. Clean the nebulizer and its accessories according to instructions.

Batteries

1. To open the battery compartment cover, gently press and pull the cover in the direction of the arrow. Then open the cover.
2. Insert alkaline or rechargeable batteries (type AA). Do not mix different brands of batteries.

5. Ensure that the batteries are inserted correctly.
6. Close the cover by sliding it back in the opposite direction of the arrow..

Let op

When replacing the batteries, ensure that the medication container is completely empty to avoid the risk of leakage. If you notice that the nebulization becomes less effective, it is advisable to replace the batteries. If you will not be using the device for an extended period, remove the batteries from the device.

Using the device with a USB-C adapter

1. Insert the USB cable into the port.
2. Plug the adapter into the socket.

Let op

Remove the batteries when using the device via the USB-C adapter. You need to purchase a Mobile Charger yourself. To ensure safe use, use mobile chargers that comply with the electrical safety standard GB 9706.1-2007 or IEC 60601-1:

Using the Optimera® Mesh Nebulizer

Use the nebulizer as advised by your doctor, discussing how frequently to use it and the amount of medication or isotonic saline solution to use.

1. Twist open the medication container (2) clockwise using the container's screw cap.
2. Fill the container with isotonic saline solution or medication. Do not overfill to prevent leakage. The minimum capacity is 5 ml and the maximum capacity is 18 ml.
3. If the recommended dose is less than 5 ml, add isotonic saline solution until it reaches at least 5 ml. Thick medications may require dilution. Always follow doctor's instructions.
4. After filling, screw the cap back on counterclockwise.
5. Attach the mouthpiece or mask to the nebulizer.
6. Start the treatment using the power button. The power indicator light (green) will illuminate.

Ensure that the medication container (2) and the device (1) are securely connected. Incorrect assembly can cause short circuits and prevent the nebulizer from functioning.

Breathing Techniques

Proper breathing technique is important to ensure that the medication particles are widely distributed throughout the airways. Use the nebulizer only to treat respiratory conditions as advised by your doctor. Some medications are available only with a doctor's prescription.

1. Sit in a quiet environment. Hold the Optimera® Mesh Nebulizer in your hand. Keep the device as upright as possible and do not tilt it more than 45 degrees in any direction.
2. For throat treatment, use the mouthpiece. For treatment of the nose or throat cavities, use the mask.
3. When using the mouthpiece, keep your lips slightly closed. When using the mask, ensure it covers your nose and mouth. Breathe calmly and deeply through your nose and exhale through your mouth.
4. When using the mouthpiece, inhale through your mouth and exhale through your nose.
5. During a treatment session, you can use either the mask or the mouthpiece.
6. Additionally, clean the nebulizer after each use.

7. It is possible to use the nebulizer in different modes. Press the power button to switch modes.

Essential oils, cough medications, gargling solutions, and skin application drops or for use in steam baths are entirely unsuitable for inhalation using the nebulizer.

Use the mouthpiece correctly. Do not swallow liquid and ensure that liquid does not go directly to affected areas. Improper use or irregular breathing can cause respiratory tract obstruction.

If the vibrating mesh, located in the medication container (2), is not submerged in the medication, the device will not be able to nebulize. Tilt the device slightly towards you and ensure there is at least 5 ml of liquid in it.

The nebulizer automatically turns off after approximately 10 minutes.

After treatment

1. After treatment, turn off the device using the power button.
2. The power indicator light will turn off.

3. If using a mobile charger, unplug it from the socket.
4. Remove the mask or mouthpiece.
5. Hold the device and unscrew the cap, then remove the medication from the medication container.
6. Clean the device and allow loose parts to dry thoroughly. Store it in a clean environment for future use.

Let op

When the liquid is depleted, the device automatically turns off. Due to technical reasons, a small amount always remains in the device. If the device is not functioning, check if the batteries need replacement. Alternatively, connect the nebulizer using the USB-C adapter to verify operation.

4. Cleaning and disinfecting

Follow these hygiene instructions carefully to prevent health risks. The nebulizer and accessories are intended for repeated use.

Please note that cleaning and hygienic requirements may vary depending on different application areas.

1. Make sure to turn off the device.

2. Remove the batteries or adapter before cleaning the device.

Cleaning

Disinfect the mesh after each day's final treatment.

1. Remove the mouthpiece or mask from the nebulizer and clean it.
2. After use, remove any remaining medication.
3. Add 6 ml of clean water (30-60 degrees Celsius) to the medication container (2).
4. Turn on the device to remove all residues from the mesh for 1 to 2 minutes.
5. If there are any traces of medical solutions or impurities remaining on the gold-colored contact points, use medical cotton swabs to remove them.
6. Wipe the cap of the container with distilled water and gently wipe the device.

Never wipe the mesh with sharp objects or touch it directly. Also, do not use paper tissues, as this may interfere with the normal functioning of the mesh.

Disinfecting

1. Add 2 ml of hot water (not exceeding 80 degrees Celsius) to the medication container OR disinfect with 1 part vinegar to 3 parts distilled water.
2. Nebulize until the water in the container is depleted, then turn off the device.

Cleaning and disinfecting the mouthpiece or masks

1. Remove the mouthpiece or mask.
2. Clean the mouthpiece and mask with pure water for 2 to 3 minutes (water temperature $\leq 50^{\circ}\text{C}$).
3. Dry the moist parts with a clean cloth.
4. Wipe the mask and mouthpiece with 75% medical alcohol for at least 3 minutes to disinfect, and store it in a dry environment.

Drying

1. Gently dry all parts with a soft and clean cloth.
2. Tilt the nebulizer gently forwards and backwards (5 to 10 times) to remove all water.
3. Place the parts on a dry, clean, and absorbent surface, and allow them to air dry completely for at least 4 hours.
4. Once all parts are fully dry, reassemble everything and store the device in a dry place. Ensure the container is completely dry by shaking it.

Storage

1. Do not store the device in a humid environment, such as the bathroom. Also, do not transport it together with damp objects.
2. Store the device away from direct sunlight.

Let op

The device is recommended for up to 4000 uses within 3 years. The nebulizer should be stored in a temperature range between 20 and 55 degrees Celsius. Avoid storing it directly in sunlight or near open flames or heat sources.

5. Technical specifications

Name: Mesh Nebulizer

Model: JSL-W303

Dimensions: 43 mm (L) x 58 mm (W) x 121 mm (H)

Power Source: DC 3V±10 (2 "AA" Alkaline Battery) or USB-C adapter (Input: 100240V, 50/60 Hz, Output: 3V-5V AC, 12A)

Noise: < 50dB

Power Consumption: 1.5W

Vibration Frequency: 120 KHz

Condensation Rate: > 0.2 ml/min

Nebulized Particle Size: 5µm±25%

Medication Capacity: Minimum 5 ml – 18 ml

Battery Life: Can be used for approximately 4 days with 20 minutes of treatment time each day using 2 AA batteries.

Operating Conditions: Temperature 10°C-40°C

Air Pressure: (66.0--106.0) kPa

Storage Temperature: 20°C—55°C. Avoid humid environments.

6. Cleaning and disinfecting

If the nebulizer is not functioning properly, first check if you can resolve it with the solutions below. If the solution to your problem is not listed here, please contact wmtecbv@gmail.com before returning the product to us. We may be able to provide technical support or send you replacement parts first.

Problem	Possible cause
The nebulizer produces little or no aerosol.	1. Too little medication in the vein. 2. The device is not held upright. 3. The medicine is not suitable for nebulization. 4. The metal parts in the medicine container (2) are dirty. 5. The batteries are not fully charged.
The output is too low	1. The batteries are empty. Replace the batteries or connect the adapter and try again. 2. There are air bubbles in the medication container that prevent the medication from reaching the gauze. Check this and remove the air bubbles. 3. Particles on the mesh hinder the output. To remove this, fill the nebulizer with 2 to 3 drops of vinegar and 3 to 6 ml of water and nebulize this substance completely. Do not inhale it and be sure to clean and disinfect the device. 4. The mesh is worn.
Which medications are suitable for inhalation?	Only a doctor can advise which medication is best to use to treat your complaints. With a nebulizer you can nebulize medication with a viscosity of less than 3.
Some of the inhalation fluid remains	This is normal and happens for technical reasons. Stop inhaling when you hear a

inside the nebulizer	different sound or the device turns off automatically
What steps should be taken when used for children or babies?	For babies and children, the mask must cover the mouth and nose to ensure effective inhalation. Only use the device with adult support and consult a doctor.
Inhaling with the mask takes longer	That's right. Less medication is inhaled through the mask per breath.
Should each user use their own accessories?	This is recommended to prevent cross-contamination. In short, for hygienic reasons.
The device turns itself off.	The device has an automatic delay function. To prevent damage to the mesh, the device switches itself off when the medicine or liquid runs out. Or automatically after ~10 min.

7. Warranty

The device (1) has a 12-month warranty. The medical container (2) (including mesh plate) has a 3-month warranty. During the warranty period, we depend on the damage to decide on maintenance or replacement of damaged parts. Accessories such as the mouthpiece and mask are consumables and are not covered by the warranty.

The following is not covered by the warranty:

- The damage caused by human actions such as falling, improper touching or by immersing the entire device in liquid substances.
- The damage caused by failure to comply with the correct operating instructions.
- Accidentally damaged.
- Disassemble, alter or repair the product without permission from our company.
- The product serial number is torn or cannot be properly recognized or there is no invoice showing the purchase date.

Please keep the packaging securely so that it can be used when you ship the product to our company under warranty use. To use the warranty, please first contact us by email at wmtecbv@gmail.com. We are not liable for damage due to incorrect use of the product.

8. Safety Information & Warnings

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1. Before use, ensure that there is no visible damage to the device or accessories. If you are in doubt, do not use the device and contact our company.
2. Use of this device is not a substitute for a consultation with your doctor. If you have pain, are experiencing health problems or are ill, always consult a doctor first.
3. Only use medication prescribed by a doctor.
4. If the device does not function normally or you start to feel unwell, stop using it immediately.
5. Keep the device away from your eyes during use as some medications may cause damage to your eyes.
6. Do not use this device near flammable gas, open flames or electromagnetic transmitters.
7. The appliance is not suitable for children or persons with physical or mental disabilities. Or persons who have little experience or knowledge of the safe use of this appliance, unless they are supervised by a person who will take responsibility for its safe use.
8. Do not immerse the device in water or other liquids.
9. Do not use the device in areas with high humidity, such as bathrooms.
10. Make sure that no liquids enter the device.

11. Remove the batteries from the device if you do not use it for a long time. Leaking batteries can cause damage.

1. Store the device properly in a space that protects the product from external influences.
2. Do not use liquids with a viscosity higher than 3, as they may irreparably damage the mesh.
3. Do not use medication in the form of powder (even dissolved).
4. Do not shake the atomizer, this may cause liquid to leak and damage the device.
5. Remove the charging cable and adapter after charging.
6. Turn off the device immediately if it is not working properly.

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