


(Diapers)

The anti-bacterial adult nappy is designed to be worn by men or women and features super-absorbent fibre for even absorption. The breathable, skin friendly adult diaper will keep skin dry and comfortable while the secure, re-fastenable tape and elastic leg gathering provide a comfortable fit. With other features including wetness indicator, easy tape panel, leak guard and standing leg cuff, plus a soft and dry non-woven surface and odour control, the adult diaper provides full protection for adult incontinence. Available in two sizes.

Size.....	Medium (820x635 mm)	Net Weight (kg)	1.10
Absorption (ml).....	1900	Gross Weight (kg)	1.13
No. in Pack.....	10		

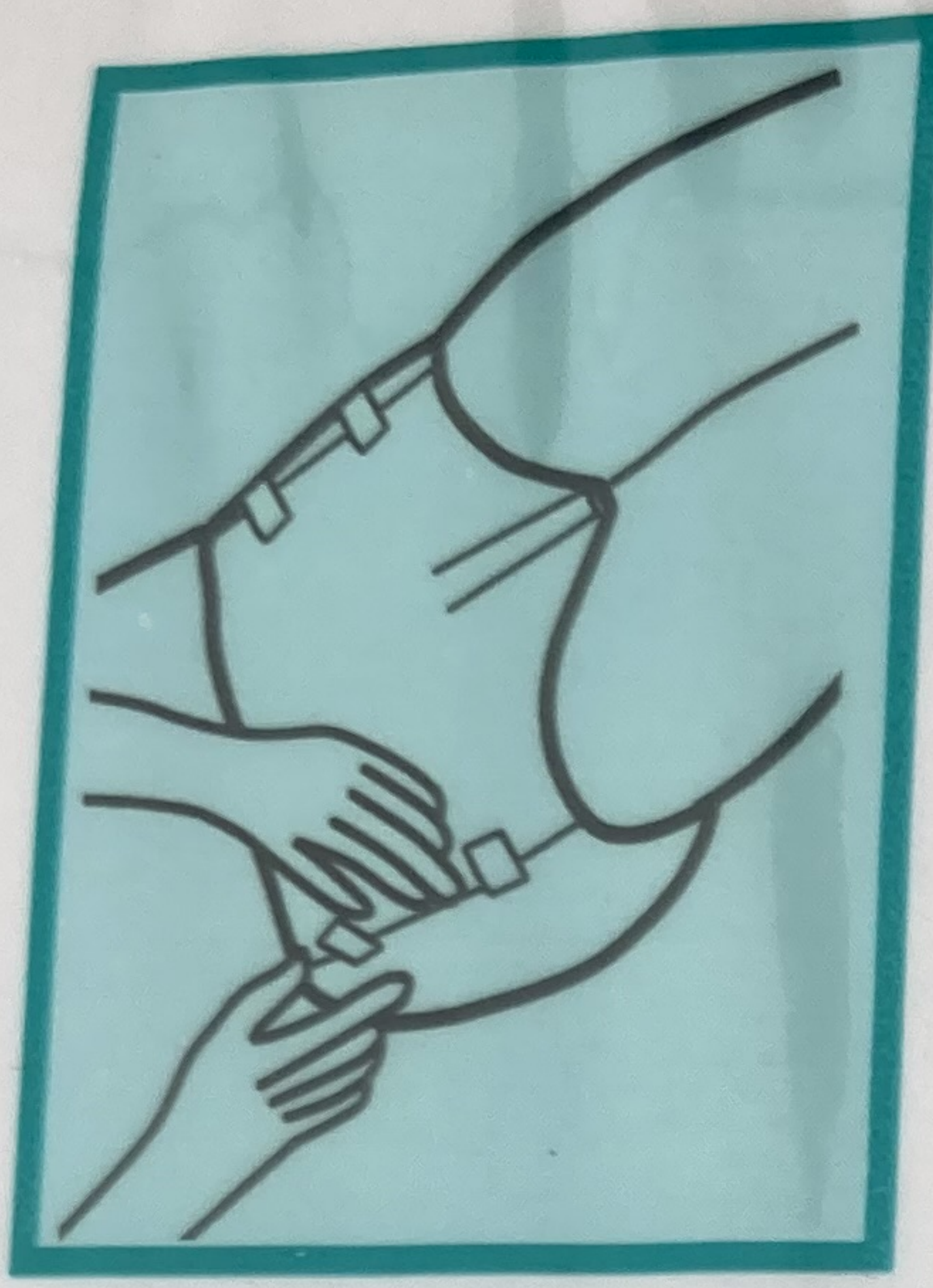

Fitting Instructions

1. Lay the user down on his/her side. Open the diaper and place the bed facing upward.

2. Roll the user onto his/her back and center the user on the open diaper.

3. Pull out the front of the diaper and fasten onto the back of the diaper.


Caution: Avoid direct contact of the fastening tapes with the skin. Change diaper immediately upon soiling. If irritation or discomfort occurs, discontinue use and consult your physician. Keep plastic bag away from children. This product will burn if exposed to flames.

Midapt®

Independent Living Solutions



VM846

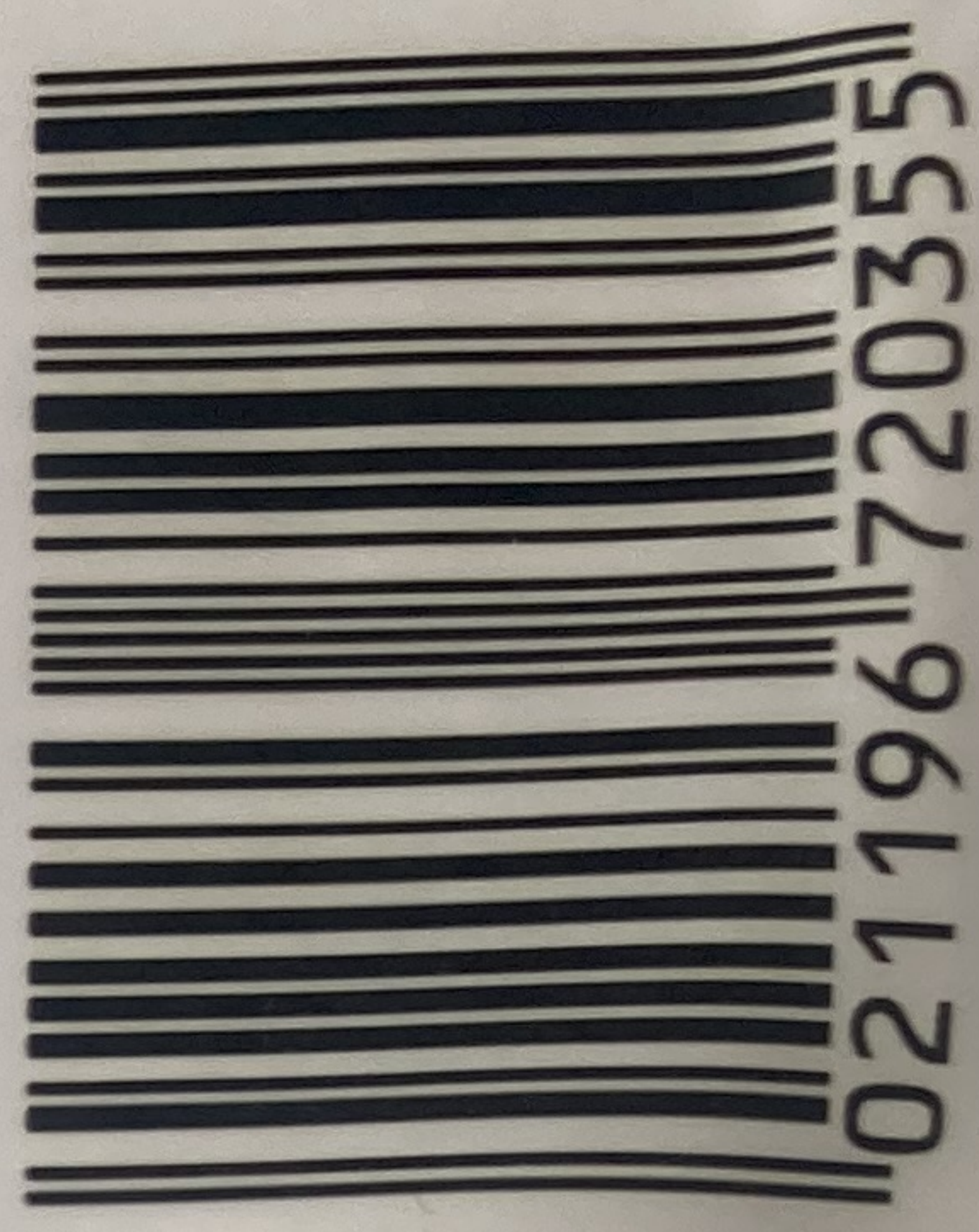

5 021196 720355