

BEE&YOU RHINAPI NASAL SPRAY

BEE&YOU RHINAPI Nasal Spray is a medical device. BEE&YOU RHINAPI Nasal Spray is a water-based, salt solution that aims to empty, clean, and moisturize the nasal passages without irritating the damaged skin. It is non-sterile and mechanically effective. It is used for the maintenance of nasal dryness, nasal congestion, and nasal physiology. It supports the nasal treatment of conditions such as sinusitis, colds, and flu. It is used to remove allergens and irritants that cause allergic rhinitis and asthma from the nasal cavity. It ensures the continuation of physiological moisturization in the nose.


Usage Instructions: To ensure the proper and effective use of BEE&YOU RHINAPI, please follow these usage instructions carefully:

Positioning: Hold the spray nozzle upright, directing it towards the inside surface of the nose. This allows for precise application and targeting of the nasal passages.

Activation: Gently press on both sides of the spray head with your fingers. This action activates the spray mechanism, releasing a controlled dose of the solution.

Dosage: Apply three doses of BEE&YOU RHINAPI into each nostril daily. This recommended dosage helps to ensure optimal nasal hygiene and relief from nasal conditions.

Suitable for Adults: BEE&YOU RHINAPI is specifically formulated to be used by adults.

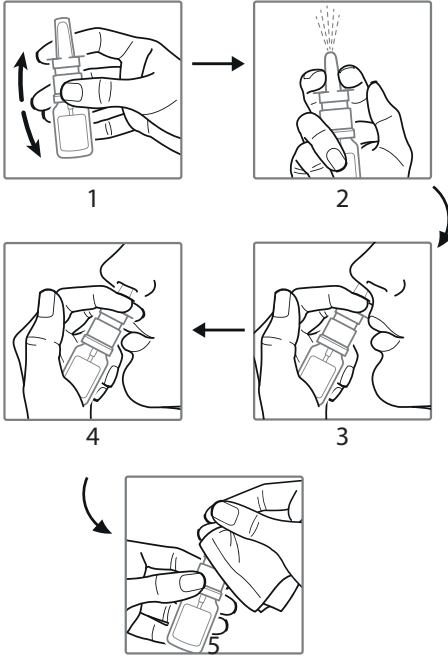

Bileşenler: The product contains deionized water, sodium chloride, glycerin, eucalyptus oil, and propolis extract.

Endikasyon: It is used for the treatment of excessive nasal discharge and nasal congestion associated with rhinitis.

Mode of Action: BEE&YOU RHINAPI Nasal Spray is a product formulated with deionized water, glycerin, eucalyptus oil, propolis extract, and sodium chloride to provide support in the treatment of symptoms associated with rhinitis, rhinosinusitis, and nasal congestion that occur through various pathways. BEE&YOU RHINAPI Nasal Spray effectively helps to improve respiration within a very short timeframe by cleansing the nasal passages.

The spray apparatus comprises a nozzle, pump, and suction hose. To initiate pumping, pressure is applied to the nozzle. This pressure, in conjunction with the action of the pump, draws the solution from the glass bottle through the hose and sprays it into the nasal passages.

In addition to its mechanical cleaning feature, it also has a mechanical moisturizing effect on the skin. It ensures the continuation of physiological moisturization in the nose.

Application

Cautionary Note: Do not purchase products with damaged packaging. Avoid contact with the eyes; in case of contact, rinse thoroughly with plenty of water. Individuals who have an allergic reaction to bee products should refrain from using this device. Check the composition list, and do not use it if you are allergic or sensitive to any ingredient in the product.

Precautions: Before using, carefully read the user manual. The product is intended to be used as a nasal spray only. Pregnant or nursing mothers can use it with a doctor's recommendation.

Storage Conditions: Store the product with the cap closed at room temperature. Protect it from higher temperatures and sunlight.

MAIN OFFICE

Osmangazi Mh. Ziya Sokak No:4 Sancaktepe, İstanbul

CONTACT

Phone: 0216 499 48 00

Fax: 0216 499 47 00



BEE&YOU
FROM THE FASCINATING ANATOLIA



www.rhinapi.com