Telano

# PREGNANCY TEST MIDSTREAM

FDA CE 0197

DE

2

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**FACTORY: W.H.P.M. BIORESEARCH & TECHNOLOGY CO., LTD.**

No.2, Zhongxin Street, LouZizhuang, Jinzhanxiang,  
Chaoyang District, Beijing, 100018 - P.R. China

**MEDICAL DEVICE SAFETY SERVICE GMBH**

Schiffgraben 41, 30175 Hannover, Germany

**TELANO B.V.**

De Bosschen 10, 5151 RT Drunen, Netherlands

## INSTRUCTIONS FOR PREGNANCY TEST MIDSTREAM (10mIU/ml)

Telano's pregnancy test is easy to take at home. The test detects the human chorionic gonadotropin (hCG) in your urine. Your body starts producing the hCG hormone as soon as you are pregnant. In early pregnancy, the pregnancy hormone increases rapidly in the first few days. The hormone doubles every 2 to 3 days and reaches its peak between 7 to 12 weeks of pregnancy (100,000mIU/ml to 288,000mIU/ml). The Telano pregnancy test is very sensitive, allowing you to test for the first time as early as 6 days before your expected period. The Telano pregnancy test gives a positive result at hCG levels from 10mIU/ml. If you test on the day of your missed menstruation, you can do the test at any time of the day. If you test earlier, we recommend using morning urine because the concentration of hCG will be higher. Avoid drinking too much before testing.

The results of this test are very reliable (>99%) and easy to read from the control window. Please read this manual carefully before use.

### Test procedure

1. Remove the test from the packaging.
2. Remove the plastic cap from the midstream pregnancy test so the absorbent strip becomes visible.
3. Hold the midstream pregnancy test at the thumb grip. Urine for at least 10 seconds over the absorbent strip with the tip down. Caution: Do not urinate over the display screen. If you prefer, you can also collect some urine in a clean container. Here you can hold the absorbent strip for 10 seconds to make sure the strip contains enough urine.
4. Immediately place the cap over the absorbent strip as soon as you have finished collecting the urine.
5. Place the test on a flat surface with the test window facing up.
6. You will now see that 1 or 2 lines appear in the test window. Read the test result within 5 minutes.

**Please note:** do not wait longer than 5 minutes to review the test results. Otherwise the result may be invalid.

### Result

**Positive (pregnant):** One pink line appears in the test window (T) and one in the control window (C).

**Note:** The colour intensity of the (C) and (T) line may vary. A light coloured (T) line is also positive. The colour intensity of the (T) line may vary because the concentration of the hCG hormone may vary at different stages of pregnancy.

**Negative (not pregnant):** One pink line appears in the control window (C). No line appears in the test window (T).

**Invalid:** No line appears in the control window (C). Repeat the test with a new test. Not enough urine may have been absorbed. If the problem persists, contact Telano.

### Package contents

1. Telano midstream pregnancy test
2. Desiccant
3. Package leaflet

### Supplies

1. Clock or timer
2. Clean container (if you want to collect the urine first)

EN

### Warnings and precautions

1. Read this leaflet carefully before you start testing.
2. For in vitro diagnostic use only
3. For external use only, do not swallow
4. Do not use this test after the expiry date.
5. Do not open the foil bag until you are ready to test. Do not use the test if you see that the foil bag is damaged or has already been opened.
6. This test is not suitable for reuse. After use, the test thrown in the trash.
7. Do not touch the absorbent strip of the test. Hold the test by the thumb grip.
8. Keep this test out of the reach of children.

### Storage

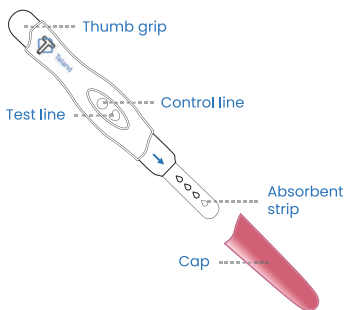
The Telano pregnancy test should be stored in the unopened foil bag at a room temperature between 4 and 30°C and away from heat or moisture. Do not freeze the test.

### Restrictions

1. Avoid drinking a lot before performing the test.
2. This pregnancy test allows you to test for the first time 6 days before your expected period. However, in a very early pregnancy the Hcg hormone is present in the urine in a very low concentration. This can cause in a negative result. If you suspect you are pregnant, repeat the test after 2-3 days.
3. If you have a negative result, but suspect that you are pregnant anyway, contact a doctor.
4. Fertility drugs containing hCG can give misleading results.
5. After childbirth, ectopic pregnancy or miscarriage, it can take about 8 weeks for the hCG hormone to completely disappear from your body. This allows you to test positive while you are not pregnant. In this case, consult a doctor for the test result.
6. Alcohol can have a distorting effect on the result of the test. Therefore, avoid drinking alcohol before performing the test.
7. An ectopic pregnancy, just like an ordinary pregnancy, results in a positive test.
8. An early miscarriage or missed abortion can lead to a slightly positive result and later a negative result.
9. The pregnancy test can give a negative result at a very high concentration of Hcg (>500,000mIU/ml). This is called the 'prozone effect'. The average values of hCG during pregnancy are 5 mIU/ml up to 288,000 mIU/ml up to week 12. After that the Hcg values decrease again to about 117,000 mIU/ml in week 40. The prozone effect is therefore very rare.  
If you do suspect the prozone effect, you can perform the test as usual. You should collect the urine in a cup and dilute it with as much water as there is urine in the cup. A line should now reappear when it comes to the prozone effect.

### Performance features

10mIU/ml = extra early pregnancy test

**Meaning symbols**

- |               |            |                                  |                      |              |                          |                                   |   |
|---------------|------------|----------------------------------|----------------------|--------------|--------------------------|-----------------------------------|---|
|               |            |                                  |                      |              |                          |                                   |   |
| tenable until | LOT number | this packaging contains one test | For invitro use only | Do not reuse | Store between 2 and 30°C | keep out of the reach of children | before use, read the instructions for use |

**FACTORY: W.H.P.M. BIORESEARCH & TECHNOLOGY CO., LTD.**

No.2, Zhongxin Street, LouZizhuang, Jinzhanxiang, Chaoyang District, Beijing, 100018 - P.R. China

**MEDICAL DEVICE SAFETY SERVICE GMBH**

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