# Hydrogel Pads

**Instructions for use**  
**Gebruiksaanweisung**  
**Mode d'emploi**  
**Istruzioni d'uso**  
**Gebruiksaanwijzing**  
**Instrucciones de uso**  
**Οδηγίες χρήσης**  
**הוראות שימוש**  
**Bruksanvisning**  
**Brugsanvisning**  
**Käyttöohjeet**  
**Bruksanvisning**  
**Instrukcja użycia**  
**Инструкция по применению**  
**Használati utasítás**  
**Návod k použití**  
**Kullanma talimatları**  
**使用说明**  
**사용설명서**  
**使用说明**  
**使用说明**

Medela AG  
 Lättichstrasse 4b  
 6341 Baar, Switzerland  
 Phone +41 41 562 51 51  
 www.medela.com

Transport: -20°C to +5°C  
 Storage: +5°C to +25°C

STERILE R CE 0123

© Medela AG 200.1726/2016 - 10/G

**FR**

**Mode d'emploi**

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**EN**

**Instructions for use**

Wash hands before using the product.

- Open pouch.
- Remove gel pad from the plastic liner.
- Gently place the gel side of the pad directly over your nipple.
- Do not use if the pouch is open or damaged.
- Prior to breastfeeding, remove the gel pad.
- Place on a clean surface.
- Gently wash the breast with warm water and pat dry.
- Then start breastfeeding.
- After breastfeeding, reapply the gel pad.

*Do not use soaps or detergents to clean pad • Store gel pads in a cool, dry place • For more cooling sensation, gel pads can be stored in the fridge • Reusable for 24 hours • If gel pad becomes cloudy or milky, please discard and use a new one.*

**Optional:** To improve soothing, briefly pass the gel side of the pad under a cool stream of water (1–2 seconds), shake off excess water and set aside for 2 minutes. Do not rinse more than twice a day.

**Important Safeguards**

Use only as indicated • Keep out of reach of children, especially the plastic liner • Extra care should be taken if using with cracked nipples or fungal infection • If nipples are cracked, bleeding or have a burning sensation, contact your lactation consultant or physician • To prevent nipple pain it is important that baby feeds at mother's breast in a correct position.

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