
En USER GUIDE

!!! Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of !!!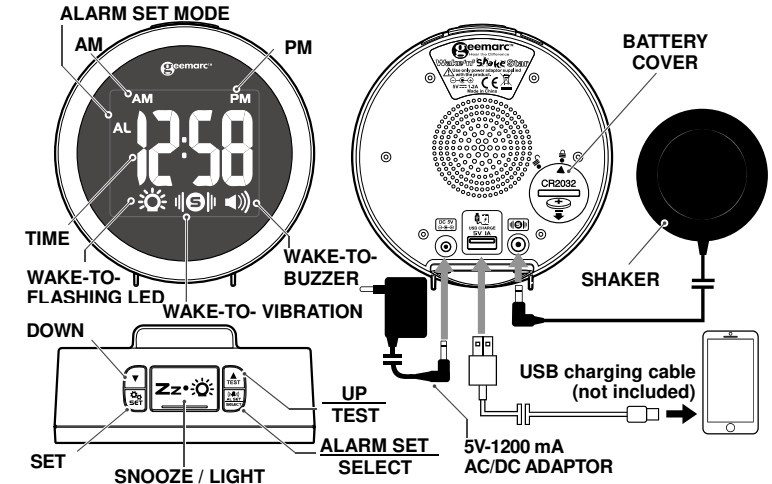

BACKUP BATTERY INSTALLATION

Your clock requires one CR2032 lithium battery (not included) to provide backup power to the clock in the event of a temporary power interruption. Place your unit face down on a flat surface.

- Use coins and turn counterclockwise to open the battery door located on the back of the unit.
- Insert one new CR2032 lithium battery into the battery compartment with the "+" side facing up as indicated.
- Place the battery door on the battery compartment, use coins and turn clockwise to lock the battery door.

It is not necessary to insert a back-up battery for the clock to function; however, fitting the backup battery will keep the time and alarm settings in the event of a power cut or the DC adaptor being unplugged.

If mains power is interrupted or the adaptor is unplugged, the LED clock display will go off and the alarm will not function.

Please replace the battery every year to ensure that your device is not damaged.

INSTALLATION

Plug the AC/DC adaptor to an AC household outlet, and then plug the DC jack to the back of the unit. Your clock is ready to work for you.

TO SET TIME AND THE SNOOZE DURATION

- Press **SET** once, the hour digits "00" flash. Press **▲** or **▼** to set the time in hours (hold to advance rapidly).
- Press **SET** again, the minute digits "00" flash. Press **▲** or **▼** to set the time in minutes. (hold to advance rapidly).
- Press **SET** again, the display shows "24Hr" and flashes. Press **▲** or **▼** to select "12Hr" or "24Hr" time format. If "12Hr" is selected, the **AM** will appear on the upper left to indicate the morning time and **PM** will appear on the upper right to indicate the afternoon.
- Press **SET** again, the display shows "05" and flashes. Press **▲** or **▼** to select your snooze time from 5 to 60 minutes.
- Press **SET** again or if no key pressed for approx. 10 seconds to exit the setting mode.

TO SET THE ALARM

- Press and hold **AL SET / SELECT**, **AL** appear and the alarm's hour digits flash. Press **▲** or **▼** to set the alarm in hours. (Hold **▲** or **▼** to advance rapidly).
- Press **AL SET / SELECT** once again, the minute digits flash. Press **▲** or **▼** to set the alarm in minutes. (Hold **▲** or **▼** to advance rapidly).
- Press **AL SET / SELECT** again, LED displays "HI" (vibration at HI level), press **▲** or **▼** to select "LO" (vibration at Lo level)
- Press **AL SET / SELECT** once again, LED displays "F-1 (ex. 800 Hz)" and flash, press **▲** or **▼** to select the alarm tone (frequency) :

- F-1 (low tone, 800 Hz), F-2 (mid tone, 1 KHz), F-3 (high tone, 1.5 KHz)
- Press **AL SET / SELECT** again, LED displays "L1" (for the Low ringer level), press **▲** or **▼** to select "L3" (for the high ringer level)
 - Press **AL SET / SELECT** once again or no button around 10 seconds to exit the alarm setting mode.

TO TURN ON/OFF AND SELECT THE ALARM MODE

- At normal time display mode, press **AL SET / SELECT** one at a time to activate the alarm(s) and turn on corresponding alarm indicator(s) on the lower part of the display:

	: only shaker activated
	: both sound and shaker alarm activated
	: both strobe light and shaker alarm activated
	: strobe light, sound and shaker alarm activated

- Press **AL SET/SELECT** once more to turn off all the alarms, all , and alarm indicators are turned off.

Note: To use the shaker alarm, please plug in the shaker plug to the back of the clock. If user doesn't want to use the shaker alarm, unplug the shaker plug from the clock.

TO STOP AND RESET THE ALARM TO COME ON THE NEXT DAY

When alarm is sounding, the corresponding alarm indicator flashes. Press **AL SET / SELECT** once to stop the alarm and reset it to come on the following day. After that, the corresponding alarm indicator stays on the display.

TO USE SNOOZE (default snooze duration is 5 minutes)

When alarm is sounding, presses **Zz •** once, the alarm will stop temporarily and the corresponding alarm indicator keep flashing. The alarm will come on again after the set snooze duration is over.

TO USE THE HI-MED-LO DIMMER (LED TIME DIGITS)

There is a 3-level brightness control for the LED clock display. Press **Zz •** to adjust the brightness (**HI / MED / LO**) of the LED display. This can only be done when the unit is not alarming. At the start, the brightness is setting on Hi position.

TO USE THE RING OF LED LIGHT

Press and hold **Zz •** button to turn on the ring of LED light, the brightness level of LED ring (Hi-Med-Lo) corresponds to the time display brightness (Hi-Med-Lo). When the LED ring lights up, press **Zz •** button one at a time to adjust time display brightness and LED ring brightness accordingly. Press and hold **Zz •** button to turn off the ring of LED light. (doesn't forget to readjust LCD dimmer after).

CHARGE YOUR MOBILE PHONE WITH USB PORT

Connect the charging cable of your mobile phone to the USB port at the back of the clock and then connect the other end to the micro USB charging socket on your mobile phone. The clock will charge the mobile phone automatically.

Note:

1. Make sure the cable supplied with your mobile phone is used for charging.
2. Make sure the adaptor supplied with this clock (5V 1.2A) is used to power it.
3. It is not possible to charge an iPad® or notepad from the USB socket on this clock.
4. No USB charging provided during the alarms are sounding / active.

TEST MODE

Press and hold **▲ / TEST** button, strobe light flashing, shaker vibrating and buzzer sounding.

TROUBLE SHOOTING

If your clock displays incorrect time or does not function correctly, which may be caused by electro-static discharge or other interference. Takes heavy current when electric timer on heater switches on/off (poor mains filtering). The mains adaptor for the clock should only be plugged into a mains wall socket-not a mains extension that is shared with electric heater (or other high wattage equipment).

CARE OF YOUR PRODUCT

1. Place your clock on a stable surface, away from sources of direct sunlight or excessive heat or moisture.
2. Protect your furniture when placing your unit on a natural wood and lacquered finish by using a cloth or protective material between the unit and the furniture.
3. Clean your unit with a soft cloth moistened only with mild soap and water. Stronger agents such as benzene, thinners or similar materials can damage the surface of the unit. Make sure the unit is unplugged before cleaning.
4. Remove the back-up battery if the clock is not going to be used for a month or longer to prevent possible corrosion. Should the battery compartment become corroded or dirty, clean the compartment thoroughly and replace the battery.

SPECIFICATIONS

Alarm duration	: 1 hour	
Snooze duration Adjustable	: 5 to 60 minutes	
Default settings	Time format	24 hour
	Time	0:00
	Alarm	6:00
	Alarm and shaker level	Hi
	Alarm sound frequency	F-1
Audible alarm	Alarm sound level	L1 (low)
	Snooze duration	05 (5 minutes)
Audible alarm	: up to 95 dB at 1m	

GUARANTEE

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

CE Directives: Geemarc Telecom SA hereby declares that this product is in compliance with the CE Directive requirements.

The CE declaration of conformity may be consulted at www.geemarc.com

Electrical connection: The apparatus does not incorporate an integral power on/off switch. To switch off the apparatus, disconnect the power adaptor.

When installing the apparatus, ensure that the mains power socket is readily accessible

Recycling Directives: The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way. When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip.
- Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at www.geemarc.com telephone 01707 387602 or fax 01707 832529

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