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Off key after the measurement. Otherwise the thermometer will automati- cally switch off after 10 minutes.
Memory
The last measured value will be stored automatically. This value is displayed when the On/Off button is pressed. The memory symbol is also shown beside the measured value, e.g. 37.5°C.M.
Changing the battery
The clinical thermometer contains a long-life alkali-manganese battery (LR 41, 1.55 V), which will last for approx. 2 years. The battery has to be replaced when "T" appears in the bottom right corner of the display. Press down the battery cover at the end of the device with your fingers and then pull it firmly back. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then put the battery cover back onto the case. Used batteries do not belong in the regular domestic refuse. Dispose of them through your local dealer for electric goods or your local depot for special waste. You are legally obliged to dispose of the batteries.
Note: Batteries containing pollutant substances are marked as follows: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. These batteries contain mercury (Hg) and lead (Pb).

Cleaning/Disinfection
The thermometer's flexible tip as well as its display are waterproof. This part of the thermometer can easily be submerged in water or disinfectant solution during cleaning or disinfection.

Under no circumstances should the thermometer be boiled! Use one of the following disinfectants in accordance with the manufacturer's instructions. Other cleaning agents or methods could impair the function of the thermometer or damage it.
Name of the agent: Isopropanol 70% Pharmacy; Gigasept FF; Lysoformin; Aseptisol.

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Off key after the measurement. Otherwise the thermometer will automati- cally switch off after 10 minutes.
Memory
The last measured value will be stored automatically. This value is displayed when the On/Off button is pressed. The memory symbol is also shown beside the measured value, e.g. 37.5°C.M.

Changing the battery

The clinical thermometer contains a long-life alkali-manganese battery (LR 41, 1.55 V), which will last for approx. 2 years. The battery has to be replaced when "T" appears in the bottom right corner of the display. Press down the battery cover at the end of the device with your fingers and then pull it firmly back. Remove the old battery and replace it with a battery of the same type. The + sign on the battery must point upwards. Then put the battery cover back onto the case. Used batteries do not belong in the regular domestic refuse. Dispose of them through your local dealer for electric goods or your local depot for special waste. You are legally obliged to dispose of the batteries.
Note: Batteries containing pollutant substances are marked as follows: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury. These batteries contain mercury (Hg) and lead (Pb).

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