



VM925E – Eye Drop Dispenser

Intended Use


Your Aidapt Eye Drop Dispenser has been designed to assist you when self-administering eye drops.

Installing Eye Drop Bottle

1. Open the body of the eye drop dispenser using the thumb guides.
2. Place bottle inside the dispenser so that the ridge fits firmly around the neck of the bottle.
3. Loosen the top of the eye drops but do not remove fully.
4. Close the dispenser by clicking its sides together securely.

IF YOU SUSPECT THE UNIT TO BE FAULTY, PLEASE DO NOT USE AND CONTACT YOUR SUPPLIER IMMEDIATELY.

USAGE INSTRUCTIONS

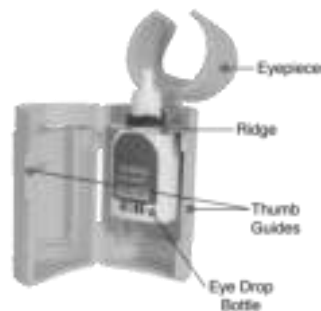

1. Place your dispenser in an upright position and open the eyepiece.
2. Remove the cap from eye drop bottle and close the eyepiece.
3. Tilt your head back and place eyepiece securely over the eye ensuring that the lip of the eyepiece is resting gently against the upper eye socket.
4. Use your finger to pull down the lower eyelid.
5. Gently squeeze the dispenser to administer the eye drops. One squeeze per drops.
6. When finished open the eyepiece and screw the bottle cap back on.



SERVICE WARRANTY

Aidapt guarantees the product free from defects in materials and workmanship for a period of twelve months.

- The product you buy may sometimes differ slightly from illustrations. This warranty is in addition to, and does not affect, your statutory rights.
- Our guarantee is administered by our retailers.
- If your product arrives damaged, you must contact the retailer from whom you bought it. The retailer's contact details will be on the invoice that arrived with the product, or on the e-mail you received when you placed the order. Do not contact Aidapt, only your retailer can arrange a replacement.
- If your product fails within the guarantee period, and needs to be repaired or replaced, you must contact the retailer from whom you bought it.
- The retailer's contact details will be on the invoice that arrived with the product, or on the e-mail you received when you placed the order.
- If you have just received your product and require technical help in using it, please call our Help Desk on (0845) 459 4816
- If you have any other technical queries about the product, please call our Help Desk on (0845) 459 4816.

Aidapt Bathrooms Ltd.,
Lancots Lane, Sutton,
St. Helens, Merseyside.
WA9 3EX
Website: www.aidapt.com