

**Let op!**

- Maak de Medicastle oor reiniger altijd schoon voor- en na gebruik
- Niet geschikt voor kinderen onder de 3 jaar
- Bij enige pijn of duizeligheid dient u meteen het gebruik van de oor reiniger te staken
- Reinig je oren niet met de Medicastle oor reiniger als je in het verleden last heb gehad van een geperforeerde trommelvlies of als er bloedingen, drainage of pijn optreden.

**Deze set bestaat uit: 1 oor spuit, 3 Zachte opzetstukken****Instructies voor gebruik:**

1. Draai de zachte opzetstuk in de opening aan het uiteinde van de oor spuit totdat de opzetstuk stevig vastzit.
2. Dompel de top in warm water (ongeveer lichaamstemperatuur) en trek aan de zuiger om de spuit te vullen
3. Kantel in zittende positie uw hoofd richting het te reinigen oor en met een kom onder het oor om overtollig water op te vangen. Steek de zachte opzetstuk voorzichtig helemaal in het oor en druk langzaam op de zuiger
4. Mogelijk moet u dit proces herhalen totdat het oor goed is schoongemaakt

**Schoonmaken:**

- Was de oor spuit en de opzetstukken na elk gebruik met warm zeepsop, spoel af en laat het luchtdrogen
- De oor spuit dient koel en droog bewaard te worden.

**Contacteer bij twijfel altijd uw huisarts!**

**CAUTION!**

- Clean the MediCastle Earwax remover before and after each use.
- Not for use on children under the age of 3 years.
- Never irrigate tubes, bleeding, drainage or when pain is in present.

**Including in the MediCastle Earwax kit:**

- 1 syringe.
- 3 soft tips.

**Use instructions:**

1. Screw the soft tip into the opening on the end of the syringe until tip is secure.
2. Dip tip into warm water( about body temperature) and pull the plunger to fill the syringe.
3. In a seated position tilt your head towards the ear that will be cleared and with a bowl under the ear to catch the excess water. Gently insert the soft tip all the way into the ear and slowly press down on plunger.
4. You may need to repeat the process until the ear has been cleared.

**Cleaning:**

- Wash the syringe with warm soapy water after each use, rinse and let it air dry.
- Syringe should be stored in a cool and dry place.

**When in doubt, always contact your doctor!**



