

## INSTRUCTIONS FOR Clavicle support MB.2300

### APPLICATION:

1. Place your arms through the arm straps.
2. Pull the arm straps to the desired position and attach the Velcro closures.
3. Remove the hook and cut off any excess strap. Re-adjust arm straps if needed to achieve better fit.
4. Make sure the straps are tight enough so that the shoulders are pulled back.

### WARNING:

This product should be given by an Orthopedic Doctor and applied by a professional. Application should be done with great care to ensure correct fit and efficiency. For hygiene reasons, it is proposed to be used by a single patient.

In exceptional cases of very sensitive patients, irritation may be caused. In this case, please contact your Doctor.

Recommended product use up to 2 years.

Please get your doctor to check your back and shoulders before and after the use of this product.

### WASHING INSTRUCTIONS:



1. Check the condition of the product before use. If it is worn off, it needs replacement.
2. Hand wash with cold water. Use a soft detergent. Do not use bleach.
3. Allow to dry on its own. Do not wash it in a washing machine or dry it in a tumble dryer.



### **MEDICAL BRACE**

22 Tegeas st.

Peristeri

PC: 12133

Phone: +30 210 5984020

FAX: +30 210 5984043

Email: [mb@medicalbrace.gr](mailto:mb@medicalbrace.gr)

Web: [www.medicalbrace.gr](http://www.medicalbrace.gr)