EN
<span></span>
<b>MILEX</b> <sup>®</sup> <small>a CooperSurgical Brand</small>
<b>SHAATZ Folding Pessary</b>
Support for Stage I and Stage II prolapse complicated by a mild cystocele
<b>R<sub>x</sub>Only</b>

<b>AVAILABLE MODELS</b>
<b>[REF]</b> <b>MXPSH</b> (Pessary Only)
<b>MXKPSH</b> (Pessary, TRIMO-SAN™, Jel Jector)

<b>Single-Patient Use</b>
<b>CE0086</b>
<b>Wash pessary with mild soap and thoroughly rinse prior to initial use.</b>

#### DESCRIPTION

The Milex® SHAATZ Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

#### WARNINGS

- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary.
- TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.

#### INDICATIONS FOR USE

For effective support in Stage I and Stage II prolapse, complicated by a mild cystocele.

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<b>CooperSurgical</b>

#### CONTRAINDICATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient
- Sexually active patient

#### RECOMMENDED CLEANING INSTRUCTIONS\*

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer’s recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak and Scrub
  - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
  - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
  - Following scrub, inspect device for visible soil residue.
- Rinse
  - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
  - Allow the device to dry.

\*New pessaries are powdered with food-grade powder.

#### PATIENT INSTRUCTIONS FOR PESSARY USE

#### CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

- To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner.
- It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

#### REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge
- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out

**For medical emergencies and for all medically related advice, consult your healthcare professional.**

#### FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

#### INSTRUCTIONS

- Wear dry gloves. When necessary, the entering end of the pessary can be coated with TRIMO-SAN.
- Use one finger to depress the perineum (see Figure 1).
- Compress the pessary (see Figure 1). Hold almost parallel to the introitus. Guide the pessary.
- Once the large flat disc is inside the vagina, push the pessary upward until the cervix rests behind the flat disc (see Figure 2).
- After fitting, the patient is told to sit, stand, and bear down slightly. Examination is made while the patient is in the standing position. The patient should not feel the pessary once it is in position.

**Note:** If the patient is unable to urinate with the pessary in position, remove it and fit her with the next smaller size. Repeat as necessary.

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- If the patient can void without difficulty, and the pessary remains in position upon re-examination, and the patient is comfortable with the pessary in place, this is a good indication that the correct size may have been selected. Patient experience may vary.
- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.
- The healthcare professional should be able to sweep one finger between the pessary and the vaginal walls. If there is not enough space to do this, the next smaller size should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.
- It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged.
- Ulcerations and erosions frequently occur in cases of complete prolapse due to irritation of the exteriorized cervix or vaginal wall. Whenever possible, reducing the mass and treating the irritation are primary steps before using a pessary. Patient experience may vary. Prolapse reduction may resolve cervical vaginal irritation. Verification of cervical cytology (pap) and or biopsy is clinically indicated.
- During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning douching, discharge, disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary.
- At the physician’s discretion, the patient can be instructed in the proper removal, cleaning and reinsertion techniques for her own pessary. This process can be performed nightly or even weekly by the patient under ideal circumstances.

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<b>Figura 2</b>

#### TO REMOVE

- Use one finger to depress the perineum.
- Insert one finger of other hand into the center hold to dislodge the pessary.
- Turn the pessary, bringing it almost parallel to the introitus.
- Using “corkscrew” motion, gently ease the pessary out.
- Wash the pessary with mild soap and water and rinse thoroughly.

#### RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

**Note:** The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

<b>CooperSurgical</b>
<b>95 Corporate Drive</b> Trumbull, CT 06611 USA Phone: (800) 243-2974 Fax: (800) 262-0105 <b>International</b> Phone: +1 (203) 601-9818 Fax: +1 (203) 601-4747 www.coopersurgical.com
<b>CE0086</b>
<b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP The Hague The Netherlands
<b>38654• Rev. A • 2/17</b>

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<b>EXPLANATION OF SYMBOLS</b>	
<b>[REF]</b>	Reorder number
<b>[LOT]</b>	Batch code
	Use-by date
	Consult instructions for use
	Not made with natural rubber latex
<b>R<sub>x</sub>Only</b>	<b>CAUTION:</b> U.S. Federal law restricts this device to sale by or on the order of a physician
<b>[EC REP]</b>	Authorized Representative in the European Community.
	Manufacturer

Dawn® is a registered trademark of Procter & Gamble, Cincinnati, Ohio 45202
TRIMO-SAN™ is a trademark of CooperSurgical, Inc.
Milex® and CooperSurgical are registered trademarks of CooperSurgical, Inc.
© 2017 CooperSurgical, Inc.

<b>CooperSurgical</b>
<b>95 Corporate Drive</b> Trumbull, CT 06611 USA Phone: (800) 243-2974 Fax: (800) 262-0105 <b>International</b> Phone: +1 (203) 601-9818 Fax: +1 (203) 601-4747 www.coopersurgical.com
<b>CE0086</b>
<b>EMERGO EUROPE</b> Prinsessegracht 20 2514 AP The Hague The Netherlands
<b>38654• Rev. A • 2/17</b>

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TRIMO-SAN™ es una marca comercial de CooperSurgical, Inc.
Milex® y CooperSurgical son marcas comerciales registradas de CooperSurgical, Inc.
TRIMO-SAN™ is een handelsmerk van CooperSurgical, Inc.
Milex® en CooperSurgical zijn geregistreerde handelsmerken van CooperSurgical, Inc.