


Nasal aspirator

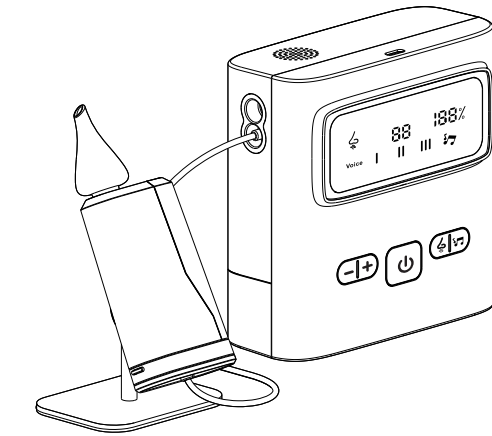

User Manual

IMPORTANT INSTRUCTIONS BEFORE USE

This device is designed to remove nasal secretions and nasal mucus and is suitable for newborns to the elderly. Before using this device please read this user manual carefully.


- 1, This device must be operated by an adult, do not allow children to use this device alone.
- 2, When not in use, keep the product out of the reach of children.
- 3, The device is only suitable for simple nose cleaning. If you or your baby's nose feels uncomfortable, it is recommended to contact a medical professional for treatment.
- 4, During the use of this device to help baby/child/adult clear nasal secretions, please ensure that the baby/child/adult keeps breathing normally. Do not make them hold their breath, shut up or swallow; to avoid pressure imbalances in the tympanic membrane caused by the vacuum pressure generated by the device.
- 5, Always clean the device after each use.

CAUTIONS:

Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.

- 1, Do not expose the device to extreme temperatures, high humidity or direct sunlight.
- 2, Do not place or store this device where it can fall, or be pulled into a tub or sink.
- 3, Never drop or insert any object into any opening or hose
- 4, Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and similar materials.
- 5, Never use while drowsy or sleeping.
- 6, Do not use the nasal aspirator when: Nasal bleeding, acute head, facial or neck injury, coagulopathy or bleeding disorder.
- 7, Only insert the tip enough to create a gentle seal around the edge of the nostril.
- 8, If irritation occurs discontinue use immediately
- 9, Do not use if it is overheating, operating irregularly, or is showing signs of damage to the housing.


PACKING LIST

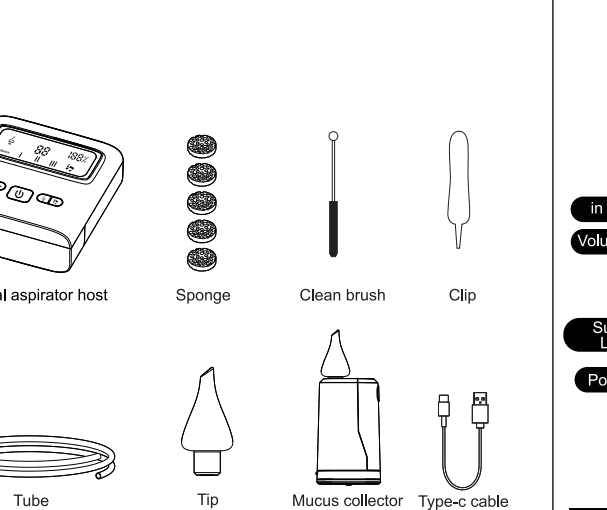

Accessories include:


Nasal aspirator host	*1	Tip	*1
Tube	*1	Clip	*1
Clean brush	*1	Type-c cable	*1
Mucus collector	*1	Sponge	*5

Product Overview

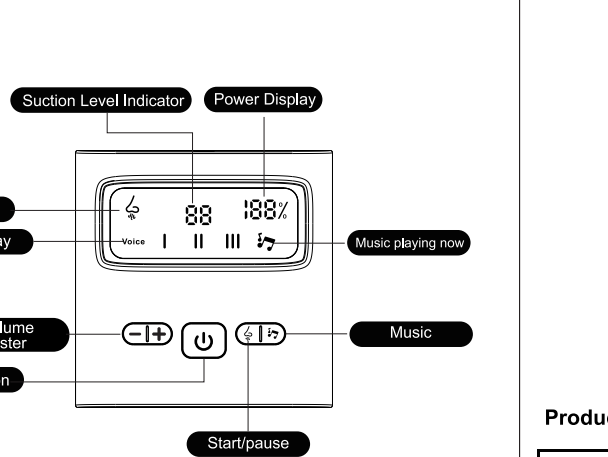

ON/OFF:	Long press for 3s on the Power Button to on/off the nasal aspirator
Start/Pause:	Press Start/Pause once to start or pause suction
Suction/Volume Level Adjuster	Press "+" suction/Volume level up; press "-" suction/Volume level down;
Power Display:	Real-time power display
Music	Press the music button to play music
Night Light:	Press the Night Light button to switch on the night light.

Mucus collector

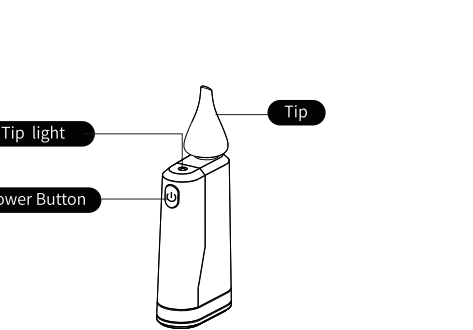


Product Specification:

Vacuum Suction Pressure:	225-412mmHg 65±5kpa (adjustable in 9 levels)
Noise Level:	45-55db
Conditions:	5-35°C
Battery:	1500mAh
Battery Life:	2.5hours, when fully charged
Charging Voltage:	5V
Input Current:	1A
Power Source:	USB-C

Operating Instructions


First Assembly After Arrival

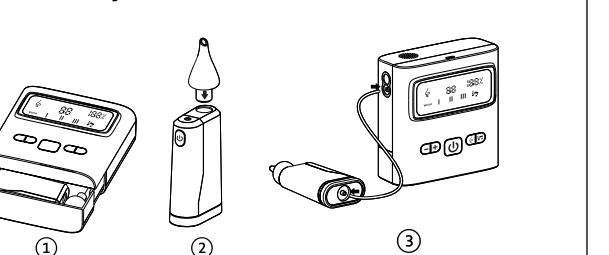

- ① Take out the snot container/tubing/appropriate tip from the host's storage drawer;
- ② Attach the silicone tip with the snot container;
- ③ Connect tubing to host and snot container, now you can power on and start suction.


How To Remove And Wash

※ The sealing plate is not washable.

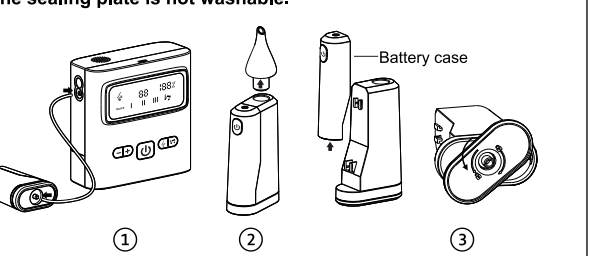

Special Notes on Sponges


Generally, there is no need to clean or replace the sponge; if the sponge gets into any mucus, you'll need to follow these steps to remove and replace the sponge. *This sponge is not a disposable product that can be washed, dried, and used again.

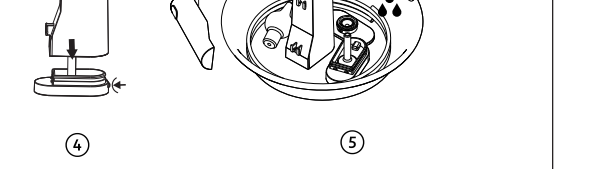

- ① Remove/Place the filter sponge on the transitional component as shown in picture 1, and loosen/fasten the cover as in picture 2;
- ② Rotate counterclockwise to unlock/lock the cover as shown in picture 3.

How To Assemble Snot Container After Cleaning

- 1, Align the buckle on the sealing plate with the bayonet on the base, press it in and turn it to the right to lock;
- 2, Dock the base with the snot container and push the base into the container until you hear a "click" to confirm it is installed.

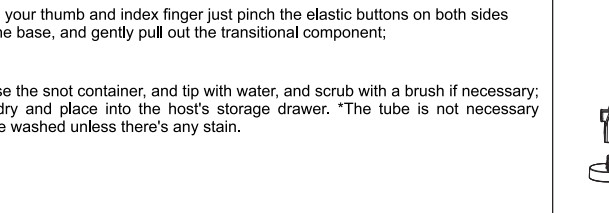

Troubleshooting

Weak or no suction:

- 1, Check if the silicone suction tip is installed correctly;
- 2, Please charge before use;
- 3, Check if other components of the equipment are assembled correctly;
- 4, Check the sponge on the sealing plate for moisture/expansion, if so, replace the sponge (multiple replacements are included in the package)

Injury or pain:

- 1, It is forbidden to use when nosebleeds, head, face and neck are acutely injured;
- 2, Please do not over-insert the silicone suction tip into the nasal cavity or the nasal cavity wall;
- 3, Please do not use it continuously for a long time or use an excessively high suction level.
- 4, In case of severe nasal congestion or other uncertain conditions, please consult a doctor.

No sniffing or only a small amount of sniffing:

- 1, Please place the suction tip correctly in the nasal cavity and keep the head upright;
- 2, Use the appropriate suction head according to the viscosity of the nasal mucus;
- 3, Spray saline into the nasal cavity as needed to dilute the nasal mucus.

Kind Reminder:

- 1, Do not use any cleaning agents or solutions other than mild soap and water.
- 2, After cleaning, the snot container (remove the base and sealing plate) and the silicone tips can be sterilized with boiling water.