

Waterpulse

User's Guide

Product Name: Nasal wash bottle

Product Model: YT-300 Capacity: 300ML

Main structure and features

Waterpulse nasal wash bottle is our company's patented product. It is manufactured with food grade materials. This product consists of a highly resilient bottle, 2 nasal wash tips (1 for adult and 1 for child), a valve button and connecting pipe. The product is simple to operate and easy to carry. It is an ideal choice for personal daily nasal care.

Range of uses

This product is suitable for routine nasal irrigation or daily nasal care for individuals who inhale large amount of dust due to occupational reasons.

How to use


1. Pour a packet of 2.7g nasal wash salt into bottle, then add warm water to 300ml mark on the bottle. The nasal cavity must be rinsed with salt solution. Make sure water temperature is around 37°C. It should feel lukewarm if touched by finger tips. When nasal wash salt completely dissolved in the water, then tighten the bottle cap.
2. Choose the correct tip for adult or children.
3. Correct posture: have your upper body leaning forward, looking down, and with head slightly leaning to one side. Hold bottle upside down, and gently insert the tip into one nostril, relax, open mouth lightly and breathe (through mouth). Do not hold your breath. Do not breathe through nose. Press the valve button (figure 1), and the salt

solution will automatically spray out. Salt solution will enter the nostril and rinse nose and sinus, then come out of the other nostril. After rinsing, release the valve button to stop the solution. Rotate to wash both nostrils.

Press valve button to spray the salt solution. Release the button to stop salt solution. (No need to squeeze the bottle.)



(Figure 1)

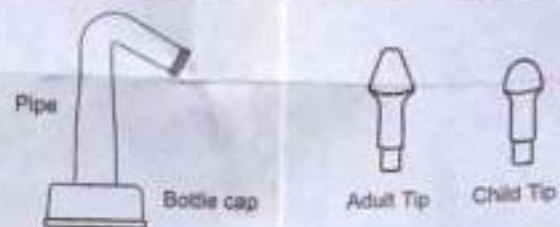

Connecting pipe

Choose the tip what you want

Precautions

1. Do not use water or regular salt instead of nasal wash salt.
2. To avoid discomfort, please configure the correct concentration of salt solution in strict accordance according to user's guide.
3. Do not use if one side of the nose plugs, or when having otitis media or other ear diseases.
4. Children 8 years or under should use this product under the guidance of the physician and parents.
5. Wash the product thoroughly before and after each use.
6. Nasal wash bottle is a personal health care product, do not share with other people.

Storage Methods

The product should be stored in a cool, dry, well-ventilated place. Avoid direct sunlight, and kept it away from corrosive liquid or gas.

Waterpulse (Shanghai) Industrial Co.,Ltd

Address: Level 1, Tower 1, 1501 Jin Sui Road, Pudong New Area, Shanghai, PRC | Tel: 86-21-50493853