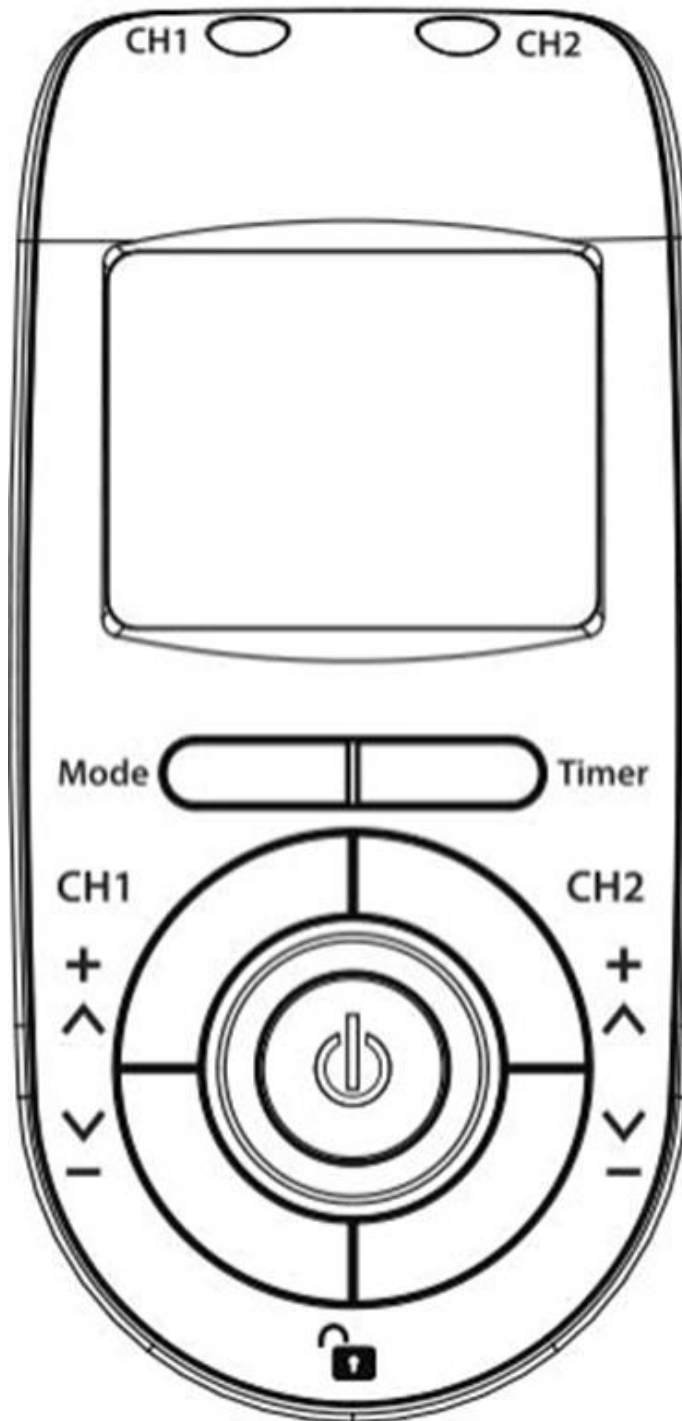
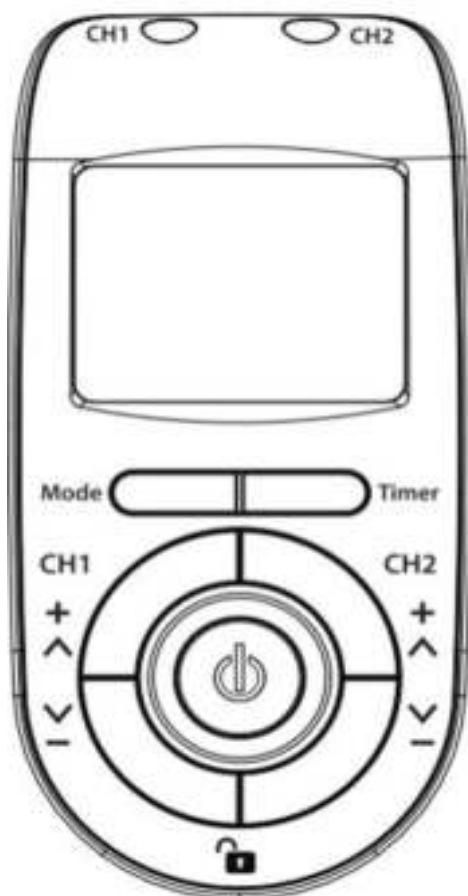
# Kluvaro Digital TENS/EMS Stimulator KRES100B

# KRES100B

Nederlands

## Digitale TENS/EMS Stimulator

### Introductie:

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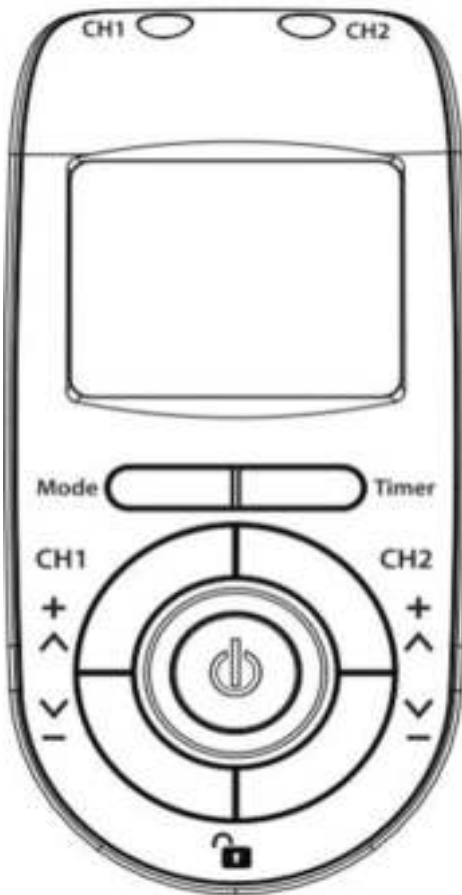
## Digital TENS/EMS Stimulator

### Introduction:

This user manual will help you understand what pain is and how you can use the Kluvaro KRES100B stimulator to relieve your pain. In this manual, you will find all the necessary information and technical data to use the stimulator and its accessories responsibly.

Please read this manual carefully before using the Kluvaro KRES100B and retain the product manual after reading. If you are receiving treatment from a doctor or therapist, always follow their instructions first! They are the right people to give you further explanations.

The Kluvaro portable TENS & EMS device is compact, stylish and equipped with safe and soft electrode pads. It is an advanced device with many technical stimulation possibilities. Many people use the Kluvaro KRES100B for pain relief and we hope that you will also benefit from this device. We wish you good luck with your therapy.

### What's pain?

Pain is a warning signal, indicating that our body is damaged or in danger of being damaged (e.g. when you come too close to a flame with your hand).

Pain can also have a recovery function, preventing damaged parts, such as broken limbs, from being overburdened and making healing difficult.

These types of pains are usually referred to as ACUTE pains.

Acute pains generally last a short time and disappear once the damage or infection has passed. Acute pains can be intense and unpleasant, but have a clear protective function.

Another type of pain, which can also be very unpleasant, is the long-lasting or CHRONIC pain. This pain no longer has a warning or recovery function and is caused by a disturbed functioning of the nerves. Often, there is no obvious cause.

## **Pain relief through nerve stimulation (TENS)**

The Kluvaro KRES100B stimulator transmits electrical pulses through the skin to the nerves by means of electrodes placed on the body. This stimulation of the skin has a dual purpose. On the one hand, the nerves are "distracted" from the pain stimuli present, so that fewer pain signals are transmitted to the brain. On the other hand, the body is encouraged to produce endorphins, which have an analgesic effect.

Pain is a personal experience. Each person experiences pain differently. The same applies to pain management by means of nerve stimulation. The degree of pain relief during stimulation, as well as the number of hours of pain relief after you stop stimulation, differs for everyone.

The application of TENS has no healing effect. It is a symptomatic treatment that suppresses the sensation of pain. Prolonged stimulation is not dangerous or harmful. However, long-term stimulation in the same place can cause skin irritation in the long run.

## **Kluvaro KRES100B stimulator**

The KRES100B is a digital two-channel stimulator. There is one channel on the left and right side of the device. The two channels work independently and do not influence each other. Of course, you can also use the unit as a single-channel unit with the other channel switched off. With the special clip, you can easily attach the stimulator to your trousers, skirt or belt.

The stimulator is powered by batteries and transmits current pulses via cables, electrodes and the skin to underlying nerves.

The device requires little or no maintenance. It is best to clean the device regularly with a damp cloth. Never use aggressive cleaning agents, such as ether, alcohol or abrasives. Never immerse the device in a cleaning liquid or the like!

## **Intended use**

**TENS:** The device is used for the symptomatic relief of chronic pain syndromes and the temporary relief of pain associated with tender and aching muscles in the arm, shoulder, neck, back, waist, abdomen and leg as a result of exertion or through normal household and work activities. It is also used for post-operative and post-traumatic acute pain problems.

**EMS:** The device is used for relaxation of muscle spasms, increasing blood flow in the circulatory system, prevention or delay of atrophy, muscle re-education, maintaining or increasing range of motion, stimulation of calf muscles immediately after surgery to prevent venous thrombosis.

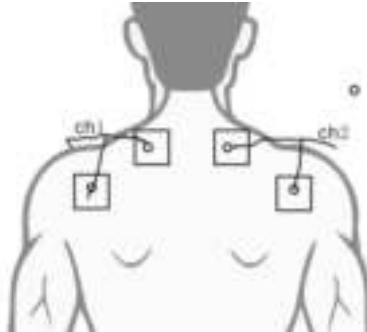
KRES100B is suitable for the shoulder, neck, back, waist, arm, elbow, abdomen, leg and human muscle parts. Kneading, tapping and twisting are the simulated artificial massage techniques to relieve fatigue and improve quality of life.

# Schematic diagram of the application areas


2. Nec


3. Shoulders

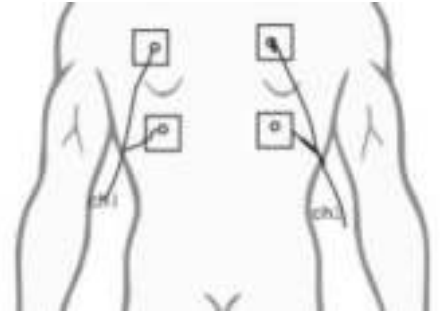4. Back

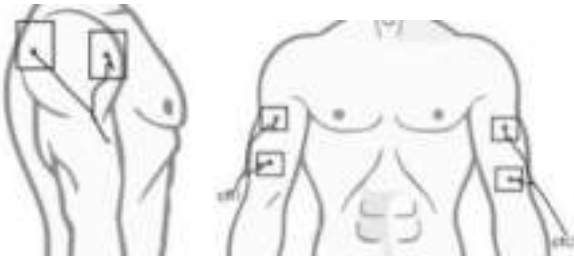
5. Arm


6. Waist

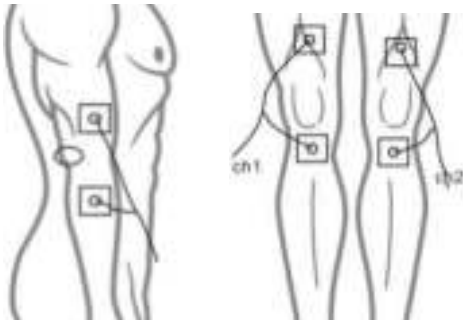7. Elbow and knee

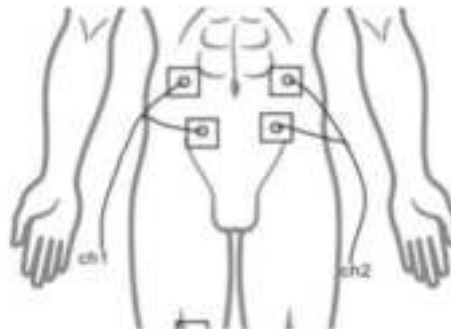
8. Abdomen

9. Leg

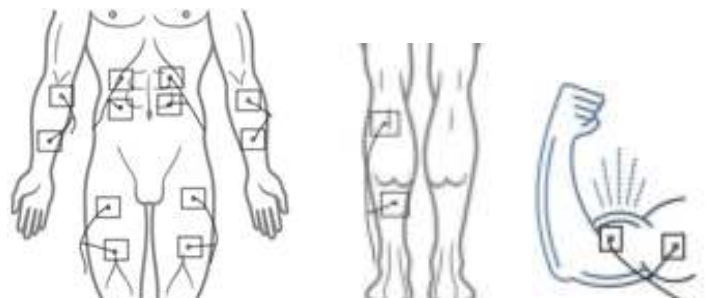10. Muscles

## Notes:

1. Before use, read and follow the instructions carefully.
2. If you feel uncomfortable using this product, stop using it immediately.
3. Turn off the device when removing or moving the electrode pads from the portable stimulator.
4. Before use, check that there is no contact between metal and the electrode pads of the portable TENS & EMS device.
5. Do not use this device for the same body part more than twice a day.
6. Do not repair or disassemble the product yourself.
7. Recycle the used batteries according to their marking. Do not dispose of used batteries at will.

## Prohibitions:

If one of the following conditions occurs: do not use a massager!

1. Please do not apply to areas near the heart or above the neck and to wounds from acute or chronic illnesses.
2. Please do not use in case of cardiac arrhythmias or other heart problems, skin diseases, fever and other illnesses. When in doubt, always contact a doctor first!
3. This product should not be used by people who use a heart rate controller, artificial heart, pacemaker or other electronic medical devices.
5. Do not stick this product in the same place for a long time without removing it.
6. Please do not use while sleeping, driving or taking a bath.
7. Do not place the product in an environment that is too dry or humid.
8. Please do not use in inflammable, explosive and unstable areas.

## Composition of the product:

### LCD-display:

**1.** Ch1, Ch2 output channel ports: Two independent controlled output ports.

**2.** LCD-interface: Displays the usage information

**3 ~ 4.** Keys:

**Mode:** used to select the desired mode

**Time:** used to set the time and select the buzzer mute function

**CH1, CH2 key:** Are the buttons of intensity adjustment: used to adjust the output intensity of CH1, CH2 (0 to 40 levels)

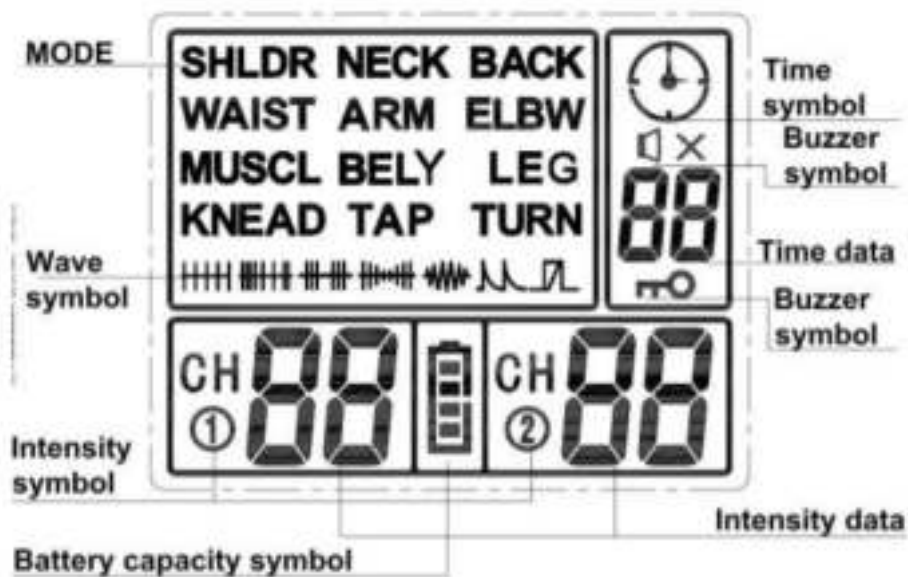
**Switch button:** Used to switch the machine and clear the level of intensity.

**5.** Belt clip

**6.** Battery cover

## The display

### **Operation of electrodes:**

1. Attach the electrode cable to the electrode pad (as shown in the illustration on the left).
2. Before use, clean the skin and gently tear off the protective foil before use (re-tape the foil after use to protect the life of the electrode pads).
3. Stick on the appropriate body parts according to the instructions.
4. Be sure to re-tape the protective film to the original electrode after use to prevent dirt buildup.

Note: Because the conductive adhesive has a strong ability to stick to material, there will be a strong adhesion force.

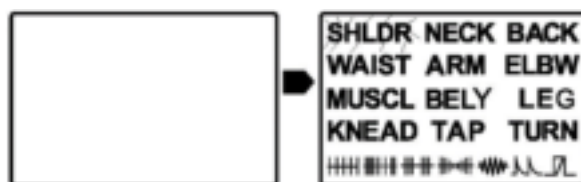
Do not pull forcefully when tearing the storage foil, to avoid the conductive adhesive becoming detached from the electrode.


## Operation of the product:

1. **Turn on the device:** In energy-saving mode, press the  button to enter standby mode.


Indication device:

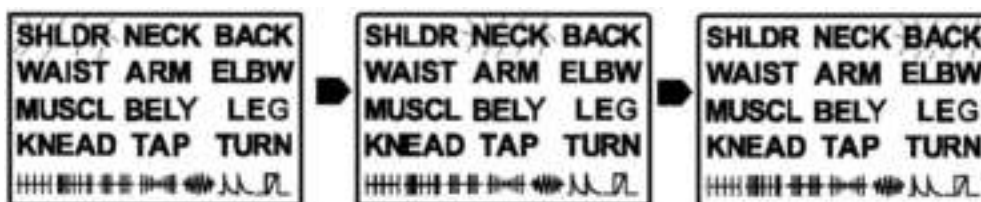2. **Mode selection:** Press the mode key to select the mode and the selected will flash.

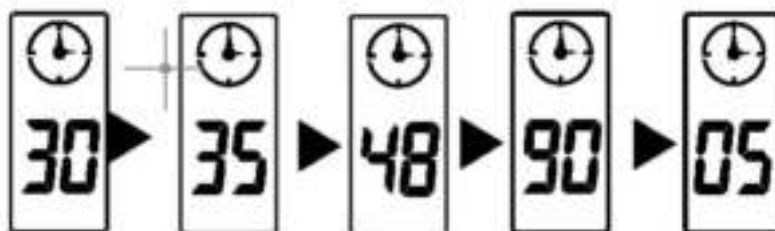3. **Time setting:** Press the clock key to set the use time. The step value is 5-90-5 in 5 minutes per step.

4. **Start treatment or increase intensity:** Press the [CH1 +] or [CH2 +] strength level 0 ~ 40, adjust 1 level in per step.

The timer counts down during the treatment and the output is automatically stopped after 30 minutes of massage.

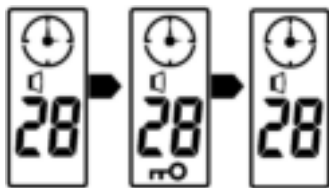
5. **Lower intensity:** Press [CH1-] or [CH2-], adjust 1 level per step.

**Note:** CH1 and CH2 are two independent control channels. CH1 is the output intensity for controlling the CH1 channel and CH2 is the output intensity for controlling the CH2 channel.


6. **End of timer:** the output stops at the end of the timer and there is a return to standby mode.

7. If you press  during the treatment, the intensity becomes 0 and the output is stopped.


8. If no key is pressed for 15 seconds during the treatment, only the machine switch and the lock key can be used. After the locking function has been activated, press the CH key to unlock the machine.

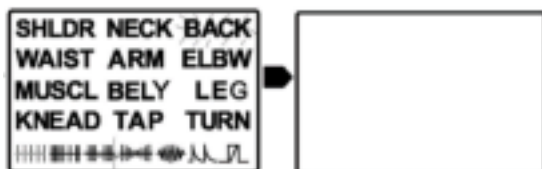
9. **Turn off:** press  in stand-by-mode.

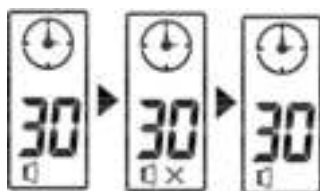
10. **Sound suppression:** Press and hold the time setting button for 2 seconds to switch.

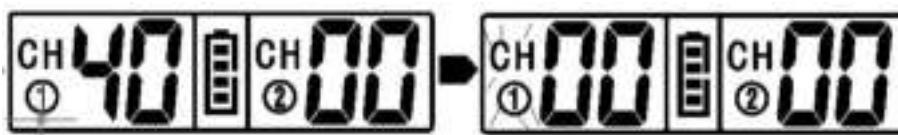11. **Safe intelligent detection function:** If the conductive adhesive does not come into normal contact with the skin, the output strength of the device will be 0 to ensure the safety of the user. To continue using the device, simply reattach the electrode to the skin and it will return to its original strength.

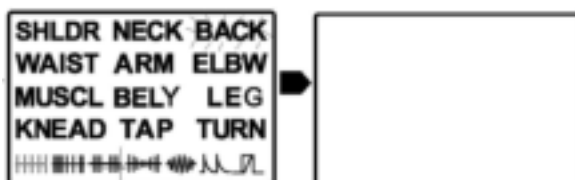12. **Automatically switches to energy-saving mode:** The device automatically switches to energy-saving mode when there is no operation for the next 3 minutes.

**Warm tip 1:** A high output intensity does not necessarily give a better effect.

The most comfortable and effective is, when the skin part feels the current output, then increase 2 ~ 3 or so.

**Warm tip 2:** During the treatment process, it is forbidden to remove the electrode plate of the TENS & EMS DEVICE. If you do, you will get an uncomfortable current stimulation in your fingers. The correct way to do this is to turn off the device and then remove the electrode.

**Low electrical indicator:** If the LCD shows a low battery, the user should replace the battery. If the battery level is too low, the device cannot be switched on and the battery needs to be replaced.

**Warm tip 3:** If you do not press a button within 15 seconds, the display will automatically darken.

## Common sense of electrodes

The conductive electrode is imported from the United States as an advanced raw material, has passed strict biological compatibility tests and can be safely applied to human skin. The conductive electrode is reusable.

After each use, gently wipe the surface of the electrode with your finger and a little water or medical alcohol to remove dirt.

Place the wiped area in the shade for a few minutes and then stick the protective film back on the electrode after the stickiness has been restored. Do not wipe the electrode with paper or a cloth as these may stick to the electrode causing damage and malfunction.

The conductive electrode contains a moisture factor and must be stored correctly. Improper storage or an excessively dry environment will affect the performance of the electrode.

## Failure analysis and exclusion:

Phenomenon	Cause analysis	Suggested elimination methods
Can't boot normally	Low battery	Replacet he battery
	The battery was not installed properly	Reinstall the battery
The key fault	The key is broken	Contact with manufacturer for repair
No pulses output, the user has no feeling	See if the display screen indicates poor connection (the strength symbol will flash)	Reconnect the electrode wire tot he electrode plate and the host
		Reattach the electrode plate to the skin
	The electrode plate is not sticky	Replace the electrode plate
	The electrode wire is broken	Replace the electrode wire
	The device is broken	Contact with manufacturer for repair
When using the device, the acmesthesia is strong in the part of electrode plate	The electrode plate is not sticky	Replace the electrode plate
	The skin is too dry or greasy	Wipe your skin with medical alcohol or a wet towel
	The electrode plate is not sticky	Rinse the electrode plate with medical alcohol or water

If you have followed the above steps and you still cannot use your device normally, or the above problems cannot be solved, please contact the manufacturer or the nearest sales outlet.

## Product specifications:

Product model : KRES100B

Housing material: ABS

Number of channels: Two channels

Power: DC 4.5V 3 \* AAA-battery

Waveform: Bidirectional wave

Load detection: Yes

Display mode: LCD-display

Mode: 12 programs

Output frequency: 1~120HZ

Output pulse width: 100~250uS

Output voltage: 40V ±20%

Grade of strenght: 0 ~ 40 levels

Weight: 120 (g)

Key voice: Yes

## Operating environment:

Temperature: 5°C to 40°C (41°F tot 104°F)

Humidity: 15%RH tot 93%RH

## Storage environment temperature:

-25°C tot 70°C (-13°F tot 158°F)

Humidity: 15%RH tot 93%RH

## Service and warranty

If you need service or information about this product, please do not hesitate to send an e-mail to [info@kluvaro.nl](mailto:info@kluvaro.nl).

We offer a two-year warranty on our products. The customer must be able to show proof of purchase and the warranty must be claimed within two years of the purchase date.

### Please note! The warranty does not apply:

- For damage caused by improper use or repair;
- For parts subject to wear and tear;
- For defects of which the customer was aware at the time of purchase;
- For defects caused by the customer's own fault;
- For damage caused by third parties.

## Disclaimer

Changes subject to change; specifications may change without notice.

Flux Forward  
Van Neslaan 67  
3742MP Baarn  
Netherlands  
12/2020 – V1.

©Kluvaro  
[www.kluvaro.nl](http://www.kluvaro.nl)