InnovaGoods®**User's Manual****ENGLISH****INSTRUCTIONS**

- Slide down the battery lid, place the two 1.5 V AA batteries in the right polarity.
- Insert a rubber tip into the suction pipe of the Suction Ear Cleaner InnovaGoods® until the last slot.
- Switch to the ON position, bring the rubber tip to the ear and introduce it with smooth movements without going too deep.
- The case of the suction pipe is detachable, as well as the inner filter. Clean these pieces and the rubber tips with soapy water. Use the brush provided.

WARNINGS

Do not mix new and used batteries in the compartment. If you are not going to use the unit for a long period of time, remove the battery. Do not expose batteries to high temperature or direct sunlight. Do not attempt to open them, do not dispose of them in fire, there is risk of explosion. Clean with a damp cloth. Do not use abrasive products for cleaning. Do not immerse the appliance in liquid. Do not handle it with wet hands. Do not use near water or damp areas. Suitable for domestic use only. Use the device for its intended purpose only. Store the device in a dry, dust-free place. Do not leave the device unsupervised while in use. To avoid risks, do not use it if any parts are damaged and do not attempt to disassemble or repair it. This device is not intended for use by children or persons with reduced physical, sensory or mental capabilities, unless they are supervised by an adult responsible for their safety. This device is not household waste. Please follow the environmental regulations in place for its recycling.

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ALL RIGHTS RESERVED. MADE IN P.R.C.