EN														
<p><b>CUBE Flexible Pessary</b></p> <p><b>TANDEM-CUBE FlexiblePessary</b></p> <p>Support for Stage III prolapse/procidentia <span style="float:right"><b>R<sub>x</sub>Only</b></span></p> <p><b>AVAILABLE MODELS</b></p> <table> <tbody><tr> <td>CUBE</td> <td>CUBE WITH DRAINAGE HOLES</td></tr> <tr> <td><span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)</td> <td><span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)</td></tr> <tr> <td><span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)</td> <td><span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)</td></tr> <tr> <td>TANDEM-CUBE</td> <td></td></tr> <tr> <td><span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)</td> <td><span><b>[REF]</b></span> <b>MXPPECT</b> (Pessary, TRIMO-SAN™, Jel Jector)</td></tr> <tr> <td><span><b>[XREF]</b></span></td> <td></td></tr> <tr> <td><b>Single-Patient Use</b></td> <td><b>CE0086</b></td></tr> </tbody></table> <p><b>Wash pessary with mild soap and thoroughly rinse prior to initial use.</b></p> <p><b>DESCRIPTION</b></p> <p>The Millex® CUBE Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.</p> <p><b>WARNINGS</b></p> <ul style="list-style-type: none"><li>Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.</li> <li>Do not use these pessaries on a patient with a known silicone allergy.</li> <li>Chemicals in various vaginal preparations can interact with the pessary</li></ul> <p><b>CooperSurgical</b></p>	CUBE	CUBE WITH DRAINAGE HOLES	<span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)	<span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)	TANDEM-CUBE		<span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPPECT</b> (Pessary, TRIMO-SAN™, Jel Jector)	<span><b>[XREF]</b></span>		<b>Single-Patient Use</b>	<b>CE0086</b>
CUBE	CUBE WITH DRAINAGE HOLES													
<span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)													
<span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)	<span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)													
TANDEM-CUBE														
<span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPPECT</b> (Pessary, TRIMO-SAN™, Jel Jector)													
<span><b>[XREF]</b></span>														
<b>Single-Patient Use</b>	<b>CE0086</b>													
ES														

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material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.

INDICATIONS FOR USE-CUBE
<ul style="list-style-type: none"><li>The CUBE pessary is effective in the support of Stage III prolapse. The CUBE pessary will give additional support in those cases associated with complete lack of vaginal tonicity. The CUBE pessary is also effective in patients having procidentia combined with rectocele and/or cystocele.</li> <li>The CUBE pessary is also indicated in vaginal wall prolapse. The unique support is due to the suction action of the six concavities, which, by a slight negative pressure, supports the prolapse and the vaginal walls.</li> <li>If only bulk is sufficient to support the prolapse, the CUBE with holes is indicated. The CUBE pessary should be removed nightly and washed with mild soap and water, then thoroughly rinsed. If possible, leave out overnight.</li></ul>
INDICATIONS FOR USE-TANDEM CUBE
<ul style="list-style-type: none"><li>For effective support in Stage III prolapse. The TANDEM-CUBE pessary will give additional support in those cases associated with complete lack of vaginal tonicity, where the patient is unable to retain even the largest size single CUBE pessary. The unique support is due to the suction action of the ten concavities, which, by a slight negative pressure, supports the prolapse and the vaginal walls. The TANDEM-CUBE pessary should be removed nightly and washed with mild soap and water, then thoroughly rinsed. If possible, leave out overnight.</li></ul>
CONTRAINDICATIONS
<ul style="list-style-type: none"><li>The presence of pelvic infections and/or lacerations</li> <li>A noncompliant patient</li> <li>Endometriosis</li> <li>Pregnant patient</li></ul>
RECOMMENDED CLEANING INSTRUCTIONS*
<ol style="list-style-type: none"><li>Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer’s recommended concentration. Prepare this solution in a container large enough to fully submerge the device.</li> <li>Soak and Scrub <ol style="list-style-type: none"> <li>Soak the device in the container of prepared soap solution for a minimum of 5 minutes.</li></ol></li></ol>

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- Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
  - Following scrub, inspect device for visible soil residue.
- Rinse
  - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
  - Allow the device to dry.

\*New pessaries are powdered with food-grade powder.

## PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION
<p>COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.</p> <ul style="list-style-type: none"><li>To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.</li> <li>It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.</li> <li>A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner.</li> <li>It is important to evaluate the effects of the vaginal gel TRIMO-SAN.</li></ul>

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN
<ul style="list-style-type: none"><li>Any difficulty in urinating</li> <li>Any discomfort</li> <li>Any changes in the color or consistency of vaginal discharge</li> <li>Any increase in the amount of vaginal discharge or vaginal bleeding</li> <li>Any foul odor associated with vaginal discharge</li> <li>Vaginal itching</li> <li>If the pessary falls out</li></ul>

**For medical emergencies and for all medically related advice, consult your healthcare professional.**

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## FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

INSTRUCTIONS
<ol style="list-style-type: none"><li>Wear dry gloves. The pessary is compressed between the thumb and forefingers (see Figure 1). If necessary, the entering edge can be coated with TRIMO-SAN.</li> <li>Use the fingers of the other hand to spread the labia and guide the pessary into the vagina. Check the fit (see Figure 2).</li> <li>Have the patient sit, stand and bear down. Examine the patient while she is in the standing position to ensure pessary has not shifted position. The patient should not feel the pessary once it is in position.</li> <li>Have the patient (or caregiver) insert and remove the CUBE pessary several times before the patient leaves your office.</li> <li>It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.</li></ol>

<span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)
<span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)	<span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)
TANDEM-CUBE	
<span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPPECT</b> (Pessary, TRIMO-SAN™, Jel Jector)
<span><b>[XREF]</b></span>	
<b>Single-Patient Use</b>	<b>CE0086</b>

**Wash pessary with mild soap and thoroughly rinse prior to initial use.**

DESCRIPTION
<p>The Millex® CUBE Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.</p> <p><b>WARNINGS</b></p> <ul style="list-style-type: none"><li>Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.</li> <li>Do not use these pessaries on a patient with a known silicone allergy.</li> <li>Chemicals in various vaginal preparations can interact with the pessary</li></ul> <p><b>CooperSurgical</b></p>

DESCRIPTION
<p>The Millex® CUBE Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.</p> <p><b>WARNINGS</b></p> <ul style="list-style-type: none"><li>Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.</li> <li>Do not use these pessaries on a patient with a known silicone allergy.</li> <li>Chemicals in various vaginal preparations can interact with the pessary</li></ul> <p><b>CooperSurgical</b></p>

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EN										
<p><b>CUBE Flexible Pessary</b></p> <p><b>TANDEM-CUBE FlexiblePessary</b></p> <p>Support for Stage III prolapse/procidentia <span style="float:right"><b>R<sub>x</sub>Only</b></span></p> <p><b>AVAILABLE MODELS</b></p> <table> <tbody><tr> <td>CUBE</td> <td>CUBE WITH DRAINAGE HOLES</td></tr> <tr> <td><span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)</td> <td><span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)</td></tr> <tr> <td><span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)</td> <td><span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)</td></tr> <tr> <td>TANDEM-CUBE</td> <td></td></tr> <tr> <td><span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)</td> <td><span><b>[</b></span></td></tr></tbody></table>	CUBE	CUBE WITH DRAINAGE HOLES	<span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)	<span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)	TANDEM-CUBE		<span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)	<span><b>[</b></span>
CUBE	CUBE WITH DRAINAGE HOLES									
<span><b>[REF]</b></span> <b>MXPEC</b> (Pessary Only)	<span><b>[REF]</b></span> <b>MXPECH</b> (Pessary Only)									
<span><b>[REF]</b></span> <b>MXPPEC</b> (Pessary, TRIMO-SAN™, Jel Jector)	<span><b>[REF]</b></span> <b>MXPPECH</b> (Pessary, TRIMO-SAN™, Jel Jector)									
TANDEM-CUBE										
<span><b>[REF]</b></span> <b>MXPECT</b> (Pessary Only)	<span><b>[</b></span>									

FR
<div><span><span></span></span></div> <div><b>MILEX</b><sup>®</sup></div> <div><small>a CooperSurgical Brand</small></div>
<div> <div><span><b>Pessaire CUBE souple</b></span></div> <div><span><b>Pessaire CUBE TANDEM souple</b></span></div> </div>
Soutien des prolapsus/procidences de stade III <span><b>R<sub>x</sub>Only</b></span>
<div> <div><span><b>MODÈLES DISPONIBLES</b></span></div> <div> <div> <div><span><b>CUBE</b></span></div> <div><span><b>CUBE AVEC ORIFICES DE DRAINAGE</b></span></div> </div> <div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPEC</b></span> (Pessaire uniquement)</div> <div><span><b>MXKPEC</b></span> (Pessaire, TRIMO-SAN™, Jel Jector)</div> </div> <div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPECH</b></span> (Pessaire uniquement)</div> <div><span><b>MXKPECH</b></span> (Pessaire, TRIMO-SAN™, Jel Jector)</div> </div> </div> </div> <div> <div><span><b>CUBE TANDEM</b></span></div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPECT</b></span> (Pessaire Uniquement)</div> <div><span><b>MXKPECT</b></span> (Pessaire, TRIMO-SAN™, Jel Jector)</div> </div>

SV
<div><span><span></span></span></div> <div><b>MILEX</b><sup>®</sup></div> <div><small>a CooperSurgical Brand</small></div>
<div> <div><span><b>CUBE flexibel livmoderring</b></span></div> <div><span><b>TANDEM-CUBE flexibel livmoderring</b></span></div> </div>
Stöd för tredje gradens framfall/procidencia <span><b>R<sub>x</sub>Only</b></span>
<div> <div><span><b>TILLGÅENGLIGA MODELLER</b></span></div> <div> <div> <div><span><b>CUBE</b></span></div> <div><span><b>CUBE MED DRÄNERINGSHÅL</b></span></div> </div> <div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPEC</b></span> (Endast livmoderring)</div> <div><span><b>MXKPEC</b></span> (Livmoderring, TRIMO-SAN™, Jel Jector)</div> </div> <div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPECH</b></span> (Endast livmoderring)</div> <div><span><b>MXKPECH</b></span> (Livmoderring, TRIMO-SAN™, Jel Jector)</div> </div> </div> </div> <div> <div><span><b>TANDEM-CUBE</b></span></div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPECT</b></span> (Endast livmoderring)</div> <div><span><b>MXKPECT</b></span> (Livmoderring, TRIMO-SAN™, Jel Jector)</div> </div>

FR
<div><span><span></span></span></div> <div><b>MILEX</b><sup>®</sup></div> <div><small>a CooperSurgical Brand</small></div>
<div> <div><span><b>CUBE flexibel livmoderring</b></span></div> <div><span><b>TANDEM-CUBE flexibel livmoderring</b></span></div> </div>
Stöd för tredje gradens framfall/procidencia <span><b>R<sub>x</sub>Only</b></span>
<div> <div><span><b>TILLGÅENGLIGA MODELLER</b></span></div> <div> <div> <div><span><b>CUBE</b></span></div> <div><span><b>CUBE MED DRÄNERINGSHÅL</b></span></div> </div> <div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPEC</b></span> (Endast livmoderring)</div> <div><span><b>MXKPEC</b></span> (Livmoderring, TRIMO-SAN™, Jel Jector)</div> </div> <div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPECH</b></span> (Endast livmoderring)</div> <div><span><b>MXKPECH</b></span> (Livmoderring, TRIMO-SAN™, Jel Jector)</div> </div> </div> </div> <div> <div><span><b>TANDEM-CUBE</b></span></div> <div><span><b><span>[</span>REF<span>]</span></b></span></div> <div><span><b>MXPECT</b></span> (Endast livmoderring)</div> <div><span><b>MXKPECT</b></span> (Livmoderring, TRIMO-SAN™, Jel Jector)</div> </div>

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