english

Thank you for purchasing the Starlyf® Handy Grasp PRO, the safety bar offering stability, balance and confidence. The Starlyf® Handy Grasp PRO can be attached quickly and easily just about anywhere. It's also portable and easy to take with you. Please read the instructions and warnings before using the product for the first time and keep them for future reference.

CAUTION: This product is designed as a temporary accessory to give the user stability. It cannot be used as a grab bar by disabled or elderly persons. It is not designed to bear the user's weight when standing up.

Safety Warnings

- Do not use in water.
- Do not use as a point of support or to keep your balance when standing up.
- Do not use at extreme temperatures.
- The product should not be used by children.
- It is not designed to help users regain their balance.
- Do not use it on porous surfaces such as cement, granite, marble or wood, or cracked, painted or rough surfaces.
- Do not use on fragile surfaces or loose tiles.
- Always test the Starlyf® Handy Grasp PRO manually before using to ensure it has not become loose since the previous use. In cases of doubt, reattach the product. The Starlyf® Handy Grasp PRO should be reattached regularly to ensure it is firmly attached.
- It is not designed for outdoor use.

CAUTION

The Starlyf® Handy Grasp PRO is designed to stay attached to smooth, clean, dry surfaces under normal conditions. However, it is not guaranteed to stay attached in all cases on all surfaces. Over time, the suction cups may come unstuck or lose adherence. It is therefore essential to always make sure it is firmly attached before each use, even the first one.

IMPORTANT: The Starlyf® Handy Grasp PRO should be attached only to COMPLETELY smooth surfaces. The slightest imperfection on the surface can prevent the suction cups from working properly.

NOTE: To ensure the product works, it must be properly attached. Follow the instructions and contact a specialized technician if you have any questions or difficulties.

The Set Contains:
Starlyf® Handy Grasp PRO (with two suction cups)
Instruction Manual

ATTACHING AND USE

Before use:

- Make sure there is no danger of losing your balance when you try to attach your Starlyf® Handy Grasp PRO. Do not try grab the handle while you are attaching it.
- The Starlyf® Handy Grasp PRO should be attached to clean, dry, solid surfaces that can hold the handle. The Starlyf® Handy Grasp PRO can be attached to tiles, glass and laminate.
- Do not attach the Starlyf® Handy Grasp PRO on damaged surfaces.

Use:

- Flip up the levers on the ends of the handle to release the suction cups (Fig. 1).
- Make sure you attach the Starlyf® Handy Grasp PRO to a flat, clean, non-porous surface. Do not attach to cracked surfaces or over tile grout lines (Fig. 2). The product may not work properly if it is not installed correctly.
- Push the Starlyf® Handy Grasp PRO against the surface and flip down both levers (Fig. 3).

Removal:

- Make sure there is no danger of losing your balance when you try to remove your Starlyf® Handy Grasp PRO. Do not try to grab the handle while you are removing it.
- Flip up both suction cup levers (Fig. 4).
- Immediately slide your finger under each suction cup and pull the edge of the suction cup to release suction (Fig. 5). Another option is to slide the handle to the edge of the surface. Now you can remove your Starlyf® Handy Grasp PRO.
- Never remove the Starlyf® Handy Grasp PRO directly, without flipping up the levers. This could cause damage to the product and even to yourself.
- If you cannot remove the product by following the above recommendations, try using a small screwdriver to help lift one end of the suction cup and release the handle.

Cleaning:

To clean your Starlyf® Handy Grasp PRO, we recommend using a damp cloth. Never immerse the handle in water. The base of the suction cups should also be cleaned periodically. Do not use abrasive chemical products or scouring pads that could scratch the plastic. To clean the surface where the Starlyf® Handy Grasp PRO was attached, first remove the handle, wash the surface and let it dry completely before reattaching the handle.

Warranty: This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country. This warranty does not cover damage resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

Australia and New Zealand only.

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law and the New Zealand Consumer Guarantees Act 1993, both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries

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Starlyf® HANDY GRASP PRO is a registered EU/CTM trade mark. International patents pending. All imitations will be prosecuted.