

Baby Nasal Aspirator

Safety Instructions

- This product is only used to clean the nose and derivatives of the baby's nasal cavity.
- Prevent solid hard objects larger than 3mm from entering the suction nozzle.
- Please do not throw it away to prevent damage to this product.
- Please use the accessories supplied by the manufacturer.
- This product is not a toy. Please keep this product in a safe place and do not let the child operate it.
- Please use AA alkaline batteries, or rechargeable batteries, do not use NiMH batteries.
- When not in use for more than two weeks, remove the battery, please follow the instructions to install the battery.

Scope Of Application

This product is suitable for use in the home to help suck the baby's fluid nasal putum or derivatives.

Precautions Before Use

- Please do not forget to use the suction nozzle and container before use. It is recommended to soak it in boiling water at 100 °C for several minutes. Do not use high temperature anti-virus such as microwave oven.
- Do not immerse the product in the liquid as a whole.
- Please check the seal for damage and assembly.
- Please wash your hands before use.
- The operating mode of this product is continuous operation. In order to extend the battery life, it is recommended to run for up to 10 minutes and stop for 5 minutes.

Features

- Three-speed suction is available.
- The beautiful music makes the baby happy.
- Reusable blister cups are easy to use, safe and hygienic and easy to keep your baby's nose clean.

Baby Nasal Aspirator is a brand new baby cleaning product that is soft, efficient and easy to clean. Infants under one year of age can only breathe through the nose, so the baby aspirator can keep the nasal passages by regularly cleaning the secretions produced by the nasal cavity!

Product preservation tips


- Clean the product with a cleaning agent such as alcohol or detergent.
- You can sterilize the removed part (the nasal mask) with boiling water or steam, but it cannot be sterilized in a microwave oven.
- After disinfection, clean the product with a soft, dry cloth.
- Avoid storing the product in direct sunlight and in a humid environment.
- Avoid using corrosive chemicals to clean the product.

Note






When the product is not used for two days, please remove the battery to prevent leakage of the product.

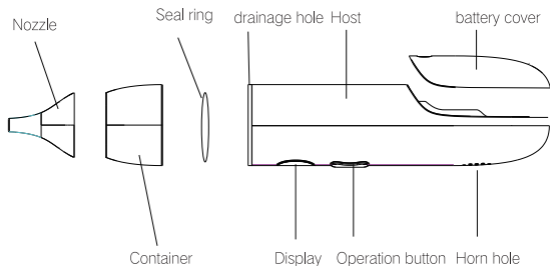
Product introduction


The baby nasal aspirator is composed of a suction nozzle, a container, a line control component, a water pump assembly, and a housing assembly.





Product instructions


- Insert the battery according to the positive and negative indications of the battery in the battery compartment.
- Press the “  ” button to activate the baby nasal aspirator.
- Press the “  ” button to adjust the gear position for a total of 3 files.
- Carefully place the suction nozzle on the outer edge of the nasal cavity and press “  ” to effectively clean the nasal cavity.
- After use, first remove the suction nozzle from the outer edge of the nasal cavity, and then press the “  ” button to turn off the device.
- before using it. You can choose whether to turn on the music, press the “  ” button to turn on the music, and then press to turn it off.


 Warning: During use, ensure that the baby's mouth is open to ensure air circulation in the nasal cavity, so as to avoid negative pressure on the head and cause discomfort.

 Warning: When the baby aspirator is working, do not block the other nostril to avoid negative pressure on the head and cause discomfort.


 Warning: Clean the product before each use.


 Warning: Do not immerse the entire product in a liquid.

 Warning: Disinfect the nasal nozzle/transparent cover after use to prevent cross-contamination.

 Warning: The product has no anti-backflow device. The baby nasal suction device needs to be equipped with a blister cup to prevent the product from inhaling the nose into the product and damage the product.



 Warning: When using two-thirds of the liquid in a blister cup, clean it to prevent liquid from flowing inside the product and damaging the product.

 Warning: After cleaning the “nozzle” with boiling water or steam, the user should clean the water stain in the suction nozzle to prevent the baby suction device from sucking the water stain into the baby nasal suction device and damage the product.


Product assembly and cleaning

Assembly

1. Open the battery cover and load two alkaline batteries.
2. Load the suction nozzle.


Cleaning

1. The body can only be wiped with a wet towel.
2. If the machine is not clear or can be used for one week, it can be inhaled with 100 °C boiling water for circulation cleaning.

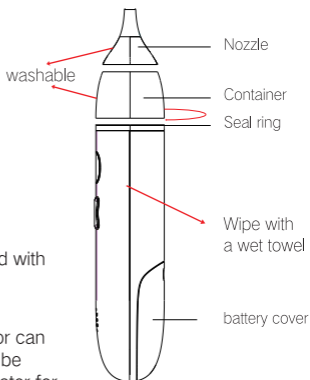

Replacement battery

1. Uncover the baby nasal suction device battery cover.
2. Load two No. 5 AA alkaline batteries and make sure the polarity is correct.
3. Install the battery cover.

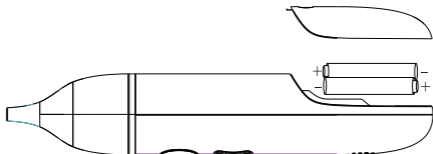

Product parameters

product name:	Baby Nasal Aspirator
Product number:	FY-A200
Package dimensions:	21.5x12x4.7cm
product weight:	300g
Suction range:	-50KPA
battery:	Two-cell alkaline battery
Blister cup maximum capacity:	5.7ml

Troubleshooting

If the following problems occur, please follow the troubleshooting guide. If you can't solve it, please contact our after-sales service department.

problem	Solution
No suction	<ol style="list-style-type: none">1. Check the seal position2. Check if the transparent cover is in place3. Check if the mouthpiece is in place
Can't open the machine	<ol style="list-style-type: none">1. Check if the battery is correct.2. Replace the new battery