Smart Chair

Travel

Geachte reiziger,

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Trouble Code

Related fault codes and instructions are as follows:

Fault	LED1	LED2	LED3	LED4	LED5	Flashing interval (s)	Recommended treatment
Unregistered					●	0.5	Contact the repair vendor
Joystick abnormal when power on				●		0.5	The joystick stays in the middle position when you turn it on
Communication trouble				●	●	0.5	Check if the connector of the connection controller is loose
Joystick abnormal			●			0.5	Contact the repair vendor
Controller failure			●		●	0.5	Contact the repair vendor
Left motor Over temperature	●					0.5	Turn off the scooter for a while before the temperature is restored.
Left motor over current	●	●				0.5	Check if the circuit connector is loose
Left motor brake is unlocked	●		●			0.5	Check if the left motor brake is open
Left motor Hall failure	●	●	●			0.5	Check if the circuit connector is loose

LED lights on joystick

See Page 36 for more info.

Left motor MOS tube failure	●				●		0.5	Contact the repair vendor
Left motor missing	●	●			●		0.5	Check if the circuit connector is loose
Left motor stall	●			●	●		0.5	Check if the left motor is entangled with foreign matter, and if the power is off, the resistance is too large.
Right motor Over temperature				●			0.5	Turn off the scooter for a while before the temperature is restored.
Right motor over current				●	●		0.5	Check if the circuit connector is loose
Right motor brake is unlocked				●			0.5	Check if the right motor brake is open
Right motor Hall failure				●	●		0.5	Check if the circuit connector is loose
Right motor MOS tube failure				●		●	0.5	Contact the repair vendor
Right motor missing				●	●		0.5	Check if the circuit connector is loose
Right motor stall				●	●		0.5	Check if the left motor is entangled with foreign matter, and if the power is off, the resistance is too large.

PS: ● LED is lighting

No.	Problem Cause	Solution
1	<p>Power Problem</p> <ol style="list-style-type: none"> 1. Faulty connection of the control panel and the connector. 2. Low power supply or the power cut off automatically. 3. Fault of the 4-pin plug connecting to the battery 	<p>Re-plug the 5-pin plug into the socket on the side of the seat.</p> <p>If the power is off automatically, the batteries must be recharged within 2 hours, and should be charged for 6-7 hours.</p> <p>Pull out the battery and check the contact for any fault and re-insert the battery.</p> <p>Charge the battery immediately.</p>
2	<p>Control Panel Fault</p> <ol style="list-style-type: none"> 1. The battery indicator displays red alarm: low battery power. 2. The control panel does not display. 3. Communication failure: the controller broadcasts the malfunction sound. The center icon is the exclamation mark. The wheelchair cannot drive. 4. The center icon display left/right wheel malfunction. The wheelchair is unable to drive. Left/right motor is broken. 5. The electromagnetic brake is in "Unlock" position and the wheelchair cannot move. 	<p>Reconnect the plugs of the two ends of the control panel cable, or replace the control panel.</p> <p>Power off the wheelchair. Then check if the plug connection of the controller is loose. Reboot after fastening.</p> <p>Re-insert and tighten the motor connector and check if the motor unlock button is locked. If the problem cannot be solved, please contact the distributor to repair.</p> <p>Push the electromagnetic brake lever to "Lock" position.</p>
3	<p>Motor Fault</p> <ol style="list-style-type: none"> 1. Noise and vibration occur in the running course of the motor. 	<p>Contact the customer service.</p>

4	Battery Range Decreases	<ol style="list-style-type: none"> 1. Drive in the Low temperature environment below zero centigrade. 2. Drive on the slopes or rough roads. 3. Battery use has exceeded its service life. 	<p>It is normal as this is the characteristics of lithium batteries.</p> <p>It is normal as driving on these places can accelerate battery consumption.</p> <p>Contact customer service to replace the battery.</p>
5	Charging Fault	<ol style="list-style-type: none"> 1. Incorrect connection between control panel and plug 2. Controller is damaged. 3. Charger plug is damaged. 4. Charging plug is loosened. 	<p>Reconnect the 3-pin plug into the battery connector on the side of the seat.</p> <p>Contact customer service to replace the controller.</p> <p>Contact customer service to replace the charger plug.</p> <p>Recheck and firmly reconnect.</p>
6	Fail to Start When Power On	<p>After not using for a period or the over-discharge protector is activated.</p>	<p>Power on again after charging. If the problem cannot be solved, please contact the distributor to repair.</p>
7	Unable to fold	<p>locking assembly</p> <p>front wheel interference</p>	<p>Check if there is something in the lock head that causes the operation jam and can't move.</p> <p>Check if the front wheel is caught in the frame during folding.</p>

Material Safety Data Sheet (MSDS) for passengerships and airlines.

Version: V1.3

MSDS

MATERIAL SAFETY DATA SHEET

Prepared For : Changzhou Fangyu New Energy Technology Co., Ltd
No. 35 Leshan Road, Xinbei District, Changzhou City, Jiangsu Province, China

Prepared By : Shenzhen LCS Compliance Testing Laboratory Ltd.
101, 201 Building A and 301 Building C, Juji Industrial Park, Yabianxuezhui, Shajing Street, Baoan District, Shenzhen, Guangdong, China

Issue Date : 2020.01.02

Report Number : LCS190610174ASD

Written by: Taylor sheng

Approved by: 

Material Safety Data Sheet

REPORT NO.: LCS19M101744SD

* The MSDS is prepared based on the information provided by client. The contents and format of this MSDS are revised as per client's request.

Section 1-Chemical Product and Company Identification

Product Name	POLYMER Lithium-Ion Battery
Model	FY24-12
Trade Mark	N/A
Rating	24V, 12Ah, 268Wh
Weight	2.41kg
Manufacturer	Changzhou Fengyu New Energy Technology Co., Ltd
Manufacturer address	No. 38 Leshan Road, Xibei District, Changzhou City, Jiangsu Province, China
Emergency Telephone	+86-519-83859198
Fax	+86-519-83859098

Section 2- Composition Information

Chemical Composition	CAS No.	Weight (%)	Trade Secret
Aluminum Foil(Al)	7429-90-5	5	*
Copper Foil(Cu)	7440-50-8	7-10	*
Lithium Cobalt Oxide(LiCoO ₂)	12190-79-3	2-5	*
Nickel(Ni)	7440-02-0	2-5	*
Ethylene carbonate(EC C ₄ H ₆ O ₃)	96-49-1	5	*
Dimethyl carbonate(DMC C ₄ H ₈ O ₃)	615-38-6	2-5	*
LiNiCoMn1-x-y-z	N/A	58-62	*
Graphite (C)	7782-43-5	12-15	*
Polyvinylidene fluoride (PVDF C ₂ H ₂ F ₂)	24937-79-8	6	*

* ** The exact percentage (composition) of composition has been withheld as a trade secret.

Section 3- Hazards Identification

Emergency overview:	N/A
Classification according to GHS:	Not a dangerous substance according to GHS

Material Safety Data Sheet

REPORT NO.: LCS190618IT6AGD

Label elements:	
Hazard pictogram(s)	Not Available
Signal word	Not Available
Hazard statement(s)	Not Available
Precautionary statement(s):	
Prevention	Not Available
Response	Not Available
Disposal	Not Available
Environmental hazards:	No relevant information
Important symptoms:	See section 11 for more information
Section 4- First Aid Measures	
Eye contact	Flush eyes with plenty of water for least 15 minutes, occasionally lifting the upper and lower eyelids. Get medical aid.
Skin contact	Remove contaminated clothes and rinse skin with plenty of water or shower for 15 minutes. Get medical aid.
Inhalation	Remove from exposure and move to fresh air immediately. Use oxygen if available.
Ingestion	Give at least 2 glasses of milk or water. Induce vomiting unless patient is unconscious. Call a physician.
Section 5- Fire Fighting Measures	
Flash Point	N/A
Auto-ignition Temperature	N/A
Extinguishing Media	H ₂ O, CO ₂
Special Fire-Fighting Procedures	Self-contained breathing apparatus
Unusual Fire and Explosion Hazards	Cell may vent when subjected to excessive heat-exposing battery contents
Hazardous Combustion Products	Carbon monoxide, carbon dioxide, lithium oxide fumes
Section 6- Accidental Release Measures	
Personal precautions, protective equipment and emergency procedures: If the battery is released, remove personnel from area until fumes dissipate. Provide maximum ventilation to clear out hazardous gases. The preferred response is to leave the area and allow the vapors to dissipate. Avoid skin and eyes contact or inhalation of vapors. Remove spilled liquid with absorbent and nonreacted. If leakage of the battery happens, liquid could be absorbed with sand, earth or other inert substance and contaminated area should be	

Material Safety Data Sheet

REPORT NO.: LCS100610174ASD

ventilated room.	
<p>Environment precautions: Do not allow product to reach sewage system or any water source. Inform respective authorities in case of seepage into water course or sewage system. Do not allow to enter sewers/ surface or ground water.</p>	
<p>Methods and material for containment and cleaning up: If battery casing is dismantled, small amounts of electrolyte may leak. Collect all released material in a plastic lined container. Dispose off according to the local law and rules. Avoid leached substances to get into the earth, canalisation or waters.</p>	
Section 7- Handling and Storage	
Handling:	<p>The battery should not be opened, destroyed or incinerated, since they may leak or rupture and release to the environment the ingredients that they contain in the hermetically sealed container. Do not short circuit terminals, or over charge the battery, forced over-discharge, throw to fire. Do not crush or puncture the battery, or immerse in liquids.</p>
Storage:	<p>Avoid mechanical or electrical abuse. Storage preferably in cool, dry and ventilated area, which is subject to little temperature change. Storage at high temperatures should be avoided. Do not place the battery near heating equipment, nor expose to direct sunlight for long periods.</p>
Other Precautions:	<p>The battery may explode or cause burns, if disassembled, crushed or exposed to fire or high temperatures. Do not short or instal with incorrect polarity.</p>
Section 8- Exposure Controls/Personal Protection	
Engineering Controls:	<p>Use local exhaust ventilation or other engineering controls to control sources of dust, mist, fumes and vapor. Keep away from heat and open flame. Store in a cool, dry place.</p>
Personal Protective Equipment:	<p>Respiratory Protection: Not necessary under normal conditions. Skin and body Protection: Not necessary under normal conditions. Wear suitable protective clothing and gloves if handling an open or leaking battery. Hand protection: Wear suitable gloves if handling an open or leaking battery. Eye Protection: Not necessary under normal conditions. Wear safety glasses if handling an open or leaking battery.</p>
Other Protective Equipment:	<p>Have a safety shower and eye wash fountain readily available in the immediate work area.</p>
Hygiene Measures:	<p>Do not eat, drink, or smoke in work area. Maintain good housekeeping.</p>
Section 9- Physical and Chemical Properties	

Material Safety Data Sheet

REPORT NO.: LCS190610174A20

Form	Solid
Color	Silvery white
Odour	Not Available
pH	Not Available
Melting point/freezing point	Not Available
Boiling Point and Boiling range	Not Available
Flash Point	Not Available
Upper/lower flammability or explosive limits	Not Available
Vapor Pressure	Not Available
Vapor Density	Not Available
Relative density	Not Available
Solubility in Water	Not Available
Auto-ignition temperature	Not Available
Decomposition temperature	Not Available
Evaporation rate	Not Available
Flammability (sol, gas)	Not Available
Viscosity	Not Available
Section 10- Stability and reactivity	
Stability	The product is stable under conditions described Section 7
Conditions to Avoid	Heat above 70°C or incinerate. Deform, Meltize, Crush, Disassemble, Overcharge, Short circuit. Expose over a long period to humid conditions.
Incompatible Materials	Oxidizing agents, acid, base.
Hazardous Decomposition Products	Carbon monoxide, carbon dioxide, lithium oxide fumes.
Possibility of Hazardous Reaction	Not Available
Section 11 – Toxicological Information	
Irritation	Risk of irritation occurs only if the cell is mechanically, thermally or electrically abused to the point of compromising the enclosure. If this occurs, irritation to the skin, eyes and respiratory tract may occur.

Material Safety Data Sheet

REPORT NO.: LCS19061017AA5D

Sensitization	Not Available	
Neurological Effects	Not Available	
Toxicogenicity	Not Available	
Reproductive Toxicity	Not Available	
Mutagenicity (Genetic Effects)	Not Available	
Toxicologically Synergistic Materials	Not Available	
Section 12- Ecological Information		
Ecological Toxicity	Not Available	
Mobility in soil	Not Available	
Persistence and Degradability	Not Available	
Bioaccumulation potential	Not Available	
Other Adverse Effects	Not Available	
Section 13- Disposal Considerations		
Product disposal recommendation	Observe local, state and federal laws and regulations.	
Uncleaned packaging recommendation	Disposal must be made according to official regulations.	
Section 14 – Transport Information		
Label for conveyance	Lithium Battery Label	
UN Number	UN 3171	
Transport hazard class(es)	9	
Packing group	—	
Main pollutant	No	
UN Proper shipping name	Battery-powered vehicle or Battery-powered equipment	
ICAO/IATA	Can be shipped by air in accordance with International Civil Aviation Organization (ICAO), TI or International Air Transport Association (IATA) DGR Packing Instructions 952 appropriately.	DGR 61 ^B
IMDG CODE	International Maritime Dangerous Goods Code under Special Provision 185	IMDG CODE (Amr. 39-18)
ADR	European Agreement concerning the International Carriage of Dangerous Goods by Road under Special Provision 185	ADR 2019

Material Safety Data Sheet

REPORT NO.: LCG100E101746SD

RID	Regulations concerning the International Carriage of Dangerous Goods by Rail under Special Provision 188	RID 2013
May be shipped without being declared as Class 3 dangerous goods, when meet to the requirements above.		
The dangerous goods regulations require that each battery design be subject to tests contained in Section 38.3 of the UN Manual of Tests and Criteria prior to being offered for transport.		
Section 15- Regulatory information		
<p>Law information</p> <p>(Dangerous Goods Regulations) (Recommendation on the Transport of Dangerous Goods Model Regulations) (International Maritime Dangerous Goods) (Technical Instructions for the Safe Transport of Dangerous Goods) (Classification and code of dangerous Goods) (Consumer Product Safety Act) (CPSA) (Federal Environmental Pollution Control Act) (FEPCA) (Resource Conservation and Recovery Act) (RCRA) (European Agreement concerning the International Carriage of Dangerous) (Regulations concerning the International Carriage of Dangerous)</p> <p>In according with all Federal, State and local laws.</p>		
Section 16- Other Information		
<p>The information above is believed to be accurate and represents the best information currently available to us. However, concrete makes no warranty of merchantability or any other warranty, express or implied, with respect to such information, and we assume no liability resulting from its use. Users should make their own investigations to determine the suitability of the information for their particular purposes. Although reasonable precautions have been taken in the preparation of the data contained herein, it is offered solely for your information, consideration and investigation. This material safety data sheet provides guidelines for the safe handling and use of this product; it does not and cannot advise on all possible situations, therefore, your specific use of this product should be evaluated to determine if additional precautions are required.</p>		

-- End of Report --

Skyline

your guide in travel mobility