# LIFEVAC<sup>®</sup>

Welcome to the LifeVac family

**LIFEVAC INFORMATION AND INSTRUCTION BOOKLET**

*GIVING PEACE OF MIND WHILST SAVING LIVES*

[www.lifevac.uk](http://www.lifevac.uk)

**INFORMATION PAGE**

**CONTENTS**

Upon receiving your LifeVac unit check to ensure you have received what you ordered. The standard contents for each type of LifeVac configuration are as follows. You can purchase additional masks of the size required. A practice mask is provided with instructions for its use so that you may familiarize yourself with LifeVac prior to use in an emergency situation.

Upon receiving the LifeVac unit, visually inspect contents for any defects or cracks.

**LV01 - LifeVac Standard Kit - contents:** 1 x LifeVac device, x3 Masks, sized identified by coloured label, 1 x Instructions for use booklet, 1 x BLS protocol leaflet, 1 x Feedback card.

**LV07 - LifeVac EMS Kit - contents:** 1 x LifeVac device, 1 x Extra-large adult mask identified with red label, 1 x Large adult mask identified with orange label, 2 x Medium adult mask identified with green label, 1 x Small adult mask identified with yellow label, 1 x Paediatric mask identified with white label, 1 x Instructions for use booklet, 1 x BLS protocol leaflet, 1 x Feedback card.

**LV08 - LifeVac Wall mounted Kit - contents:** 1 x LifeVac device, 1 x Large adult mask or 1 x paediatric mask, 2 x Medium adult mask, 1 x Small adult mask, 1 x Instructions for use booklet, 1 x BLS protocol leaflet, 1 x Feedback card, 1 x A3 protocol poster, 1 x Wall mounting fixings.

Additional accessories are available for your LifeVac Unit.

These are as follows:

X-Large, Large, Medium, Small and Paediatric Masks, An empty wall mountable plastic case, Clear plastic EMS bag, LifeVac Travel bag. Please visit our website [www.lifevac.uk](http://www.lifevac.uk) for more details.

**LIFEVAC ONLY TO BE USED ON CHILDREN BELOW 10KG WHEN PEDIATRIC MASK FITTED, IDENTIFIED BY A WHITE LABEL**

**INDICATIONS FOR USE**

The LifeVac unit is a single use lifesaving apparatus developed for resuscitating a choking victim when Standard Basic Life Support (BLS) has been followed without success or cannot be applied. LifeVac is simple to use, available to the general public, and may be the last resort to save a life of a choking victim when all other avenues have failed.

**GENERAL CARE**

Store the LifeVac unit at room temperature. Do not store in direct sunlight. Do not store in garages or attics as extreme heat or cold could affect the integrity of the unit. An ideal place to store the unit would be a kitchen cabinet or an accessible place in case of an emergency. Once you feel comfortable using the LifeVac, dispose of the practice mask, repack your unit, masks and literature back into the storage bag as it was received. Make sure no masks are attached to the unit. Inspect masks periodically and strongly consider replacing every two years. To order replacement masks go to [www.lifevac.uk](http://www.lifevac.uk) LifeVac is not made with natural rubber latex.

**RETURN POLICY**

Check the unit for any crack or defects. If any defects are found contact [www.lifevac.uk](http://www.lifevac.uk) or [info@lifevac.uk](mailto:info@lifevac.uk) immediately for either an exchange or return for credit. All defects must be reported within 5 days of invoice date.

**CONTRAINDICATIONS FOR USE**

LifeVac should be used when standard BLS protocol has been followed without success or cannot be applied. LifeVac should ONLY be used for the resuscitation of a choking victim. LifeVac should not be used if the victim is not choking and is breathing on their own. LifeVac is for use on humans only. It should only be used on the face. LifeVac does not contain latex. LifeVac should not be used on people under 10 kilos without using a paediatric mask. Patients with an endotracheal tube should not use LifeVac. Masks are provided and the manufacturer's guidelines apply. Manufacturers suggest a 24-month shelf life for masks.

**RISKS**

The LifeVac unit is not guaranteed to resuscitate a choking victim. A mask is provided with instructions for orientation so you may familiarize yourself with the unit before an emergency situation. Use at your own risk. LifeVac is not responsible for injury and/or death as a result of use. Risks may include bruising of the face from application of the mask and coughing after the removal of the object. In order to reduce these risks LifeVac must be used in accordance with the instructions provided. Visit our website [www.lifevac.uk](http://www.lifevac.uk) for further information and instructional videos.

⚠ Caution read all documents before use    ⚠ Product is latex free  
 ♻ Re-use is not allowed    ☀ Keep out of direct sunlight    🧤 Handle with care

**Orientation/Testing Procedure**

- Unwrap TEST mask and insert the TEST mask into the base of the LifeVac unit, with a twisting motion, whilst applying pressure. Make sure you check the mask to ensure that it is attached correctly and firmly.
- With one hand the mask, hold it in place firmly and depress the handle with the other hand. You will hear the rush of air come out of the unit.
- When you depress the handle all the way, the handle should be pulled up quickly. You will feel the negative pressure that the LifeVac unit generates.
- Once you are familiar with the LifeVac unit, remove the mask by twisting and pulling it, while holding the base of the LifeVac unit. It should be hard to remove the mask. If so you know you had secured the mask properly. Keep the test mask for further training and label test mask "not to be used in an emergency".
- Repack the LifeVac unit, masks and literature back into the original packaging. Store the LifeVac kit in a safe, dry, accessible place in case of an emergency.

**Instructions for use**

**FOLLOW ALL STANDARD BLS CHOKING PROTOCOL FIRST AND DIAL EMERGENCY SERVICES**

- Unwrap relevant unused mask. Insert mask into the base of the unit with a push and twisting motion, whilst applying pressure. Make sure to check the mask is attached firmly to the LifeVac unit.
- Tilt head back, place top of mask on the bridge of the nose, between the eyes, bottom of mask over the mouth. Lip of mask should be situated between bottom lip and chin. If needed open mouth with thumb if mouth is closed. Mask must be held firmly over nose and mouth with hand at all times to ensure seal is created.
- Holding mask firmly in place with one hand, place your other dominant hand on handle. Hold the handle and then push handle down to compress bellows.
- Once bellows are depressed, pull handle upwards in a firm quick motion, whilst always holding the mask firmly in place with other hand.
- Roll person onto their side and sweep mouth to clear any debris. Also check LifeVac unit for debris, repeat steps up to 5 times if necessary.

**IF NO SPONTANEOUS RESPIRATION IS NOTED, THEN FOLLOW STANDARD BLS CPR PROTOCOL**

**INSTRUCTIONS FOR USE IN THE CASE OF CHOKING**

**If a casualty is a victim of choking**

- 1 Encourage the victim to cough.
- 2 If the cough is not effective give 5 back slaps.
- 3 If obstruction has not been cleared, follow by 5 abdominal thrusts.

Repeat sequence until blockage is removed or victim becomes unconscious

**Once unconscious or if steps 2 and 3 cannot be applied, call emergency services and use LifeVac.**

Place over mouth and nose creating a good seal with correct sized mask    Push in handle/bellows    Pull handle up sharply

**PLACE • PUSH • PULL**

REPEAT UNTIL BLOCKAGE IS REMOVED OR UP TO 5 TIMES

**If blockage is not removed commence CPR**

LifeVac Europe LTD Unit 2,  
 Hill Village Business Park, Barnstaple, EX320RP  
[www.lifevac.uk](http://www.lifevac.uk)

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LifeVac Europe Ltd, Unit 2, Hill Village Business Park
LifeVac Europe Ltd, Horswell Farm