NL – OtoMel® oordruppels

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Product

OtoMel® oordruppels, flacon 10 ml

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OtoMel® ear drops, vial 10 ml

Product description

OtoMel® is a soothing, honey-based lotion that cares for the outer ear canal. OtoMel® contains 100% pure honey. OtoMel® is beneficial for irritated skin in the ear. OtoMel® does not contain any substances known to cause allergic reactions. The preservative is based on potassium sorbate, a safe substance, permitted for natural healthcare products.

Ingredients

Aqua, Mel, Glycerine, Sorbitol, Potassium Sorbate.

Mode of application

- Drip into the external ear canal 1 to 3 times daily; 1 to 3 drops at a time.
- You can also apply OtoMel® by cutting the gauze to size, rolling it up and placing it in the ear. Then drip OtoMel® into the gauze and let it soak in.

Applications

OtoMel® can be applied to care for the ear and is beneficial for irritated skin. OtoMel® is suitable for all ages.

When not to use

- Do not apply if you are hypersensitive to honey or any of the other ingredients.
- Do not apply if you have a perforated eardrum, for example due to ear infection.
- Do not use if the packaging is damaged on receipt of the product.
- Do not use after the expiry date.

Special attention and precautions

- Use for a maximum of 30 consecutive days.
- Consult your doctor if your symptoms do not improve, or if new symptoms appear.
- Keep out of reach of children.

Side effects

- If there are serious adverse effects related to use of this product, report this to the manufacturer and to the competent authorities (of the EU member state) in which the user is located.

Storage conditions

- The expiry date is indicated on the packaging.
- Store between 4°C and 30°C.

Date of last revision of the instructions for use

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Dos Medical B.V.
Genex B.V.