INSTRUCTIONS FOR USE

1	
5	Name <p>Carban® silicone sheets - Elastic and Light</p> <p>Product Description <p>Carban® silicone sheets are a prevent technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone gloves glove and finger), washable and can be used for a wide range of medical purposes.</p> <p>-Carban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.</p></p>
2	Composition <p>-Textile layer: polyamide, elastane <p>-Medical grade silicone</p></p>
6	Properties <p>Carban® silicone sheets ensure that scars become flatter, smoother and thinner and reduce complications (like persistent redness, pulling of the skin, itching, restriction of joints movement, discomfort and cosmetic concerns).</p> <p>The silicone sheet provides semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration means that the damaged skin recovers more quickly and that functional objectives (complaints) are reduced.</p> <p>-Carban® silicone sheets provide UV protection (UPF50) as standard.</p>
3	Indications/applications <p>Carban® silicone sheets are suitable for: <ul style="list-style-type: none">-both old and new scars -all skin types -all ages -treatment and prevention of abnormal scars (hypertrophic and keloid) -treatment of burn scars -treatment of surgical scars -use under clothing and in compression therapy -treatment of persistent functional complaints caused by the scar, such as itching, redness, pulling of the skin, restriction of joint movement, discomfort and cosmetic concerns</p>
4	Instructions for use - Carban® silicone sheets <p><i>(For instructions on how to use the glove and finger, see below)</i></p> <p>For optimum results preferably start applying Carban® silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scars).</p> <p>It is recommended that Carban® silicone therapy be built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to get used to the correct use of the product. For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 23 hours per day. Carban® silicone therapy should be used for a maximum of 23 hours per day (see also section 11).</p> <p>Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.</p> <p>Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar and by cleaning the skin thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding (fig. 2).</p> <p>Remove the protective film from the back of the silicone sheet (fig. 3) and apply Carban® to the scar (fig. 4).</p> <p>If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).</p> <p>After removal of the silicone sheet, the skin may be treated with a hydrating cream.</p>
7	Instructions for use - Carban® silicone sheet - Glove <p>The Carban® silicone sheet glove is used for scarring on the hand or fingers.</p> <p>-To put on the glove, first fold the cuff outward. This prevents the cuff from moving over the fingers when pulling it on and helps the glove slip easily onto the hand.</p> <p>If necessary, the hand (and especially the fingers) can be rubbed with neutral water-based lotion/cream in order to allow the glove to slide on easily. This will ensure that the hand glides easily into the glove.</p> <p>Pull the finger sections of the glove over the fingers, using twisting movements.</p> <p>After donning, turn back the cuff of the glove.</p>
8	Instructions for use - Carban® silicone sheet - Finger <p>The Carban® silicone sheet finger is used for scarring on the finger.</p> <p>First determine the length of the silicone sheet that it is needed. To do this, lay the finger on the sheet. There are several colored markings on the side of the sheet. If the sheet is too long, the silicone sheet can be cut at the bottom edge of the colored marking.</p> <p>Store at temperatures between 5 and 25°C.</p>

9	Duration of use <p>-Each pack of Carban® contains one or more silicone sheets.</p> <p>-If used correctly and cared for properly (according to the instructions for use), each Carban® Silicone® Elastic is sufficient for approximately 4 to 8 weeks of treatment. While each pack of Carban® Light is sufficient for approximately 12 to 24 weeks of treatment.</p> <p>-In the event of long-term therapy, several packs may therefore be necessary.</p>
10	Special attention and precautions <ul style="list-style-type: none">-Keep out of reach of children. -Carban® silicone sheets are only suitable for external use. -Use Carban® silicone sheets only on healed, closed skin (after the skin has completely healed). -Never dry Carban® silicone sheets in the sun or near a heat source. -Temporarily remove the Carban® silicone sheet when exercising or showering. Respiration of the skin with water may cause the Carban® silicone sheet to detach. -Do not use if you are hypersensitive to any of the ingredients. -Do not use if the packaging is damaged when you receive the product.
11	Side effects <p>-In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin is no longer irritated and then restart treatment.</p> <p>-Start the treatment again and let the skin slowly get used to the Carban® silicone sheet (applying for an additional hour every day). -Skin problems persist, consult your physician. -In the event of serious adverse reactions to the product, these should be reported to the manufacturer and to the competent authorities (of the Member State) where the user is located.</p>

12	Storage <ul style="list-style-type: none">-The expiry date is stated on the packaging. -Carban® silicone sheets can be stored for 3 years in its unopened original packaging. -Store at temperatures between 5 and 25°C.
13	Date of latest revision of instructions for use <p>January 2021</p>

14	Carban® Silicone Cleaning Soap <p>Carban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.</p>
-----------	--

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EN

44	Name <p>Carban® silicone sheets - Elastic and Light</p> <p>Product Description <p>Carban® silicone sheets are a prevent technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone gloves glove and finger), washable and can be used for a wide range of medical purposes.</p> <p>-Carban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.</p></p>
45	Composition <p>-Textile layer: polyamide, elastane <p>-Medical grade silicone</p></p>
46	Properties <p>Carban® silicone sheets ensure that scars become flatter, smoother and thinner and reduce complications (like persistent redness, pulling of the skin, itching, restriction of joints movement, discomfort and cosmetic concerns).</p> <p>The silicone sheet provides semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration means that the damaged skin recovers more quickly and that functional objectives (complaints) are reduced.</p> <p>-Carban® silicone sheets provide UV protection (UPF50) as standard.</p>

47	Indications/applications <p>Carban® silicone sheets are suitable for: <ul style="list-style-type: none">-both old and new scars -all skin types -all ages -treatment and prevention of abnormal scars (hypertrophic and keloid) -treatment of burn scars -treatment of surgical scars -use under clothing and in compression therapy -treatment of persistent functional complaints caused by the scar, such as itching, redness, pulling of the skin, restriction of joint movement, discomfort and cosmetic concerns</p>
48	Instructions for use - Carban® silicone sheets <p><i>(For instructions on how to use the glove and finger, see below)</i></p> <p>For optimum results preferably start applying Carban® silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scars).</p> <p>It is recommended that Carban® silicone therapy be built up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to get used to the correct use of the product. For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 23 hours per day. Carban® silicone therapy should be used for a maximum of 23 hours per day (see also section 11).</p> <p>Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.</p> <p>Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar and by cleaning the skin thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding (fig. 2).</p> <p>Remove the protective film from the back of the silicone sheet (fig. 3) and apply Carban® to the scar (fig. 4).</p> <p>If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).</p> <p>After removal of the silicone sheet, the skin may be treated with a hydrating cream.</p>

49	Instructions for use - Carban® silicone sheet - Glove <p>The Carban® silicone sheet glove is used for scarring on the hand or fingers.</p> <p>-To put on the glove, first fold the cuff outward. This prevents the cuff from moving over the fingers when pulling it on and helps the glove slip easily onto the hand.</p> <p>If necessary, the hand (and especially the fingers) can be rubbed with neutral water-based lotion/cream in order to allow the glove to slide on easily. This will ensure that the hand glides easily into the glove.</p> <p>Pull the finger sections of the glove over the fingers, using twisting movements.</p> <p>After donning, turn back the cuff of the glove.</p>
50	Instructions for use - Carban® silicone sheet - Finger <p>The Carban® silicone sheet finger is used for scarring on the finger.</p> <p>First determine the length of the silicone sheet that it is needed. To do this, lay the finger on the sheet. There are several colored markings on the side of the sheet. If the sheet is too long, the silicone sheet can be cut at the bottom edge of the colored marking.</p> <p>Store at temperatures between 5 and 25°C.</p>

51	Duration of use <p>-Each pack of Carban® contains one or more silicone sheets.</p> <p>-If used correctly and cared for properly (according to the instructions for use), each Carban® Silicone® Elastic is sufficient for approximately 4 to 8 weeks of treatment. While each pack of Carban® Light is sufficient for approximately 12 to 24 weeks of treatment.</p> <p>-In the event of long-term therapy, several packs may therefore be necessary.</p>
52	Special attention and precautions <ul style="list-style-type: none">-Keep out of reach of children. -Carban® silicone sheets are only suitable for external use. -Use Carban® silicone sheets only on healed, closed skin (after the skin has completely healed). -Never dry Carban® silicone sheets in the sun or near a heat source. -Temporarily remove the Carban® silicone sheet when exercising or showering. Respiration of the skin with water may cause the Carban® silicone sheet to detach. -Do not use if you are hypersensitive to any of the ingredients. -Do not use if the packaging is damaged when you receive the product.
53	Side effects <p>-In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin is no longer irritated and then restart treatment.</p> <p>-Start the treatment again and let the skin slowly get used to the Carban® silicone sheet (applying for an additional hour every day). -Skin problems persist, consult your physician. -In the event of serious adverse reactions to the product, these should be reported to the manufacturer and to the competent authorities (of the Member State) where the user is located.</p>

54	Storage <ul style="list-style-type: none">-The expiry date is stated on the packaging. -Carban® silicone sheets can be stored for 3 years in its unopened original packaging. -Store at temperatures between 5 and 25°C.
55	Date of latest revision of instructions for use <p>January 2021</p>

56	Carban® Silicone Cleaning Soap <p>Carban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.</p>
-----------	--

57	Carban® Silicone Cleaning Soap <p>Carban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.</p>
-----------	--

58	Carban® Silicone Cleaning Soap <p>Carban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.</p>
-----------	--

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