



Download Method



Apple users use the **App Store** to search for **EndscopeTool** to download

Android users use **Google Play** to search for **EndscopeTool** to download



Connection Flow

PRODUCT MANUAL

2 First of all, when you get the product, ensure sufficient power, press switch key to open



Power Button

3 Open phone settings WiFi Function



4 Connect softish-xxx



5 Open the APP click the start



6 If you encounter this prompt, please do not switch WiFi, continue to use



7 Use APP to record and take pictures



Ear Pick Installation

Earpick convex point, align with the notch of the metal tube and insert it down and turn it clockwise to tighten and buckle directly Rotate and tighten



Note: be sure to screw tightly, avoid drop in use.

Product Accessories

1 Earpick into the ear canal to take out the dirt in the ear



2 Micro charging cable*1 charge the product extremely

