C CARE

TRA34

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Fabrikant:

Foshan Oscar Medical Instrument Co., Ltd, No2 - Workshop C,
Nanhai National Eco-Industrial Demonstration Park, Danzao
Town, Nahai District, Foshan City, Provincie Guangdong, China

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C CARE

INSTRUCTION MANUAL

We congratulate you on purchasing our C-CARE lightweight aluminum Walker TRA34 and thank you for the confidence you have placed in our brand and products.

We are convinced that with this purchase you have acquired a high quality product that you will enjoy using for many years to come.

Please read these operating instructions carefully before using the product, even if you are familiar with the type of product. It gives you useful tips to get the most out of your product.

If, after reading this manual, you still have questions about this product and its use, please send an e-mail to the address below; we will answer your question immediately.

CRESTA INTERNATIONAL B.V.

ROLBRUGWEG 4

NL-1332 AS ALMERE

info@cresta-international.com

www.cresta-care.com

Tel ; + 31 36 3030063

C CARE

PLEASE READ THIS MANUAL CAREFULLY BEFORE USING THE PRODUCT, EVEN IF YOU ARE FAMILIAR WITH THIS TYPE OF PRODUCT.

Follow the instructions below to assemble the walker. The maximum loadable weight of this walker is 136 kilos. Carefully remove all packaging. Avoid using knives or other sharp objects as this may damage the surface of the product.

Check the product for visible damage. If you see damage or suspect a defect, do not use your product, but contact your supplier for assistance.

INTENDED USE

Your walker is designed to help you walk and carry small groceries in the included shopping bag. The shopping bag is specially designed to fit your walker. Only use the bag that came with your walker. Do not use or combine bags and/or baskets from other manufacturers.

When you are not using the walker, you can easily fold it and store it until you need it again. This walker is designed to be used both inside and outside your home.

CHARACTERISTICS

- User-friendly loop brakes with lock
- Easy brake cable adjustment
- Soft plastic solid wheels
- Swiveling front wheels
- Folding locking mechanism
- Height adjustable

C CARE

SPECIFICATIONS

Length	:	70 cm
Width	:	64 cm
Handle height	:	85,5 - 95, 5 cm
Seat W x D	:	46 x24,5 cm
Seat height	:	54,5 cm
Front wheel	:	20 cm
Rear Wheel	:	20 cm

ASSEMBLY INSTRUCTIONS

Carefully remove all packaging, including the cardboard tubes covering the knurled tube part of the brake unit. Check the product thoroughly for damage or obvious mistakes.

If you suspect that the device is not working properly, do not use it, but contact your supplier for service immediately.

Components of the Walker

C CARE

1. Handgrip
2. Break lever
3. Backrest
4. Height adjustable handles
5. Height adjusting button
6. Seat
7. Shopping bag
8. Front wheel
9. Cane holder
10. Rear wheel

Assembling your walker

Step 1

Attach the handles by pressing the locking pin on the handle base and sliding it into the holder on the main frame. Repeat on the other side. See image on the left

When attaching the handle, make sure that the handle faces the rear wheels. The ergonomic fin on the handle should face inward

Adjust the height level of the handles to suit the user. Press the locking pin on the main frame and pull the handle up or down. Release the buttons as the lever moves. This automatically locks the next possible height. Repeat this process until the desired height is reached. Make sure both handles are locked at the same level - see picture at Step 3

C CARE

Step 2

Install the rear wheel into the holder on the back of the frame while pressing in the locking pin. The locking pin will pop up when the wheel is securely in place. Repeat step 2 to attach the second rear wheel. See image left.

Step 3

Install the front wheel into the holder on the front of the frame. The red button pops up when the wheel is firmly in place. Repeat step 1 to attach the second front wheel. See image left

Step 4

Slide the canvas tabs of the shopping bag onto the hooks on the sides of the seat. Use the longer 'Velcro' canvas straps to attach the bag to the uprights of the main frame, see image left

Step 5

Attach the cane holder to the frame. The cane holder base is located at the bottom left of the frame. See image left.

C CARE

Step 6

Unfold the walker by pressing down on both sides of the seat frame. See image left.

Step 7

To unfold the walker, pull the seat up via the black handle on the seat plate. The walker can now be fully unfolded. See image left

Step 8

IMPORTANT

Before use, make sure that all locking pins are securely fastened as shown in the picture left

C CARE

OPERATING THE BRAKES

To stop, gently squeeze both brake handles. This will cause your walker to slow down and stop. See image on the left

Releasing the handles will release the braking function.

The brakes of your walker have a parking position. To use the parking position, push both brakes down at the same time. You will feel a very slight click on the brakes. Your brakes are now in the parking position. See image on the left

To unlock, pull the brakes back up, you will hear a slight click when the park position is disengaged.

BRAKE ADJUSTMENT

Your walker braking system needs to be adjusted from time to time. To adjust the brakes, loosen nut B and then tighten/unscrew nut A. Tighten nut B again after adjustment. If further adjustment is required, loosen the locking screw and pull the cable down about a millimeter with pliers. Tighten the locking screw again. The brakes can then be further adjusted by loosening nut B and tightening or loosening nut A, see picture left.

If you wish to lend this product or are about to put it back into service, check all parts thoroughly for safety.

This includes:

- Bent or damaged pipes or cracks in the pipe connections.
- Loose or worn horseshoe clips
- Elongated adjustment holes.
- Loose handles.
- Loose or worn wheels and wheel attachments - if applicable.
- Smooth, worn or split end sleeves / rubber ends.

Replace any worn, missing, or damaged part if necessary. When in doubt, do not lend or use, but contact your dealer for service support.

CARE - MAINTENANCE

1. Only use your walker for its intended purpose, i.e. as a walking aid. Your walker is intended to be used only on standard sidewalks or at home. Do not use the walker on rough terrain or surfaces with potholes. Using your walker

C CARE

on these surfaces can seriously compromise your safety and can irreparably damage your walker.

2. Keep your walker clean by wiping it with a damp cloth with a mild detergent. Do not under any circumstances use an abrasive or solvent based cleaner as this will seriously damage the walker and void your warranty.
3. Wipe all surfaces with a dry cloth and use a maintenance product such as WD40 on all metal joints.
4. Check that all wheels and folding mechanism are correctly working and that all wheels can move freely. Pay particular attention to the front wheels; make sure they rotate freely and unimpeded and that they are securely attached to the frame.
5. Stop using your walker if one of the wheels is difficult to turn or not properly secured to the frame or if you suspect damage or other malfunction.
6. Do not install any parts or accessories on your walker that are manufactured by another company. Doing so will void your warranty and may seriously compromise your safety.
7. We recommend that you visually check your walker every day. However, we recommend that you service your walker every three months or more frequently as needed if it is used by different users.

CLEANING

Clean your walker with a non-abrasive detergent or a mild detergent with a soft cloth. Abrasive cleaners and/or scouring pads can cause serious irreparable damage to the product and should not be used. Always make sure to dry the equipment after cleaning.

Regularly carry out a safety check of the product or if you have any concerns.

C CARE

WARRANTY

CRESTA INTERNATIONAL B.V. warrants this product to be free from manufacturing 'defects and material errors for a period of 36 months from the date of purchase. In the event of a defect within this period, CRESTA will at its discretion repair or replace the device free of charge for the same or an equivalent device. This warranty does not apply in the event of normal wear and tear or if, in the opinion of CRESTA INTERNATIONAL B.V. the defect was caused by improper use, falling or impact, water damage, leaking batteries, if applicable, and the like. The warranty is only applicable if the warranty claim can be demonstrated with the purchase receipt.

Do not try to open the product and repair it yourself. Your warranty claims will lapse

Requirements:

If a repair is necessary, you can return the product to the store where you bought it. You can also send the product directly to us. Send a message in advance with your complaint description and do not forget to state your full address details - not a PO box - when sending so that we can return the product to you after repair or replacement. If you send the product to us, you must frank it sufficiently. After repair or exchange you will receive your product from us delivered free.

Service address:

Cresta International B.V
Rolbrugweg 4 | NL-1332 AS ALMERE
info@cresta-international.com
www.cresta-international.com

Printing errors and modifications reserved.

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