RING Folding Pessaries

For uterine prolapse (Stage I and Stage II) RxOnly

AVAILABLE MODELS

RING without SUPPORT	RING with SUPPORT
REF MXPER (Pessary Only)	REF MXPRS (Pessary Only)
MXPKR (Pessary, TRIMO-SAN™, Jel Jector)	MXPKRS (Pessary, TRIMO-SAN™, Jel Jector)

RING with KNOB without Support	RING with KNOB with SUPPORT
REF MXPKR (Pessary Only)	REF MXPKSK (Pessary Only)
MXPKR (Pessary, TRIMO-SAN™, Jel Jector)	MXPKSK (Pessary, TRIMO-SAN™, Jel Jector)

Single-Patient Use **€ 0086**

Wash pessary with mild soap and thoroughly rinse prior to initial use.

DESCRIPTION

The Milex® RING Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

WARNINGS

- Sizes 11 through 13 contain a wire coil - remove before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.

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PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

- To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner.
- It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge
- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out

For medical emergencies and for all medically related advice, consult your healthcare professional.

FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

INSTRUCTIONS

- Wear dry gloves. When necessary, lubricate only the entering end of the pessary with TRIMO-SAN. Hold as illustrated in Figure 1.
- The pessary is folded along the axis of the bigger outer holes (or the inner notches for the RING without SUPPORT) by bringing the small round holes together. The KNOB will be at the top of the arch (see Figure 2). The arch formed points downward as shown with the RING pessary.

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