BASAL THERMOMETER TYPE DMT-3032 Instructions and warranty

Before use, read the instructions and warnings given in this handbook carefully and keep the handbook in a safe place.

Basal thermometer with a large LCD display for easy reading. It measures body temperature in 60-120 seconds. Fast and accurate fitted with a sound signal when the temperature reading has been taken. Stores the last read temperature, to evaluate any variations. Extremely easy to use, it is kept in a practical case to avoid risk of breaking. With flexible and automatic switching-off function.

KEY TO SYMBOLS	
	Caution! Read the user instructions carefully
	Symbol for "Type BF applied parts"
	Manufacturer
	Storage and Transportation Temperature Limit: -20°C - 55°C (-4°F - 131°F)
	IP27: Degree of protection of electrical equipment covers, where the first figure indicates the degree of protection against penetration by solid foreign bodies (from 0 to 6) and the second figure the degree of protection against penetration by liquids (from 0 to 8).

GENERAL WARNINGS

1) Self-measuring means control, not diagnosis and treatment, any unusual readings must be discussed with your doctor. Under no circumstances must you change the medicine doses that have been prescribed by your doctor.
2) Do not squeeze, bend, drop, bite or break the thermometer.
3) Keep it out of the reach of children.
4) Do not expose to sunrays or direct sources of heat. Keep in a dry clean place.
5) Do not immerse the thermometer in water or other liquids, only the probe is waterproof.
6) Do not take the measurement when the thermometer is wet because the results could be inaccurate.
7) This thermometer reads only the armpit, rectal or oral temperature. Do not use it for reading auricular temperatures.
8) Do not drink hot or cold drinks, do not smoke, do not take a shower before measuring body temperature.
9) Do not use the thermometer orally after it was used rectally without disinfecting it first.
10) For children under two years of age, do not use the thermometer orally.
11) The device may give inaccurate readings if its used in temperatures or humidity levels outside the limits indicated in the "Technical Specifications" paragraph.
12) Do not use near strong magnetic fields and therefore keep away from radio or mobile phone systems (for more information on interferences, see the paragraph "Electromagnetic compatibility information").
13) Choking Hazard: Thermometer cap and battery may be fatal if swallowed. Do not allow children to use this device without parental supervision.
14) Do not place thermometer battery near extreme heat as it may explode.

SPECIFICATIONS

- Type: Basal Thermometer (Not Predictive)
- Measure Range: 32.00°C-42.99°C (90.08°F-109.90°F) (°C /°Fchosen by manufacturer)
- Accuracy: ±0.10°C (±0.20°F) during 35.50°C-42.00°C (95.90°F-107.60°F) at 18°C-28°C (64.40°F-82.40°F) ambient operating range ±0.20°C (±0.40°F) for other measuring and ambient operating range
- Operating mode: Direct Mode
- Display: Liquid crystal display, 4 1/2 digits Memory: For storing the last measured value
- Battery: One 1.5 V DC. button battery (size LR41or SR41, UCC 392)
- Battery life: Approx. 200hours of continuous operation or 1 year with 3 measurements per day
- Dimension: 13.8cm x 2.2cm x 1.2cm (L x W x H)
- Weight: Approx. 12 grams including battery
- Expected service life: Three years
- Ambient operating range: Temperature: 5°C-40°C (41°F-104°F) Relative humidity: 15%-95%RH Atmospheric Pressure: >700hPa - 1060hPa
- Storage and transportation condition: Temperature: -20°C-55°C (-4°F-131°F) Relative humidity: 15%-95%RH Atmospheric Pressure : 700hPa - 1060hPa
- Contraindications: None

DIRECTIONS

1. Press the On/Off Button next to LCD display. A tone will sound as the screen shows **188.88**, followed by last recorded temperature. After showing the self-test temperature, the thermometer is now in the testing mode.
2. Position thermometer in desired location (mouth, rectum, or armpit).
a) Oral Use: Place thermometer under tongue as indicated by “/” position shown in Figure. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air. Normal temperature between 35.70°C and 37.30°C (96.26°F and 99.14°F).
b) Rectal Use: Lubricate silver probe tip with petroleum jelly for easy insertion. Gently insert sensor approximately 1cm (less than 1/2”) into rectum. Normal temperature between35.20°Cand 37.70°C (97.16°F and 99.86°F).
c) Armpit Use: Wipe armpit dry. Place probe in armpit and keep arm close to the body side.
From a medical viewpoint, this method will always provide inaccurate readings, and should not be used if precise measurements are required.
Normal temperature between 35.20°C and 36.70°C (95.36°F and 98.06°F).
3. The degree sign flashes throughout the testing process. When flashing stops an alarm will beep for approximately 10 seconds. The measured reading will appear on the LCD simultaneously. The minimum measurement time until the signaling tone (beep) must be maintained without exception. The measurement continues even after the buzzer notification.

***Note:** Normally the buzzes are "Bi-Bi-Bi- Bi-"; Alarm beeps more rapidly when temperature reaches 37.80°C (100.04°F) or higher, and the buzzes are" Bi-Bi-Bi----- Bi-Bi-Bi----- Bi-Bi-Bi"

4. To prolong battery life, press the On/Off Buton to turn unit off after testing is complete. If no action is taken, the unit will automatically shut off after around 10 minutes.

INSERTING/REPLACING THE BATTERIES

1) Pull the battery cover in the direction shown on the arrow.
2) Carefully extract the battery compartment to the right to ease the battery removal. Use a pointed object to remove the old batteries.
3) Place the new batteries with the positive pole (+) pointed upwards and the negative pole pointed downwards.
4) Gently push the battery compartment into its seat and re-assemble the cover. Should the display show the symbol  , it means that they have to be replaced.
5) Remove the used batteries as described under points 1-2 and dispose of them as special waste at the collection points for recycling. For further information concerning the disposal of exhausted batteries, refer to the store where the device containing the batteries was purchased, the Municipality or the local waste disposal service. Caution: For safety reasons, it is recommended never to allow children younger than 12 years to remove the batteries.

TROUBLESHOOTING		
Error message	Problem	Solution
- - -	Temperature taken is lower than32.00°C (90.08°F)	Turn off, wait one minute and take a new temperature via close contact and sufficient rest.
 	Temperature taken is higher than 42.99°C (109.90°F)	Turn off, wait one minute and take a new temperature via close contact and sufficient rest.
	The system is not functioning properly.	Unload the battery, wait for 1 minute and repower it. If the message reappears, contact the retailer for service.
	Dead battery; Battery icon is flashing, can't be measurable.	Suggest to replace the battery.

MAINTENANCE

Clean the thermometer probe using a cloth moistened with alcohol or with any type of topic disinfectant before and/or after each use. Any other type of solution can damage the device. The thermometer body has to be cleaned using a dry cloth. Do not immerse the thermometer in water or other liquids. Never use chemical or abrasive products. The probe is the most fragile part of the thermometer. Pay utmost care during its cleaning. Do not use alcohol to clean the display. Preserve the thermometer in a dry and clean place, away from direct sunlight. Do not sterilize or boil the thermometer. Preserve the thermometer carefully in its standard-supplied casing. Do not attempt to disassemble or repair the thermometer. Doing so may result in inaccurate readings. It is recommended the performance should be checked every 2 years.

DISPOSAL PROCEDURE

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (Dir. 2012/19/Eu-WEEE). At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions, if the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 40 m without having to purchase a new similar device. This procedure of separated collection of electric and electronic devices is earned out forecasting a European environmental policy aiming at safeguarding, protecting and improving environmental quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions. To correctly dispose of batteries (**Dir. 2013/56/Eu**) **do not throw them into household waste** but rather dispose of them as special waste at specialised recycling centres. For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

LIMITED WARRANTY

The thermometer is guaranteed for one year from the date of purchase. If the thermometer does not function properly due to defective components or poor workmanship, we will repair or replace it free of charge. All components are covered by this warranty excluding the battery. The warranty does not cover damages to your thermometer due to improper handling. To obtain warranty service, an original or copy of the sales receipt from the original retailer is required.

Question? E-mail support@femometer.com

	Disposal of this product and used batteries should be carried out in accordance with the national regulations for the disposal of electronic products.		JOYTECH HEALTHCARE CO.LTD. No.365, Wuzhou Road, Yuhang Economic Development Zone, Hangzhou city, 31100 Zhejiang,China
Made in China			Shanghai International Holding Corp.GmbH(Europe) Eiffestrasse 80, 20537 Hamburg, Germany
	The product is in compliance with the requirements of MDD 93/42/EEC, " 0197" is the identification number of notify body.		Document No.: JDMT-7204-047 Article No.: 001 Version: Z Date of Issue: 2016.02

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Electromagnetic Compatibility Information

The device satisfies the EMC requirements of the international standard IEC 60601-1-2. The requirements are satisfied under the conditions described in the table below. The device is an electrical medical product and is subject to special precautionary measures with regard to EMC which must be published in the instructions for use. Portable and mobile HF communications equipment can affect the device. Use of the unit in conjunction with non-approved accessories can affect the device negatively and alter the electromagnetic compatibility. The device should not be used directly adjacent to or between other electrical equipment.

Guidance and declaration of manufacturer-electromagnetic emissions			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Emissions test	Compliance	Electromagnetic environment-guidance	
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.	
Harmonic emissions IEC 61000-3-2	N/A		
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A		

Guidance and declaration of manufacturer-electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environm entguidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Electrostatic transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	N/A	
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	N/A	
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	< 5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycle 70% UT (30% dip in UT) for 25 cycle <5% UT (>95% dip in UT) for 5 secretary	N/A	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m; 50Hz or 60Hz	30 A/m; 50Hz or 60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and declaration of manufacturer-electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environmentguidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 Mhz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 Ghz	10 V/m	Recommended seperation distance <p>$d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$ 80 MHz to 800 MHz</p> <p>$d = \left[\frac{7}{E_1} \right] \sqrt{P}$ 800 MHz to 2.7 Ghz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <p></p>
RF Wireless Communication Equipment IEC 61000-4-3	380MHz, 27V /m 450MHz, 28V /m 710MHz,745 MHz,780MHz 9V/m 810MHz,870 MHz,930MHz 28V/m 1720MHz,1845 MHz,1970MHz 28V/m 2450MHz, 28V /m 5240MHz,5500 MHz,5785MHz 9V/m	380MHz, 27V /m 450MHz, 28V /m 710MHz,745 MHz,780MHz 9V/m 810MHz,870 MHz,930MHz 28V/m 1720MHz,1845 MHz,1970MHz 28V/m 2450MHz, 28V /m 5240MHz,5500 MHz,5785MHz 9V/m	

Recommended separation distances between portable and mobile RF communications equipment and the device		
The device is intended for use in an electromagnetic environment in which radiated therefore disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.		
Rated maximum output power of transmitter	Separation distance according to frequency of transmitter m	
W	80 MHz to 800 MHz	800 MHz to 2.7 GHz
	$d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	$d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0.01	0.12	0.23
0.1	0.38	0.73
1	1.2	2.3
10	3.8	7.3
100	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.		
NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.		
NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.		

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