## DISCREET

UP TO 12h NON-STOP ACTION\*  
Cold Sore Treatment  
Acts like a virus shield\*\*

www.compeed.com

Laboratoire HRA Pharma  
200 avenue de Paris  
92320 Châtillon – France  
Made in EU

Rev 04/2021  
(CE) 2019

### EN COMPEED® Discreet Cold Sore Healing Patch with up to 12h non-stop action\*

COMPEED® Cold Sore treatment is the first discreet healing patch for cold sores.

#### Up to 12 hours non-stop action from Day 1 - to heal fast.

Each discreet patch creates an optimal healing environment and stays in place for up to 12 hours\*.

\* Individual experiences may vary. In a clinical study involving 174 subjects 25% of the subjects had a wear time of 12 hours or more.

#### Acts like a virus shield\*\*

\*\*The patch acts like a virus shield to reduce risk of contamination from the cold sore.


#### Hides Cold Sore

Make-up or lipstick can be applied over the patch.

### WHEN TO USE THE PATCH

- For best results, use a patch as soon as you feel first signs of an outbreak such as tingling, itching and burning sensations.
- Use a patch continuously throughout day and night.
- Continue to use until lesion is fully healed.
- Consult your pharmacist or doctor if the cold sore becomes severe and/or an outbreak lasts unusually long.

### HOW TO APPLY THE PATCH

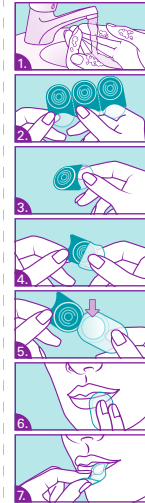- Wash your hands before and after use. Ensure your skin is clean and dry. Do not apply any cream on the lip, as the patch may no longer stick to your lip.
- Hold the 2 products by the flat part and tear off one product along the perforated edge.
- Hold the clear paper applicator at the curved edge.
- Using both hands, carefully peel off the clear paper applicator from the protective teal coloured cover.
- You will see the round cold sore patch on the upper part of the applicator. Take care not to touch the adhesive side of the patch.
- Apply this part of the applicator to the cold sore and using a mirror, gently press down to firmly place the patch on to the cold sore.
- Gently peel back the applicator leaving the patch in place. Make-up or lipstick can be applied over the patch.

### WHEN TO REMOVE THE PATCH

- If the patch stays on, there is no need to remove or change it.
- When it starts to detach from the skin, remove the patch gently and replace it with a new patch.
- To remove the patch, loosen it at the edge. Use both hands to hold loosened edge of patch. Stretch the patch sideways, pulling hands apart as it is peeled away.
- Do not reuse the patch, to avoid infection of the wound and spreading of the virus.

- Dispose the patch after use in the bin with your household rubbish to avoid any contamination.

### ADDITIONAL INFORMATION

- You can eat and drink while wearing the patch.
- For optimal usage, store at room temperature and do not refrigerate.

### CONTRAINDICATIONS

- Do not use on infected wounds other than cold sore.
- Consult the doctor before use if diabetic or immunosuppressed.
- Do not use on children, pregnant and lactating women.
- Do not use another local treatment for cold sore under the patch.

### TAKE EXTRA HYGIENIC CARE COLD SORES ARE INFECTIOUS

The patch covers the lesion and therefore reduces the risk of contamination from the wound. However, it is important to take the following precautions during an outbreak:

- Do not touch the sore, as you may spread the virus to other parts of the body or to other people.
- If you have touched the sore, wash your hands immediately.
- Do not share eating or drinking utensils, lipstick, toothbrush or even a towel with others.
- Always keep the patches in the packaging box.

### FACTS ABOUT COLD SORES

#### WHAT IS A COLD SORE?


A cold sore is an infection which is caused by the Herpes Simplex Virus and the outbreak manifests itself in liquid filled blisters that erupt around the lips, nose or chin.

#### WHAT TRIGGERS AN OUTBREAK?

Various factors such as physical and emotional stress, anxiety, fever, cold or flu, fatigue, exposure to sun, cold weather, menstruation or hormones.

#### HOW LONG DOES AN OUTBREAK LAST?

It usually lasts 7-12 days and goes through 5 stages of symptoms. The COMPEED® Discreet Cold Sore Healing Patch treats all 5 stages of symptoms fast and discreetly.

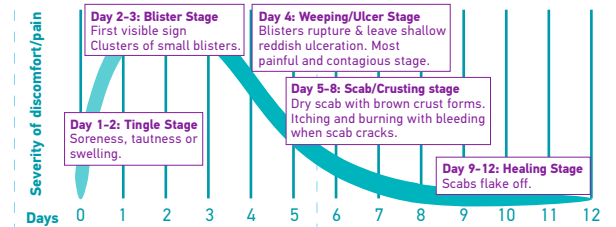\*This graph shows the typical stages of a cold sore outbreak that lasts 12 days. Some outbreaks may be longer or shorter than this.

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