HANDLEIDING/MANUAL MANUEL/ANLEITUNG

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DE (7)

WARNING: Do you already have braces? Then contact your orthodontist before you start using them. The dentist or orthodontist can place the PurelyGoods Anti-snoring Mouthpiece for you and make it fit properly. This will ensure that the material does not bind to any brackets.

CONSULT A DENTIST IN CASE OF:

- A missing tooth or teeth;
- A click in the jaw;
- Ulcers in the mouth or bleeding gums;
- A temporomandibular diagnosis

USE: Before the PurelyGoods snoring mouthpiece can be used, it must first be prepared. With a comprehensive step-by-step plan and smart tips, the PurelyGoods snore mouthpiece will be ready before you know it.

Read the instructions carefully on to avoid problems. Still stuck? Feel free to send us an e-mail at klantenservice@purelygoods.nl. We'll be happy to help!

SUPPLIES:

- PurelyGoods Anti-snoring mouthpiece
- 2 bowls or glasses
- Hot but not boiling water 80/90°C
- Cold water
- Mirror
- Fork

CLEANING AND STORAGE: The PurelyGoods Anti Snoring Mouthpiece can be polished with a normal toothbrush and toothpaste. After washing, rinse and dry the mouthpiece thoroughly. Store the snoring mouthpiece gel in the PurelyGoods storage box and store in a dry room.

WARNINGS:

- Try to avoid unnecessary stretching the material not when the mouthpiece is soft
- Keep the anti-snoring mouthpiece away from heat sources, flames and ambient temperatures above 80 degrees.
- The material of the mouthpiece may stick to nail polish or artificial nails. Be careful with shapes and places.
- Avoid situations where inhaling or swallowing the snoring mouthpiece poses a risk.

DO NOT USE THE SNORING MOUTHPIECE WITH:

- Dentures
- A loose tooth, filling or cavities
- A poor fit
- Epilepsy
- Not suitable for people with respiratory problems or sleep apnea.

DISCONTINUE USE IF:

- The occurrence of acidification, bleeding or other reactions
- The development of problems such as pain in the jaw, teeth, ears, neck or head
- Damage or wear of the mouthpieces

Contact a dentist if you experience pain while using the PurelyGoods Anti snoring mouthpiece.

EN (8)

STEP-BY-STEP PLAN

The PurelyGoods Anti-snoring mouthpiece is more sophisticated than a standard melt and bite snoring mouthpiece. So follow the step-by-step plan below carefully for the ideal fit.

STEP 1 PREPARATION

- Wash your hands and the snoring mouthpiece with soap.
- Adjust the snore mouthpiece so that it is best adjusted for your teeth. By pressing push (both sides) and adjusting the mouthpiece this way.
- Have two glasses/bowls ready: one with hot and one with cold water.

Tip: Check if the water is up to temperature by feeling. Do you burn your finger? Then the water is just too hot.

Note: never let the snore guard melt in boiling water.

STEP 2 MELT THE MOUTHPIECE

- Fully immerse the bracket in hot water and soak it for about 30 seconds, until the bracket is soft.
- Remove the bracket from the glass with a fork and let the bracket cool for 15 seconds.

Note: do not touch the snoring mouthpiece during/after melting.

STEP 3 PLACE IN THE MOUTH

- Check that the bracket is completely soft.
- Grasp the snore mouthpiece by the hard part and place the mouthpiece in the mouth.
- Use a mirror for optimal visibility.
- Make sure that the lower and upper row of front teeth are directly under each other.

STEP 4 SHAPING

- Use the fingers, cheeks and lips to shape the front of the snoring mouthpiece properly to the teeth.
- Use your tongue to shape the inside of the snore guard to the teeth.
- Suck the air out of the mouthpiece by breathing in slowly using the tongue and lips.
- Leave the mouthpiece in the mouth for 1 minute.

STEP 5 REMOVING

- Carefully remove the snoring mouthpiece from the mouth.
- Place the snoring mouthpiece in a bowl/glass with cold water for 2 minutes.
- After 2 minutes, place the snoring mouthpiece back in the mouth.

The PurelyGoods Anti-snoring mouthpiece is ready for use!

Is the PurelyGoods Anti-snoring mouthpiece still uncomfortable? Follow the step-by-step plan again until you find the right fit.

Do you have any questions or need help?
We can be reached at

klantenservice@purelygoods.nl

www.purelygoods.nl

Material: The PurelyGoods Anti-Snoring Mouthguard is made of BPA-free EVA material (Ethylene Vinyl Acetate), which is safe, hypoallergenic, and comfortable for long-term use.

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