

## INSTRUCTIONS FOR USE



### Hernia Truss support belt single with PAD MB/5000

#### **APPLICATION:**

1. Place the brace at the height of the lumbar spine.
2. Apply the pad to the affected area and secure it with the strap on the back of the brace.

#### **WARNING:**

The initial application of the brace is done by a professional. Read the instructions carefully. This product should be given by an Orthopedic Doctor and applied by a professional.

Application should be done with great care to ensure correct fit and efficiency of the brace.

For hygiene reasons, it is proposed to be used by a single patient. In exceptional cases of very sensitive patients, the pads may cause irritation. In this case, please contact your Doctor.

Recommended product use up to 2 years.

#### **WASHING INSTRUCTIONS:**



1. Check the condition of the product before use. If it is worn off, it needs replacement.
2. Hand wash with cold water. Use a soft detergent and remove the pads.
3. Allow to dry on its own. Do not wash it in a washing machine or dry it in a tumble dryer.



**MEDICAL BRACE**

22 Tegeas st.

Peristeri

PC: 12133

Phone: +30 210 5984020

FAX: +30 210 5984043

Email: [mb@mediclabrace.gr](mailto:mb@mediclabrace.gr)

Web: [www.medicalbrace.gr](http://www.medicalbrace.gr)