mr Safe

DIGITAL ALCOHOL TESTER

GB NL

FR D ES

QRG / MrSafe Alcohol Tester K5+ / 11-19 / V02
Copyright©Mr Safe

DECLARATION OF CONFORMITY

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Company /Entreprise:	TE-Group NV
Address: / Adresse:	Kapelsestraat 61, 2950 Kapellen, Belgium
declare that the following equipment: / déclare que le dispositif suivant :	
Product name: / Nom du produit:	mr Safe AT-K5+
Product type: / Type de produit:	Fuel Cell Alcohol tester

conforms with the following safety requirements of the directives 2014/30/EC, 2011/65/EC & 2015/863/EU. Conformity is guaranteed by the CE-symbol. This product has been tested against following standards and specifications, applying versions valid in November 2019. The full text of the EU declaration of conformity is available at the following internet address: http://www.mrsafe.be/doc_at-k5plus/

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
EN55011:2009+A1:2010
EN61326-1:2016
IEC62321

J. Van Os, Product & Research Manager TE-Group NV
Kapellen, November 2019
Authorized Signature

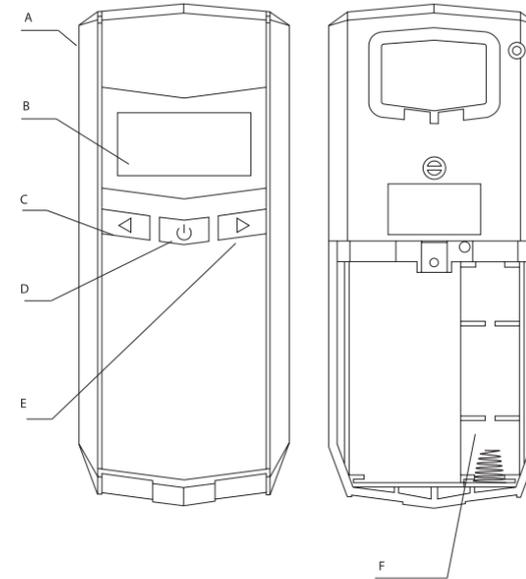ILLUSTRATIONS

BE RESPONSIBLE.
DON'T DRINK
AND DRIVE.

1. IMPORTANT INSTRUCTIONS

GB

! DO NOT DRINK AND DRIVE !
Also never let anyone else drink and drive or operate machine
Always use a designated driver to drive you safely!

! WAIT at least 20 minutes, without drinking or smoking
before testing. Rinse your mouth with water before testing!

! The alcohol level in your blood can keep increasing up to 2 hours after
drinking your last alcoholic beverage. A continuous negative test-result
cannot be guaranteed !

! We advise you to store & use this product at room temperature. Avoid
storing the device inside the car under extreme winter or summer
temperatures !

! DO NOT blow smoke, liquid or food residue into the device !

! This test and your test-result cannot be held as juridical proof in case of
law encounters !

The BAC results are for reference only and the test-results cannot be used as a
tool to determine whether you should operate a vehicle or not !
The manufacturer, distributor and importer disclaims any legal responsibility
for the measured test-result.

2. PARTS & BUTTONS

- A: Mouthpiece Slot
- B: Digital LCD Display
- C: LEFT button (Select / Begin Test / Navigation)
- D: POWER button (OK / Confirm)
- E: RIGHT button (Navigation)
- F: Battery Compartment

3. INSTALLING THE BATTERY

Slightly press down the battery cover and slide it down the device.
Insert 1 X AA Alkaline batteries according to the scheme visible.
Slide the cover back in place until you hear it click.

4. ON / OFF

Press the POWER button (D) to turn on the device.
You will hear a beep and "RDY" will be displayed on the LCD Display (B).
Note: The device will automatically turn off after 30 seconds of inactivity
To turn off the device, press the POWER button (D) again.

5. TESTING YOUR ALCOHOL LEVEL

STEP 1

Fold out the mouthpiece on the upper left side of the unit. Add an additional
mouthpiece (optional) if necessary.

STEP 2

Press the POWER button (D) for 2 seconds to turn on the
device. You will hear a beep and see the countdown timer
will be displayed on the LCD Display (B). The display will show
"RDY" and you will hear a beep. The device is now ready for
use.

STEP 3

Initialize the sensor by pressing the LEFT button (C). On the
LCD Display (B) "C---" will be displayed and you will hear a c
ontinuous beep sound. You can now start blowing into the
mouthpiece.

STEP 4

Keep blowing until you hear 2 'click' sounds to indicate the
sampling process is successful.

STEP 5

The device will now analyze the collected breath sample
and calculate the values. "C---" will be displayed on the LCD
Display (B).

Note: If "FLO" appears on the screen it indicates that your
blowing was less than 5 seconds. Restart the test again, blow
softly and continuously again until you hear the beep-sound

STEP 6

Reading the result:
The reading will be in range from 0.00 to 4.00 ‰ (or 0.000 to 0.400 %BAC or
0.000 to 2.000 mg/l) depending on how the BAC display is set.

RED Background: NOT Safe (positive) – Exceeds alert lever set by user

Note: If you want to take several tests, wait at least 1 minute with normal
results, 3-5 minutes with high results.

6. NAVIGATING THE LAST TEST RESULTS

When turning on the device, the LCD Display (B) will start a countdown sequence.
Press the POWER button (D) during the countdown to verify how many tests
you have taken with the device. This number gives you an indication on when
a calibration is needed. To reset the number, press the LEFT button (C) for 2
seconds.

Press the POWER button (D) again to display the latest test record (indicated
by the number "1" on the screen) and it will be followed by the corresponding
result. To navigate through the latest tests, press the LEFT button (C) or the
RIGHT button (E).

To exit the test memory, press the POWER button (D).

7. ERROR NOTIFICATIONS

Sometimes the display can show an error notification:
RPT: Insufficient or uneven blowing during test detected. Repeat the steps in
chapter 5

SNR: Sensor not ready: this means that the sensor is not ready due to a high
concentration of alcohol residu left on the sensor from the last test. Switch off
the device, wait for a few minutes and repeat the steps in chapter 5.

FLO: Not enough sample collected due to insufficient blowing.
Repeat the steps in chapter 5

H: HIGH: Alcohol concentration is over 4.00‰BAC.

Wait 15-30 minutes before trying again

OUT: This means that the test has been conducted in either too cold or too
hot temperature environment. Please bring the device to a normal room
temperature, wait a few minutes and redo the test by repeating the steps in
chapter 5.

LO: Battery low, please replace the 1 AA Alkaline batteries in your device.

8. CUSTOMER SUPPORT

For all questions and additional information concerning this product, please
contact support@mrsafe.eu.

9. GUARANTEE

Copyright © mr Safe®. mr Safe® is a registered trademark of TE-Group NV. The
mr Safe® brand stands for superior quality products and excellent customer
service. Therefore mr Safe® warrants this product to be free from defects in
materials and workmanship for a period of two (2) years from the original
purchase date of the product.

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Copyright © mr Safe®. Mr Safe® is een gedeponerd handelsmerk van TE-