# Nasal Irrigator

EN

Class I medical device in accordance with CE standard (EU) 2017/745 MDR – BPA Free – Latex Free – 6+ months

## KEEP FOR FUTURE REFERENCE

### Instructions for use / precautions

Before starting, wash your hands and use a saline solution.

- 1) Open the individual packaging and remove the device.
- 2) Draw saline into the syringe, reading the correct scale for the required dose.
- 3) Place the user in a position where the saline enters the nasal cavity and then flows easily out the other nostril or the mouth.
- 4) Holding the syringe in the hand, place the flushing connector into one nasal cavity, tilt the head slightly forward, open the mouth and push the mandrel to flush in the saline. Nasal secretions should come out with the saline through the other nostril.
- 5) Wait until the user has swallowed properly and secretions from the nose have drained away.
- 6) Rinse the product thoroughly after use with hot water and a little dishwasher detergent, then rinsing it thoroughly with clean water. Dispose the product in the waste container.

When flushing, care should be taken to push the plunger with moderate and even pressure. Try to avoid using too much force to prevent damage to the mucous membrane of the nasal cavity. Breathe, when using the product, through the mouth as much as possible during the application process and open them as wide as possible without swallowing water to prevent choking. Avoid using nasal irrigation too frequently.

### WARNING! For your child's safety

This product or the packaging is NOT a toy. Keep all components not in use out of reach of children. Always use this product with adult supervision. Inspect the product before use. Check that the detachable parts are fitted correctly. Throw away at signs of damage or weakness. Store the product in a dry place and protected from light. Report any serious accident to the manufacturer and to the competent authorities for the given country.

The product can be recycled, after use, please should be recycled according to the principle of waste separation to protect the environment.

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Shanghai Kindly Enterprise Development Group Co., Ltd.  
Addr: No.658, Gaochao Road, 201503 Shanghai, P.R.C.

Shanghai International Holding Corp GmbH (Europe)  
Addr: Effestrassse 80, 20537 Hamburg Germany  
Tel: 0049-40-2513175 Fax: 0049-40-255728

Imported by: Jinius, Polderwerker 4, 6661 MP Elst  
The Netherlands

Date of issuance: 01-10-2023  
Date of latest revision: 01-10-2023

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