
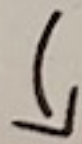


EyeDrop® glasses  Made in China 100% recycled frame

1. Wash hands



2. Fix eyedrop bottle in hole



3. Put on glasses
hold/ keep eye open
if helpful: pull back eyelid

4. Lean backwards and
press bottle to drop in eye


5. If necessary fix bottle for
other side/eye and repeat

Instructions



Information:
info@iconeurope.nl

Manufactured by Wenzhou M&I Eyewear,
No.7 Ji Xian Rd. Wenzhou, China
Authorized Representative:
Icon Eyewear Europe B.V.
Bergen op Zoom, The Netherlands

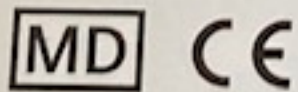

MDR 2017/745

Warning: Not suitable for Minims pipets, only use when
eyedrops can be fixed securely and cannot contact eye directly

Made in China