VULPES GOODS

BABYCARE

**DRAADLOZE BORSTKOLF
MAX**

HANDLEIDING

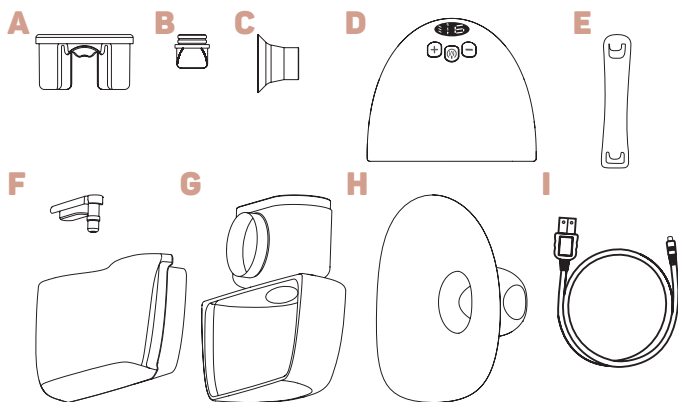
INHOUD

Product informatie	3 - 11
Tips	12
FAQ	13
Schoonmaak & onderhoud	14 - 15
Product parameters	16

CONTENT

Product information	19 - 27
Tips	28
FAQ	29
Cleaning & Maintenance	30 - 31
Product parameters	32

PRODUCT INFORMATIE

	ONDERDELEN	MATERIAAL	HITTEBESTENDIGE TEMPERAATUUR
A	Gasopslag	Siliconen	-20~180 °C
B	Eendenbekventiel	Siliconen	-20~180 °C
C	Borstchildverkleiners	Siliconen	-20~180 °C
D	Motor	ABS/siliconen	/
E	BH - bandje	/	/
F	Silicone plug + Deksel	PP	-20~180 °C
G	Opvangbak	PP	-20~180 °C
H	Borstchild	Siliconen	-20~180 °C
I	Oplaadkabel	/	/

Montage van de borstkolf

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
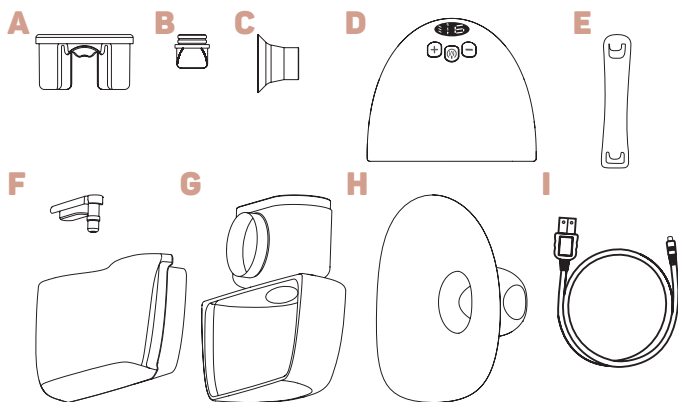
Assembly of the breastpump

When you receive the breastpump, it is already assembled. If you wish to clean the pump further before use, it's useful to know how to properly assemble it afterwards. Correctly assembling the pump and all its components is crucial for optimal use. Follow the steps accurately and ensure that each part is placed correctly.


Step 1: Inserting the silicone plug

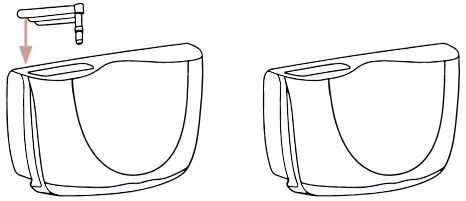Place the silicone plug carefully. This will prevent leakage and make pouring easier.


Step 2: Placement of the valve

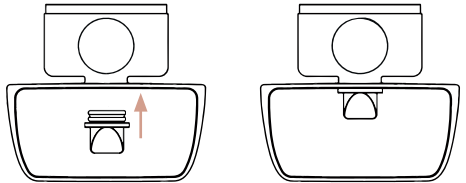The valve is placed in the collection bottle of the breast pump. Make sure to press the valve in completely. This prevents milk from flowing back


Step 3: Attachment of the collection tray

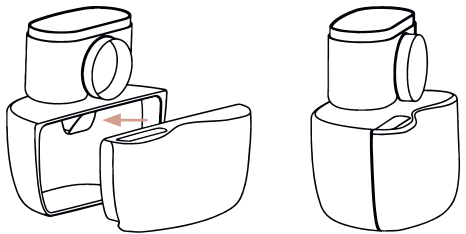Press the collection trays together. Pay attention to the correct alignment to prevent any leakages.


Step 4: Placement of the gas storage

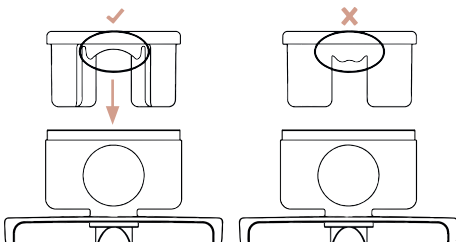Install the gas storage in the holder. Ensure that the correct curvature is at the front. Press the gas storage firmly in place.


Step 5: Placement of the breast shield

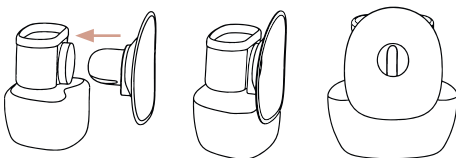The breast shield is placed in the circular holder of the collection tray. Pay attention to ensuring the correct alignment to prevent any leakages.


Step 6: Placement of the engine

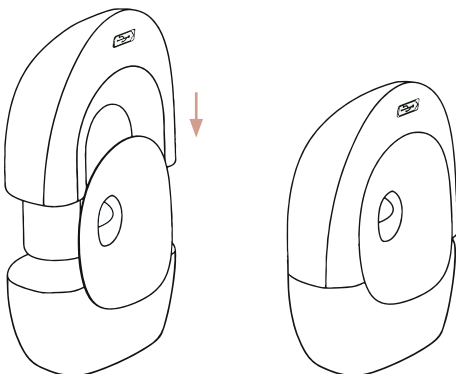Place the engine on the collection tray. Ensure a firm connection to avoid losing suction power.


Step 7: Placement of the funnels

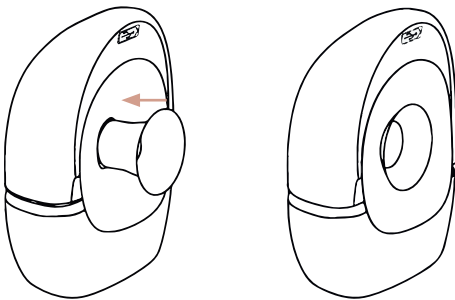Select the appropriate size of funnel and place it inside the breast shield. Press the firmly to prevent any leakage.

Step 8: Charging

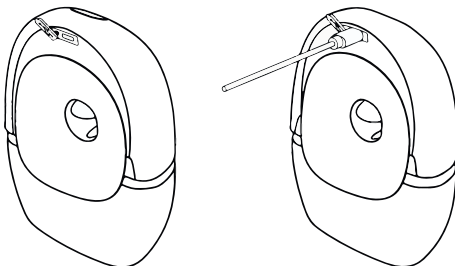
Open the charging port and insert the provided USB-C cable. Make sure to always use the supplied cable for charging.


Step 9: Pouring the milk

Remove the engine and the breast shield from the collection tray.

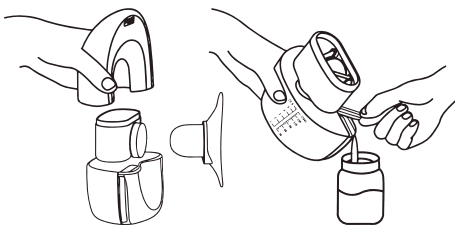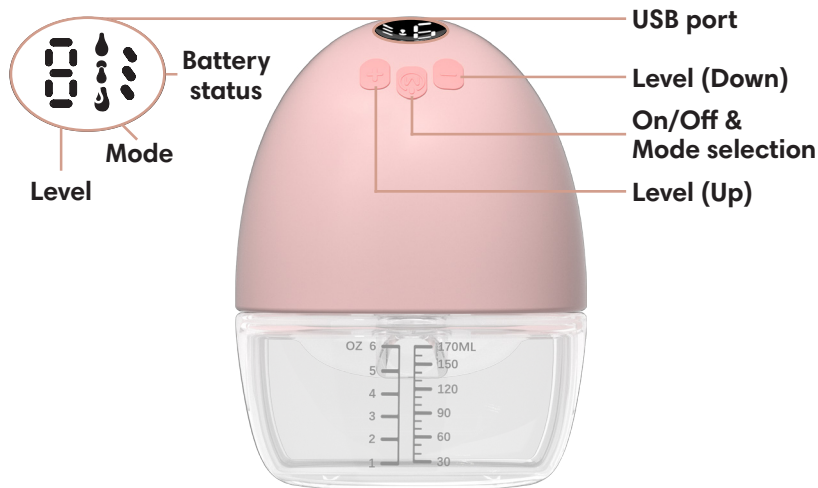Then, remove the silicone plug and pour the milk into the collection tray.

Battery status:

Indicates the remaining battery level and the charging status.

Low battery:	1 visible bar
Medium battery:	2 visible bars
Full battery:	3 visible bars

When the battery needs to be charged, you will see a single bar.
 When the battery is charging, the bars will blink sequentially.
 When the battery is fully charged, all 3 bars will be visible.

(The battery life depends on the modes and levels used.)

Mode:

The display shows the current mode, each mode serving a different function. You can change the mode using the M button.

Level:

The display shows the current level, and each level increases the suction strength. You can adjust the level using the + and - buttons.

Level + or -:

Use this function to choose the level for expressing milk. Depending on the selected mode, there are 1-9 levels available. Each level increases the suction strength. (Always start at level 1 and gradually increase to the desired level.)

Power On/Off & Mode Selection:

Press and hold the button for 2 seconds to turn on the breast pump. Press the button again to select the mode. Hold the button for 2 seconds to turn off the breast pump.

USB Port:

Use the provided charger to charge the breast pump. Insert and remove the charger carefully. Do not apply excessive force, as it may cause damage.

Memory Mode:

The breast pump remembers the last selected level for each mode. When switching between modes, the breast pump will continue at the last stopped level.

Expression/Pump Mode (1 - 9 levels): 

Use the expression/pump mode for complete milk expression. Each level offers different suction strengths during pumping.

2-in-1 Mode (1 - 9 levels): 

This mode can be very helpful for sensitive breasts and supports efficient milk expression in a short time.

Double Frequency Mode (1 - 9 levels): 

This mode can be very helpful for flat or inverted nipples.

Note: Gradually increase the levels in each mode. Start at level 1 and increase to the desired level.

Breast Shield Reducers:


Choosing the correct breast shield funnels is essential for optimal results. They ensure a proper fit and prevent leakage. The overview below will help making the right choice.

Nipple diameter for expressing milk (mm)	Nipple diameter after expressing milk(mm)	Suggested size (mm)
17-18-19	18-19-20	21mm
20-21-22	21-22-23	24mm
24-25-26	25-26-27	28mm

Placement of the breastpump

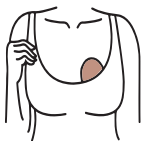Lean slightly forward and center the nipple in the breast pump. Ensure that the breast pump fits snugly against the breast.

Once the breast pump is in position, remain upright with your upper body. Ensure that the pump is positioned higher, and then begin expressing milk.

Once you're done expressing milk, lean forward to detach the breast pump. Lean slightly forward to avoid spilling any milk.

Bra strap

Adjust the length of the bra straps to customize the fit. This can make the expressing process more comfortable.

The end of the bra strap can be attached to the nursing bra.

The length of the bra strap can be adjusted by pulling it tighter.

TIPS

- Use the breast pump in a comfortable place. A comfortable setting simplifies the expressing process and can increase milk yield.
- Keep the breast pump clean and free from milk residues. This promotes the longevity and quality of expressing.
- Ensure you have the correct breast shield reducer. The right fit can increase milk yield.
- Carefully read the instruction manual. This ensures the breast pump is installed correctly.
- Always lean slightly forward or sit upright. This reduces the chance of milk getting into the motor.
- Keep all the extra bags and packaging from the package. You can use them to store the components when you're not using the breast pump for an extended period.
- Massage your breasts shortly before expressing. This can increase milk yield and provide a more comfortable feeling during expressing.
- Gradually increase the levels in each mode. Start at level 1 and increase to the desired level.

FAQ

Q: My breast pump has low suction power?

A: The gas storage and pumping unit might not be properly installed. When installing the pump into the gas storage, make sure to push it all the way in.

A: The duckbill valve is not installed correctly or is damaged. Check the component for proper placement and possible damage.

Q: The milk flows back.

A: If the gas storage is damaged or deformed, milk may flow back into the pump. In such a case, immediately stop using it. Replace the gas storage to restore the connection.

A: The duckbill valve may have a small crack, allowing milk to flow back. Inspect the valve for cracks, damage, and proper placement.

Q: Expressing is painful/not successful.

A: You might have breast engorgement or a blocked milk duct. Consult a doctor for breast massage.

A: Ensure you have the correct breast shield reducer. Choosing the right size is essential for a proper fit.

A: The pumping level used may be too strong. Start the pumping session at a lower level and gradually increase.

Q: Milk remains in the breast shield.

A: Check if you are using the correct breast shield reducer. Sometimes, using a size too small or too large can impede proper milk flow.

Q: My breast pump doesn't charge properly?

A: Ensure that you are using the provided USB cable. Connect it to a suitable adapter with the correct voltage.

CLEANING & MAINTENANCE

Non-washable parts:

The engine should never come into contact with moisture, water, or milk residues. The pump must remain dry. You can use a clean, dry cloth to wipe the main pumping unit. Keep this area dry and free from residues. Any other cleaning method outside of these instructions may void the warranty due to incorrect usage.

Washable parts:

Breast shield, collection tray, silicone gas storage, silicone duckbill valve, breast shield reducers. Cleaning the washable parts that come into contact with milk prevents the buildup of dry milk residues, which can promote the growth of harmful bacteria. This is essential for the safety of the product, you, and your baby.

After each use:

1. Soak the washable parts in a mild cleaning solution for 2 minutes and rinse under clean running water.
2. Clean immediately with drinking water or purified water.
3. Let the parts air dry or pat them dry with kitchen paper. This will prevent the release of particles that may be harmful.

Perform this once a day:

1. Boil the parts in water for five minutes, ensuring the water is deep enough to cover them completely. Be cautious of the stirring accessories during boiling to prevent distortion from uneven heat.
2. We do not recommend microwave or steam sterilization. Some steam and microwave sterilizers reach high temperatures that can cause plastic parts to melt.
3. We do not recommend ultraviolet sterilization.

Important

- Do not use chemicals, soaps, or any other substances that may contain harmful compounds. They can pose a risk to the product and the user's safety.
- Do not put the parts in the dishwasher. This may cause damage to the breastpump and is not covered under the warranty.
- Never let the engine come into contact with water or moisture. This can affect the motor's durability, and water or moisture damage is not covered under the warranty.
- Keep the engine free from dust and milk residues. This can also impact the motor's durability.
- Avoid dropping the breastpump. Impact damage can cause severe internal issues and may affect proper functionality. Damage caused by dropping is not covered under the warranty.
- Store the breastpump in a suitable place, away from moisture, water, or dust, and out of the reach of children. We recommend storing the pump in the product box.
- Always use the provided charger. Using a different charger may cause damage and is not covered under the warranty.

PRODUCT PARAMETERS

Model:	VG-598
Weight (1 pump):	300Gr
Product measurement:	11cm x 5.8cm x 14cm (LxWxH)
Nominal Power:	5W
Input:	5V $\overline{\text{---}}$ 1A
Suction Strength:	< 0.036 MPa
Battery Capacity:	1100 mAh
Modes:	3
Collection Tray Capacity:	170ML