EN **MILEX** a CooperSurgical Brand

LEVER Folding Pessaries

For stress uterine incontinence **R_xOnly**

AVAILABLE MODELS	
HODGE REF MXPEH <small>(Pessary Only)</small> MXKPEH <small>(Pessary, TRIMO-SAN™, Jel Jectör)</small>	HODGE WITH SUPPORT REF MXPHS <small>(Pessary Only)</small> MXKPHS <small>(Pessary, TRIMO-SAN™, Jel Jectör)</small>
HODGE WITH KNOB REF MXPEHK <small>(Pessary Only)</small> MXKPEHK <small>(Pessary, TRIMO-SAN™, Jel Jectör)</small>	HODGE WITH SUPPORT AND KNOB REF MXPHSK <small>(Pessary Only)</small> MXKPHSK <small>(Pessary, TRIMO-SAN™, Jel Jectör)</small>
SMITH REF MXPES <small>(Pessary Only)</small> MXKPRI <small>(Pessary, TRIMO-SAN™, Jel Jectör)</small>	RISSER REF MXPKS <small>(Pessary Only)</small> MXKPRI <small>(Pessary, TRIMO-SAN™, Jel Jectör)</small>
 Single-Patient Use CE 0086	
Wash pessary with mild soap and thoroughly rinse prior to initial use.	

DESCRIPTION

The Milex® LEVER Pessaries are a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

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CooperSurgical

WARNINGS

- These pessaries contain a wire coil which must be removed before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.

INDICATIONS FOR USE

- Stress urinary incontinence¹
- For uterine repositionment with accompanying symptoms of backache and/or dysmenorrhea². The use of a HODGE pessary is sometimes recommended to manage an incompetent cervix².

OTHER INDICATIONS FOR USE³

- To hasten postpartum involution, infertility, prolapse ovary
- Incompetent cervix

CONTRAINDICATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient

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² Reich, W.J., Nechtow, M.J. Practical Gynecology 2nd Edition, pp 345-355

³ Oster, S. Javert, K.T., Treatment of incompetent cervix with the SMITH Pessary. Obstet and Gynec. 28: 206-208.

RECOMMENDED CLEANING INSTRUCTIONS*

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak and Scrub
 - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
 - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
 - Following scrub, inspect device for visible soil residue.
- Rinse
 - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
 - Allow the device to dry.

*New pessaries are powdered with food-grade powder.

PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

- To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner. It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge

- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out

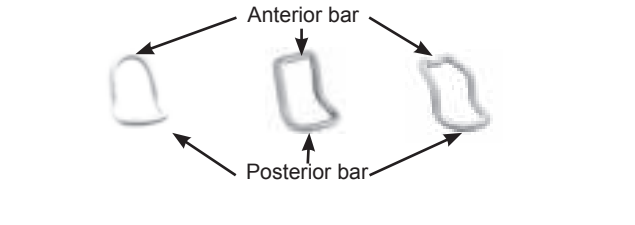
For medical emergencies and for all medically related advice, consult your healthcare professional.

FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL


Review these instructions with the patient to establish use regimen.

INSTRUCTIONS

MANUAL ELEVATION OF RETRODISPLACED UTERUS

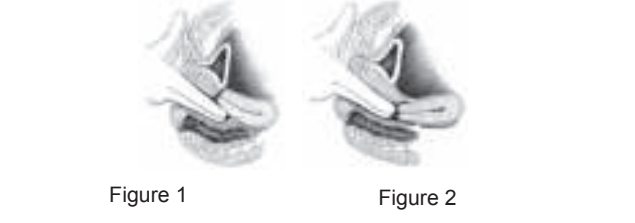- With two fingers in the vagina, the corpus is gently but firmly pushed back out of the cul-de-sac (see Figure 1).
- Bring the uterus into or above the long axis of the vagina (see Figure 2).

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CooperSurgical

 **Figure 3**  **Figure 4**

- The hand on the abdomen feels the corpus and tends to rotate the uterus anteriorly (see Figure 3).

- Pressure on the corpus with the fingers in the vagina is continued until the hand on the abdominal wall can maintain uterus in position for necessary insertion (see Figure 4). If manual elevation is difficult, then try the knee-chest position.

INSTRUCTIONS

- Wear dry gloves. When necessary, lubricate only the entering end of the pessary with TRIMO-SAN.
- Use one finger to press the perineum. Fold the pessary by bringing the sides together with the posterior bar curved upwards (see Figures 5 and 6).

 **Figure 5**  **Figure 6**

- The index finger is inserted deep into the vagina to depress the posterior bar of the pessary down and behind the cervix (see Figure 7).
- The anterior bar or knob is now brought up and tucked behind the symphysis pubis.
- Upon proper placement, the cervix spontaneously falls within the pessary (see Figure 8).

 **Figure 7**  **Figure 8**

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CooperSurgical

 **Figure 7**  **Figure 8**

- If the patient is comfortable with the pessary in position, have her go to the bathroom to empty her bladder. If she can do so without difficulty, and is comfortable with the pessary in place, and the pessary remains in position upon re-examination, this is a good indication that the correct size may have been selected. Prolapse reduction may resolve cervical vaginal irritation. Verification of cervical cytology (pap) and or biopsy as clinically indicated.

Note: If the patient is unable to urinate with the pessary in position, remove it and fit her with the next smaller size. Repeat as necessary.

- If the patient can void without difficulty, and the pessary remains in position upon re-examination, and the patient is comfortable with the pessary in place, this is a good indication that the correct size may have been selected. Patient experience may vary.

- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.
- The healthcare professional should be able to sweep one finger between the pessary and the vaginal walls. If there is not enough space to do this, the next smaller size should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.

- It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged.

- Ulcerations and erosions frequently occur in cases of complete prolapse due to irritation of the exteriorized cervix or vaginal wall. Whenever possible, reducing the mass and treating the irritation are primary steps before using a pessary. Prolapse reduction may resolve cervical vaginal irritation. Verification of cervical cytology (pap) and or biopsy as clinically indicated.

- During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning douching, discharge, disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary.
- At the physician's discretion, the patient can be instructed in the proper removal, cleaning and reinsertion techniques for her own pessary. This process can be performed nightly or even weekly by the patient under ideal circumstances.

TO REMOVE

Use one finger to depress the perineum. Hook the finger of the other hand under the anterior bar or knob and press down. Fold the pessary and gently ease it out.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

Note: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

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