EN

LEVER Folding Pessaries
For stress uterine incontinence

R_xOnly

AVAILABLE MODELS

<p>HODGE</p> <p>[REF] MXPEH (Pessary Only) MXKPEH (Pessary, TRIMO-SAN™, Jel Jector)</p>	<p>HODGE WITH SUPPORT</p> <p>[REF] MXPHS (Pessary Only) MXKPHS (Pessary, TRIMO-SAN™, Jel Jector)</p>
<p>HODGE WITH KNOB</p> <p>[REF] MXPEHK (Pessary Only) MXKPEHK (Pessary, TRIMO-SAN™, Jel Jector)</p>	<p>HODGE WITH SUPPORT AND KNOB</p> <p>[REF] MXPHSK (Pessary Only) MXKPHSK (Pessary, TRIMO-SAN™, Jel Jector)</p>
<p>SMITH</p> <p>[REF] MXPES (Pessary Only) MXKPRI (Pessary, TRIMO-SAN™, Jel Jector)</p>	<p>RISSER</p> <p>[REF] MXPRI (Pessary Only) MXKPRI (Pessary, TRIMO-SAN™, Jel Jector)</p>

Single-Patient Use **CE0086**

Wash pessary with mild soap and thoroughly rinse prior to initial use.

DESCRIPTION

The Millex® LEVER Pessaries are a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

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CooperSurgical

WARNINGS

- These pessaries contain a wire coil which must be removed before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.

INDICATIONS FOR USE

- Stress urinary incontinence¹
- For uterine repositioning with accompanying symptoms of backache and/or dysmenorrhea². The use of a HODGE pessary is sometimes recommended to manage an incompetent cervix².

OTHER INDICATIONS FOR USE³

- To hasten postpartum involution, infertility, prolapse ovary
- Incompetent cervix

CONTRAINDICATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge

RECOMMENDED CLEANING INSTRUCTIONS¹

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak and Scrub
 - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
 - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
- Rinse
 - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
 - Allow the device to dry.

¹*New pessaries are powdered with food-grade powder.

PATIENT INSTRUCTIONS FOR PESSARY USE

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

- To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner. It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge

REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

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- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out

For medical emergencies and for all medically related advice, consult your healthcare professional.

FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

INSTRUCTIONS

MANUAL ELEVATION OF RETRODISPLACED UTERUS

- With two fingers in the vagina, the corpus is gently but firmly pushed back out of the cul-de-sac (see Figure 1).
- Bring the uterus into or above the long axis of the vagina (see Figure 2).

INSTRUCTIONS

- Wear dry gloves. When necessary, lubricate only the entering end of the pessary with TRIMO-SAN.
- Use one finger to press the perineum. Fold the pessary by bringing the sides together with the posterior bar curved upwards (see Figures 5 and 6).
- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.
- The healthcare professional should be able to sweep one finger between the pessary and the vaginal walls. If there is not enough space to do this, the next smaller size should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.
- It is sometimes necessary to re-fit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged.

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FIGURE 10

EXPLANATION OF SYMBOLS

[REF]	Reorder number
[LOT]	Batch code
	Use-by date
	Consult instructions for use
	Not made with natural rubber latex
R_xOnly	CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician
[EC REP]	Authorized Representative in the European Community.
	Manufacturer

Dawn® is a registered trademark of Procter & Gamble, Cincinnati, Ohio 45202
TRIMO-SAN™ is a trademark of CooperSurgical, Inc.
Millex® and CooperSurgical are registered trademarks of CooperSurgical, Inc.
© 2017 CooperSurgical, Inc.

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TRIMO-SAN™ es una marca comercial de CooperSurgical, Inc.
Millex® y CooperSurgical son marcas comerciales registradas de CooperSurgical, Inc.