Digitale Kalenderklok

Handleiding

MENU

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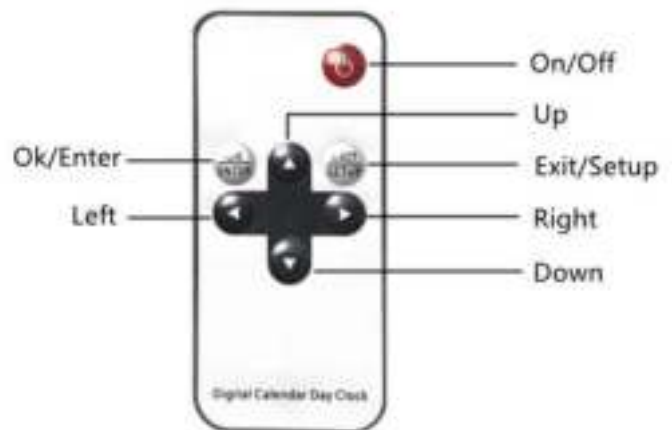
Set date: To set the desired date

Time Mode: 12-Hour or 24-Hour Display

Date mode: Desired date display setting

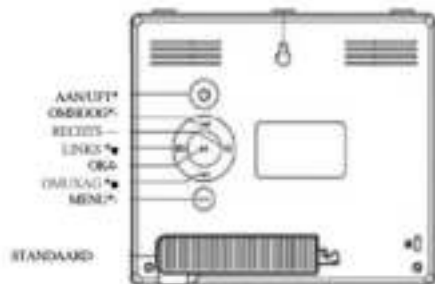
Language: Choose your preferred language

Interface Duration: This setting will display for example: "IT IS FRIDAY AFTERNOON"

You can rotate these up to 30 seconds! When set to '0 sec', this setting is not displayed.

(NOTE, this setting is NOT soothing for many people, that's why this setting is turned off by default with us)

Important Note: the buttons on the back of your clock are labeled with symbol only, please use the clearly labeled photo illustration below to identify the appropriate button functions to set your clock.

Settings for Time, Date, Medication Reminders, and Alarm:

STEP 1 Press the "Menu" button to enter the settings screen.

STEP 2 Use the "UP" and "DOWN" buttons to select the desired setting.

STEP 3 To adjust the desired setting press "OK", it will then change color.

STEP 4 With the buttons "UP" and "DOWN" you can make the desired setting.

(STEP 4B For example, to switch from hours to minutes, press the "RIGHT" button and proceed to STEP 4)

STEP 5 When you're done with that, press "OK"

STEP 6 Press the "MENU" button to return to the clock screen.

Adjust other settings:

STEP 1 Press the "Menu" button to enter the settings screen.

STEP 2 Use the "UP" and "DOWN" buttons to select the desired setting.

STEP 3 By pressing the buttons "RIGHT" or "LEFT" you can make the desired setting.

STEP 4 Press the "MENU" button to return to the clock screen.

(Note to set Time, Date, Medication Reminders, and Alarm follow the settings for this)

Remote Control: (option)

To make several modifications to set up screen, after each change simply press **DOWN** button to highlight the next field that you would like to modify and follow the instructions above.

After all the modifications are completed, press the **MENU** button to go back to the main screen.