## LEVER Folding Pessaries

For stress uterine incontinence

Rx Only

### AVAILABLE MODELS

HODGE REF: MXPEH (Pessary Only)	HODGE WITH SUPPORT REF: MXPFS (Pessary, TRIMO-SAN™, Jel Jector)
HODGE WITH KNOB REF: MXPKH (Pessary Only)	HODGE WITH SUPPORT AND KNOB REF: MXPFSK (Pessary Only)
SMITH REF: MXPES (Pessary Only)	RISSEK REF: MXPRI (Pessary Only)
MXKPEH (Pessary, TRIMO-SAN™, Jel Jector)	MXKPKS (Pessary, TRIMO-SAN™, Jel Jector)
Single-Patient Use	€ 0086

Wash pessary with mild soap and thoroughly rinse prior to initial use.

### DESCRIPTION

The Milex® LEVER Pessaries are a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

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Rx Only

### BESCHIKBARE MODELLEN

HODGE REF: MXPEH (Alleen pessarium)	HODGE MET STEUN REF: MXPFS (Alleen pessarium)
MXKPEH (Pessarium, TRIMO-SAN™, Jel Jector)	MXKPKS (Pessarium, TRIMO-SAN™, Jel Jector)
HODGE MET KNOOP REF: MXPKH (Alleen pessarium)	HODGE MET STEUN EN KNOOP REF: MXPFSK (Alleen pessarium)
MXKPKH (Pessarium, TRIMO-SAN™, Jel Jector)	MXKPKSK (Pessarium, TRIMO-SAN™, Jel Jector)
SMITH REF: MXPES (Alleen pessarium)	RISSEK REF: MXPRI (Alleen pessarium)
MXKPKES (Pessarium, TRIMO-SAN™, Jel Jector)	MXKPKRS (Pessarium, TRIMO-SAN™, Jel Jector)

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CoperSurgical

### WARNINGS

- These pessaries contain a wire coil which must be removed before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.

### INDICATIONS FOR USE

- Stress urinary incontinence.<sup>1</sup>
- For uterine retrodisplacement with accompanying symptoms of backache and/or dysmenorrhea. The use of a HODGE pessary is sometimes recommended to manage an incompetent cervix.<sup>2</sup>

### OTHER INDICATIONS FOR USE<sup>3</sup>

- To hasten postpartum involution, infertility, prolapse ovary
- Incompetent cervix

### CONTRAINdicATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient

### RECOMMENDED CLEANING INSTRUCTIONS\*

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak and Scrub
  - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
  - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
  - Following scrub, inspect device for visible soil residue.
- Rinse
  - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
  - Allow the device to dry.

\*New pessaries are powdered with food-grade powder.

### PATIENT INSTRUCTIONS FOR PESSARY USE

#### CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

- Desire the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner. It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

### REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge

<sup>1</sup> Miller D.S.: Contemporary use of the Pessary. Gynecology & Obstetrics, JJ Sciarra & Wm Drogemuller, JB Lippincott Co, Vol 1, 1992 pp 8-9

<sup>2</sup> Reich, W.J., Nechtow, M.J. Practical Gynecology 2nd Edition, pp 345-355

<sup>3</sup> Oster, S., Javert, K.T., Treatment of incompetent cervix with the SMITH Pessary. Obstet and Gynec. 28: 206-208.

- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out


For medical emergencies and for all medically related advice, consult your healthcare professional.

### FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

### INSTRUCTIONS

#### MANUAL ELEVATION OF RETRODISPLACED UTERUS

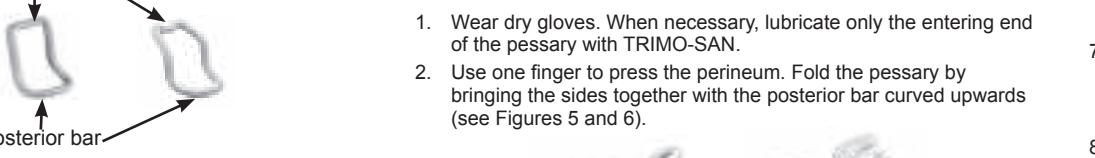- With two fingers in the vagina, the corpus is gently but firmly pushed back out of the cul-de-sac (see Figure 1).
- Bring the uterus into or above the long axis of the vagina (see Figure 2).

Figure 1      Figure 2

<sup>1</sup> Miller D.S.: Contemporary use of the Pessary. Gynecology & Obstetrics, JJ Sciarra & Wm Drogemuller, JB Lippincott Co, Vol 1, 1992 pp 8-9

<sup>2</sup> Reich, W.J., Nechtow, M.J. Practical Gynecology 2nd Edition, pp 345-355

<sup>3</sup> Oster, S., Javert, K.T., Treatment of incompetent cervix with the SMITH Pessary. Obstet and Gynec. 28: 206-208.

Figure 3

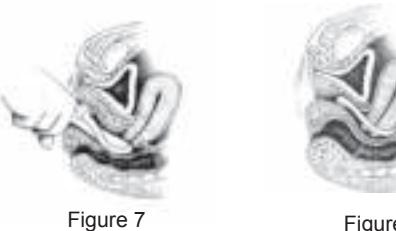
Figure 4

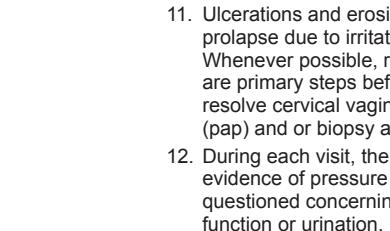
Figure 5

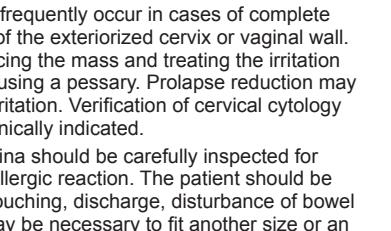
Figure 6

- The hand on the abdomen feels the corpus and tends to rotate the uterus anteriorly (see Figure 3).
- Pressure on the corpus with the fingers in the vagina is continued until the hand on the abdomen can maintain uterus in position for necessary insertion (see Figure 4). If manual elevation is difficult, then try the knee-chest position.

### INSTRUCTIONS

#### REMOVING THE PESSARY

Use one finger to depress the perineum. Hook the finger of the other hand under the anterior bar or knob and press down. Fold the pessary as necessary.

**RECOMMENDED FOLLOW-UP**

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

**Note:** The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

- If the patient is comfortable with the pessary in position, have her go to the bathroom to empty her bladder. If she can do so without difficulty, and is comfortable with the pessary in place, and the pessary remains in position upon re-examination, this is a good indication that the correct size may have been selected. Patient experience may vary.
- If the patient can void without difficulty, and the pessary remains in position upon re-examination, this is a good indication that the correct size may have been selected. Patient experience may vary.
- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel any discomfort as it may turn or be expelled and it should not be too tight as it may cause discomfort.
- The healthcare professional should be able to sweep one finger between the pessary and the vaginal walls. If there is not enough space to do this, the next smaller size should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.
- It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged.

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45202 CooperSurgical, Inc., Proctor & Gamble, Scicarra, Wm Droege, Dawn®. CooperSurgical, Inc. TRIMO-SAN™. CooperSurgical, Inc. Milex®. © 2017 CooperSurgical, Inc.