



3 Key Turner



VM932A

Usage and Maintenance Instructions

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

Designed for people who have a problem handling small door keys, this key turner has a large handle for easy gripping and turning.

INSTRUCTIONS FOR USE

The Key Turner is simple to use:

- Use the locking lever to release the central post to insert up to 3 Yale-type keys.
- Turn the locking lever back to tighten the central post and hold the keys in place.
- The keys can be folded back into the handle of the Key Turner or rotated out to use the key as required.

CLEANING

Clean your Key Turner using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9
3EX**

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 •

**Web: www.aidapt.com
Email: sales@aidapt.co.uk**