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Z 08165 M DS V3 0718

Dear customer,
We are delighted that you have decided to purchase our **alcohol tester**.
If you have any questions, please contact customer service via our website:

www.service-shopping.de

Proper use

- The device measures the approximate concentration of alcohol in your breath and calculates the approximate concentration of alcohol in your blood. No guarantee is offered for any of the measurements taken with this device.
- The device is designed for private, not for commercial use. Only use the device for the intended purpose and only as described in the instructions. Any other use is deemed improper.


Safety notes

- **The alcohol tester cannot be used with binding legal effect to determine whether a person, after consuming alcohol, is able to drive a vehicle, operate a machine or perform any other action posing a risk to oneself or other persons under the influence of alcohol. Improper handling or manipulation of the device may cause it to display lower alcohol levels! Please keep in mind that the consumption of alcohol always impairs your responsiveness and judgement.**
- Legal provisions for road traffic in your respective country of residence must be observed and complied with. However, we wish to expressly point out that the measurements displayed by the alcohol tester offer no indication as to whether the blood alcohol level of the user is within the legally prescribed blood alcohol limits for road traffic.
- Do not blow cigarette smoke onto the sensor.
- Batteries may be life threatening if swallowed. Therefore keep the batteries and products out of the reach of children and animals. If a battery has been swallowed, you must seek immediate medical attention.
- Before inserting the batteries, check whether the contacts in the battery compartment and on the batteries are clean and, if necessary, clean them.
- Only use the battery type specified in the technical data (see "Technical data").
- Always replace all batteries. Do not use different battery types, brands or batteries of different capacitances. When changing batteries ensure correct polarity (+/-).
- Remove the batteries from the battery compartment if they are empty or if you are not using the product. This way, you can avoid damages attributed to leaking.
- If a battery has leaked, avoid contact between your skin, eyes and mucous membranes and the battery acid. In the event of contact with battery acid, rinse the affected areas with clean water and immediately consult a physician.
- Immediately remove any leaking batteries from the battery compartment. Clean the contacts before you insert a new battery.

- The batteries may not be charged or reactivated with other means and they may not be disassembled, thrown in a fire, submerged in a liquid or short-circuited.
- Do not allow the product to fall and do not subject it to severe percussions.
- Keep away from fire and heat sources.
- Ensure that the product cannot get wet or fall in water or other liquids.
- If the product is defective, do not attempt to repair it yourself. In the event of damage, contact the customer service department.
- Use a dry cloth to clean the sensor / device every time after use.
- Do not use any chemicals or aggressive cleaners to clean the device.
- Do not subject the device to extreme temperatures, direct sunlight, severe temperature fluctuations or excessive moisture.
- Store the device in an area that is out of the sun and cold.
- Store the device out of the reach of children and pets.

Product overview and product contents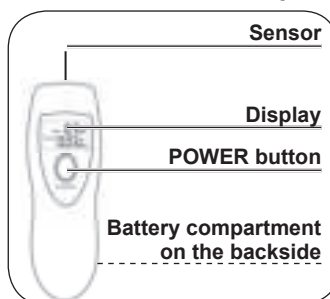

Product contents:

- 1 x alcohol tester
- 2 x 1.5 V batteries, type AAA
- 1 x hand strap

Before first use

- Unpack all parts and make sure that the product contents are complete (see "Product contents") and check for any damages from transport. If you identify damages to the components, do not use them (!), instead contact our customer service centre.
- Insert the enclosed batteries in the device (see "Changing the batteries").
- Optional: Attach the hand strap to the loop on the top part of the device.

Changing the batteries

1. Open the battery compartment on the back side by pushing it open.
2. Remove empty batteries and dispose of them in an environmentally-friendly manner.
3. Insert two 1.5 V batteries (type AAA) in the battery compartment. Make sure the polarity (+/-) is correct.
4. Close the battery compartment by pushing the battery compartment cover onto the battery compartment.

Use

ATTENTION

- Please keep in mind that the consumption of alcohol always impairs your responsiveness and judgement! Even if the device does not detect any alcohol or only a low level after alcohol consumption, you may not, drive a vehicle, operate a machine or perform an action, which, when performed under the influence of alcohol, could endanger yourself or other persons!
- Please keep in mind that improper handling or manipulation of the device may cause it to display lower alcohol levels. No guarantee is offered for any of the measurements taken with this device!
- Please keep in mind that alcohol affects different persons differently. Depending on height, body weight, age and health, the effects of the same blood alcohol level may vary from person to person.


After the measurement, the device will display the calculated **BAC** in grams per litre and per cent. **BAC** stands for "blood alcohol content" as calculated by the device. Example: "0.3 g/l (or 0.03 % BAC)" corresponds to an alcohol content of 0.3 g per litre of blood. **NOTICE:** The actual BAC may differ from the value shown!


1. Use a dry cloth to wipe the sensor to remove any contamination that could affect the test result.
2. Push the **Power** button until you hear a signal tone. The display will switch on.

3. A "Warm up" countdown from 10 to 0 will appear on the display. Wait until the countdown is complete. A signal will sound once the countdown is complete.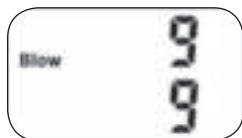

4. After the "Warm up", the "Blow" icon will appear on the display. A new countdown from 10 to 0 will appear. The alcohol concentration in your breath can be measured during this countdown.

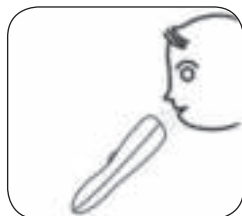5. During the "Blow" countdown, hold the sensor up to your mouth and blow forcefully into the sensor for approx. 3 – 5 seconds.

6. The "Caution" icon will appear if blood alcohol levels of 0.02 % BAC (or 0.2 g/l) to 0.05 % BAC (or 0.5 g/l) are detected.

7. The "Danger" icon will appear if a blood alcohol level of 0.05 % BAC (or 0.5 g/l) and above is detected.

8. The device will automatically switch off after a few seconds. A signal will sound when the device switches off.

Tips / notes:

- At least 20 minutes should have passed since you last consumed alcohol and before the current measurement. Do not eat, smoke or drink during this time.
- Closed, poorly ventilated rooms as well as air-conditioned rooms may affect the accuracy of the reading. If necessary, measure your BAC 2 – 3 times.
- Saliva and alcohol on the sensor may affect the accuracy of the measurements. Use a dry cloth to clean the sensor/device every time after use.
- If you have not used the device for a prolonged period of time (or before first use), you must perform 2 – 3 test measurements before you perform the actual measurement.
- Wait approximately 2 – 3 minutes between two measurements.
- Low batteries may affect the accuracy of measurements. Change the batteries when you notice that the battery power is diminishing.

Technical data

Model:	AT-642
Voltage supply:	3 V DC (2x 1.5 V battery, type AAA)
Measurement range:	0.00 – 0.19 % BAC or 0.0 – 1.9 g/l
Measuring tolerance:	0.01% BAC (0.1 g/l)
Warm-up phase:	approx. 10 seconds
Storage conditions:	Temperature range from 0 to 40 °C
Operating conditions:	Temperature range from 0 to 40 °C
Sensor:	Semiconductor sensor

Disposal

The packaging material is recyclable. Dispose of the packaging in an environmentally-friendly manner and make it available to the collection service for recyclable materials.

Dispose of the product in an environmentally-friendly manner. The product does not belong in the household garbage. Dispose of it at a recycling centre for old electric and electronic equipment. For more information, please contact the administration in your community.

Rechargeable batteries and batteries must be removed and disposed of separately before disposing of the product. To protect the environment, batteries and rechargeable batteries may not be disposed of together with normal household waste. Instead they must be turned in at appropriate collection points. Please also observe applicable statutory provisions regulating the disposal of batteries.

Customer Service / Importer:

DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Germany • ☎ +49 38851 314650 *)

*) Calls to German landlines are subject to charges. The cost varies depending on the service provider.

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All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Duitsland • ☎ +49 38851 314650 *)