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Inhoud verpakking: 1 x Hemorcool® in koker, 1 x vaseline sachet

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Made in NL

Manufacturer
Pita Medical B.V.
T + 31 (0) 6 27 25 44 97
www.hemorcool.com

The Hemorcool™ is an anal cooling device that can be used for treatment of haemorrhoids and anal thrombosis.

The cold Hemorcool™ stick causes the blood vessels to contract, so that the blood flow to the swollen tissues diminishes.

As a result of this the haemorrhoids will shrink and the anal oedema decreases. Complaints like pain, itching, burning and loss of blood will be reduced or even disappear.


The Hemorcool™ anal device has been clinically tested on more than fifteen thousand patients over the last ten years.

The results were amazing without any side-effects.

Because of its use as a medical device, the Hemorcool™ has to comply to the standards 93/42/EEG of the European Commission.

*** Directions for use:**

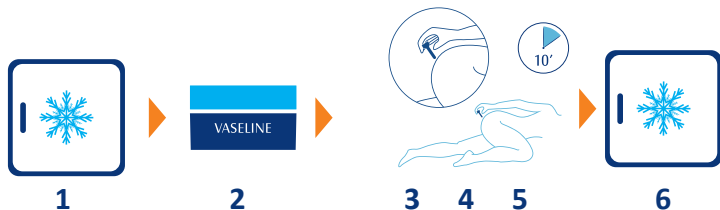1. Place the Hemorcool™ in the freezing compartment of the refrigerator for at least 2 hours.
2. Take the Hemorcool™ out of the freezer and put some vaseline on the stick and around the anus.
3. Lie on your side and insert the thinner portion of the Hemorcool™ stick into the anus (up until the broader side of the stick).
4. Let the Hemorcool™ stick remain inside the anus for just about ten minutes.
5. Remove the Hemorcool™ stick from the anus and clean it with water and soap.
6. Put the Hemorcool™ stick inside the box and leave it in the freezing compartment of the refrigerator for at least 2 hours. The Hemorcool™ will be ready for use again.

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